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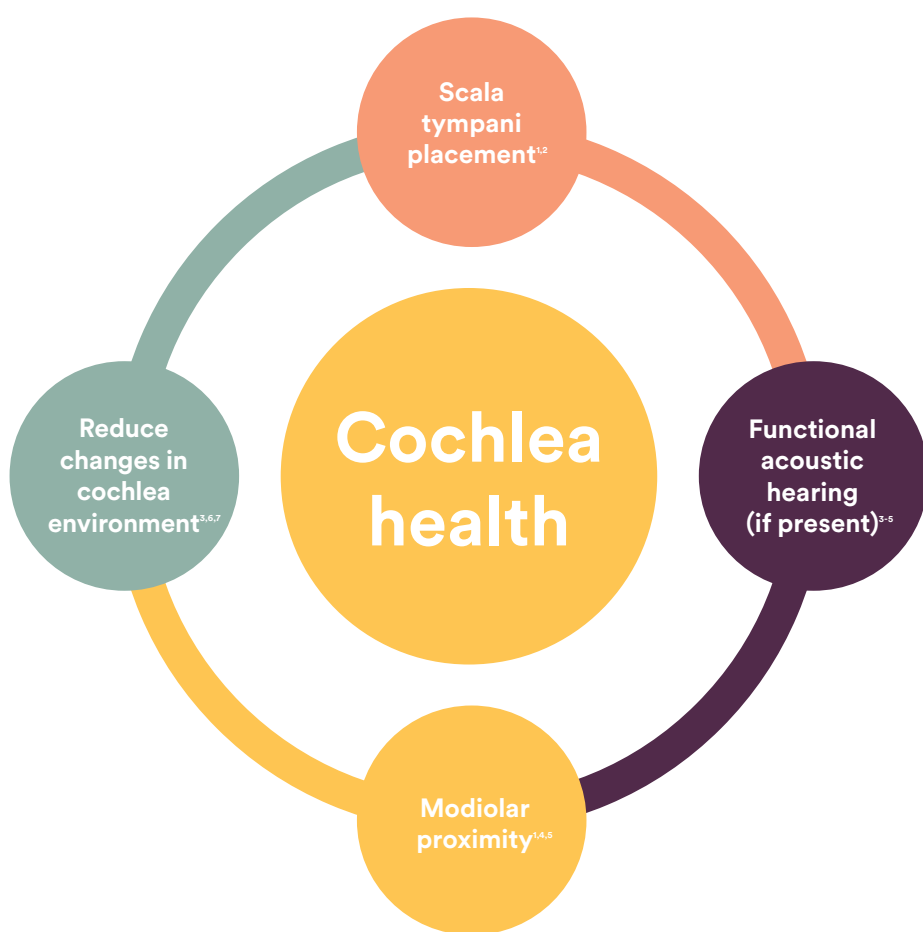
Referências

1. Holden L, et al. Factors Affecting Open-Set Word Recognition in Adults With Cochlear Implants. *Ear & Hearing*. 2013;34(3):342-360
2. Shaul C, et al. Scalar localisation of peri-modiolar electrodes and speech perception outcomes. *The Journal of Laryngology & Otology*. 2018;132(11):1000-1006.
3. Adunka O, et al. Minimum Reporting Standards for Adult Cochlear Implantation. *Otolaryngology–Head and Neck Surgery*. 2018;159(2):215-219.
4. Pflugst B, et al. Importance of cochlear health for cochlear-implant function. *The Journal of the Acoustical Society of America*. 2015;137(4):2297-2297.
5. Somdas M, et al. Quantitative Evaluation of New Bone and Fibrous Tissue in the Cochlea following Cochlear Implantation in the Human. *Audiology and Neurotology*. 2007;12(5):277-284.
6. Scheperle R, et al. Delayed changes in auditory status in cochlear implant users with preserved acoustic hearing. *Hearing Research*. 2017;350:45-57.
7. Bester C, et al. Four-point impedance as a biomarker for bleeding during cochlear implantation. *Scientific Reports*. 2020;10(1).
8. Berg K, et al. Speech recognition as a function of the number of channels in perimodiolar electrode recipients. *The Journal of the Acoustical Society of America*. 2019;145(3):1556-1564.
9. Croghan N, et al. Erratum: Re-examining the relationship between number of cochlear implant channels and maximal speech intelligibility [J. Acoust. Soc. Am. 142(6), EL537–EL543 (2017)]. *The Journal of the Acoustical Society of America*. 2018;143(5):2621-2621.
10. Buchman CA, et al. Unilateral cochlear implants for severe, profound, or moderate sloping to profound bilateral sensorineural hearing loss. *JAMA Otolaryngology–Head & Neck Surgery*. 2020;146(10):942.
11. Woodson E, et al. Initial hearing preservation outcomes of cochlear implantation with a slim Perimodiolar Electrode Array. *Cochlear Implants International*. 2021;22(3):148–56.
12. Kopelovich JC, et al. Risk Factors for Loss of Ipsilateral Residual Hearing After Hybrid Cochlear Implantation. *Otology & Neurotology*. 2014 Sep;35(8):1403–8.
13. Carlson M, et al. Survey of the American Neurotology Society on Cochlear Implantation: Part 2, Surgical and Device-Related Practice Patterns. *Otology & Neurotology*. 2018;39(1):e20-e27.
14. Bierer JA. Probing the electrode-neuron interface with focused cochlear implant stimulation. *Trends in Amplification*. 2010;14(2):84-95.
15. Ramos de Miguel Ángel, et al. Imaging evaluation of electrode placement and effect on electrode discrimination on different cochlear implant electrode arrays. *European Archives of Oto-Rhino-Laryngology*. 2018;275(6):1385-94.
16. Holder JT, et al. Matched Cohort Comparison Indicates Superiority of Precurved Electrode Arrays. *Otology & Neurotology*. 2019;40(9):1160-7.
17. Verberne J, et al. The Effect of Scala Tympani Morphology on Basilar Membrane Contact With a Straight Electrode Array: A Human Temporal Bone Study. *Otology & Neurotology*. 2017;38(1):47-3.
18. Kamakura T, Nadol JB Jr. Correlation between word recognition score and intracochlear new bone and fibrous tissue after cochlear implantation in the human. *Hearing research*. 2016;339:132-41.
19. Pflugst B, et al. Importance of cochlear health for cochlear-implant function. *The Journal of the Acoustical Society of America*. 2015;137(4):2297-2297.
20. Wright CG, Roland PS. Vascular trauma during cochlear implantation: a contributor to residual hearing loss? *Otology & Neurotology*. 2013;34(3):402-7.
21. Danielian A, et al. Morphometric linear and angular measurements of the human cochlea in implant patients using 3-dimensional reconstruction. *Hear Res*. 2020;386:107874
22. Cochlear Limited. D1655106 Competitive Comparison of Implant Intracochlear Electrode Thickness. September 2021.
23. Aschendorff A, et al. Clinical investigation of the Nucleus Slim Modiolar Electrode. *Audiology & Neurotology* 2017;22:169-179
24. Ramos-Macias A, et al. Hearing Preservation with the Slim Modiolar Electrode Nucleus CI532® Cochlear Implant: A Preliminary Experience. *Audiology & Neuro-otology*. 2018 27;22(6):317-325.
25. De Seta D, et al. Damage to inner ear structure during cochlear implantation: Correlation between insertion force and radio histological findings in temporal bone specimens. *Hearing research*. 2017; 344: 90-7.

 **Cochlear Ltd** (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia T: +61 2 9428 6555 F: +61 2 9428 6352
Cochlear Latinoamérica International Business Park, Building 3835, Office 403, Panama Pacifico, Panama Teléfono: +507 830 6220
Cochlear Colombia Avenida Carrera 19 # 100-28, piso 2, Centro de Experiencia Cochlear, Bogota D.C., Colombia Teléfono: (+57) 60-1-580-7013
Cochlear México Av. Insuburgentes Sur 1196, Cp 03200, Oficina 1401 Torre de los Parques, Tlacoquemécatl del Valle de la Ciudad de México. Teléfono: +55 52414500

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, 콘트우아, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, SoundBand, True Wireless, o logótipo elíptico, Vistafix, Whisper, WindShield e Xidium são marcas comerciais ou marcas registradas do grupo de empresas Cochlear.

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© Cochlear Limited 2025. D2423004