

English



Cochlear™

Nucleus® CP810 Sound Processor
Quick Reference Guide

Hear now. And always



Cochlear®


Your sound processor




How to . . .

Replace zinc air batteries


①

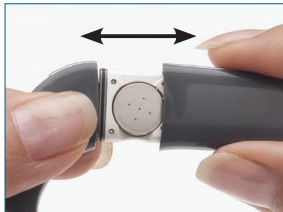

②

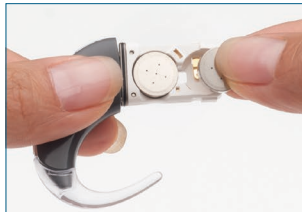

③

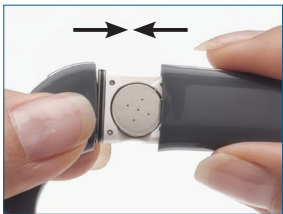

How to . . .

Detach the battery module

①



②



③



How to . . .

Turn your processor on and off



ON and OFF

Press and hold



OFF

Press and hold both buttons at the same time until you see the steady orange light

ON

Press and hold the lower button

Lock or unlock processor buttons



Quick press both buttons at the same time

Simple
Advanced



Quick press both buttons at the same time

How to . . .

Change programs



Quick press





Press and hold

Adjust volume or sensitivity



Simple option

Use the Remote Assistant.
Refer to the other side of this guide.

 Simple
 Advanced



Quick press to increase

Quick press to decrease

How to . . .

Switch to and from telecoil input



Quick press

Press and hold



Switch to and from accessory input *



Quick press

Press and hold







— Simple
— Advanced


* Connect the accessory before switching


Understanding the indicator lights


Light


What it means

-  You are turning on your processor OR
-  You are changing programs.
-  The number of flashes corresponds to the number of the selected program. For example, if you see three flashes,
-  you are using program three.






-  You are changing volume or sensitivity.
(Advanced option only.)

-  You are changing from microphones to telecoil or accessory.

-  You are changing from telecoil or accessory to microphones.

-  The microphones are capturing sound.

Understanding the indicator lights

Light	What it means
	You are locking your processor's buttons.
	You are unlocking your processor's buttons.
	The coil may be off OR your processor is not connected to the correct implant.
	Your processor's buttons are locked.
	Your processor's battery is empty.
	If you are pressing the lower button, you are turning off your processor OR There may be a fault with your processor. Contact your clinician.

For more information about your processor, refer to the CP810 Sound Processor User Guide.

Understanding private tones

Tone	What it means
Single high tone	You are changing volume or sensitivity. (Advanced option only.)
Short high tones	You are changing programs. The number of tones corresponds to the number of the selected program. For example, if you hear two tones, you have changed to program two.
Single long high tone	You are changing from microphones to telecoil or accessory OR You are changing from telecoil or accessory to microphones.
Single short low tone	Your processor's buttons are locked.
Two short low tones	Your processor's battery is low.
Sequence of short low tones	Your processor's battery is empty.
Sequence of long low tones	There may be a fault with your processor. Contact your clinician.

English



Cochlear™

Nucleus® CR110 Remote Assistant


Quick Reference Guide

Hear now. And always



Cochlear®

Your remote assistant

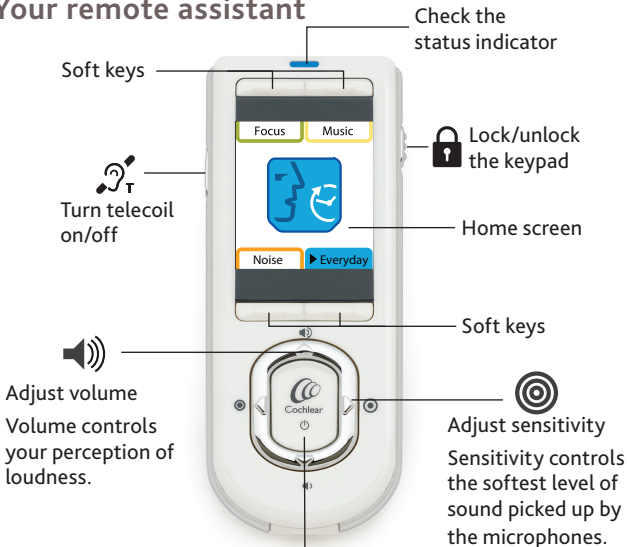


- Turn on/off (Press and hold)
- Check processor status
- Return Home

How to . . .

Select remote assistant language

After turning on remote assistant, language select screen displays if remote assistant is not paired.

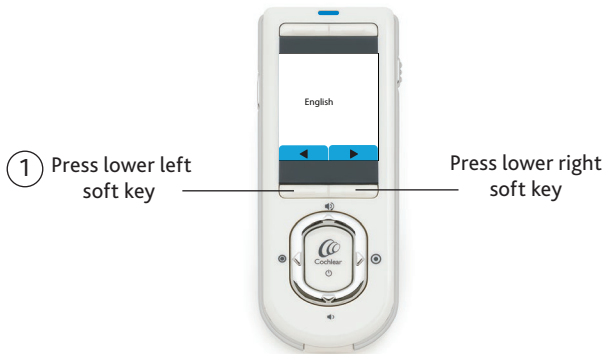

1 Press lower left soft key

Press lower right soft key


2 Now pair your remote assistant.

How to . . .

Pair your remote assistant and processor

① Make sure your processor and remote assistant are turned on.

② Place the coil on the back of your remote assistant.

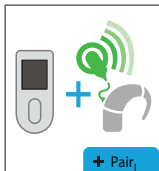

③ Press lower right soft key.


④ A pairing confirmation displays. ✓

Bilateral users: Repeat steps 2 and 3.

How to . . .

Select Simple or Advanced mode

Your remote assistant is automatically in Simple mode when you turn on your remote assistant for the first time. You can access Advanced mode, by pressing the upper left soft key when the remote assistant battery status screen displays.

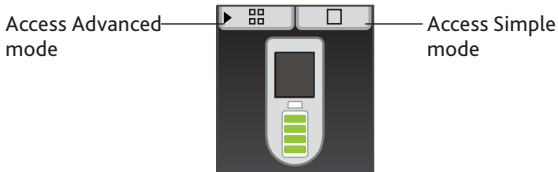

To switch between Advanced and Simple modes:


- 1 Turn off your remote assistant and turn it back on.
- 2 When the battery status screen displays, press the upper left soft key to select Advanced mode, or the upper right soft key to select Simple mode.

How to . . .

Check the status of your processor

Press the Cochlear key from the Home screen to check the status of your processor(s).

Processor
status is OK

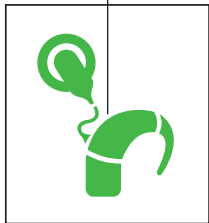

Processor
battery level




Audio input
level



How to . . .

Reset processor to values set by your clinician

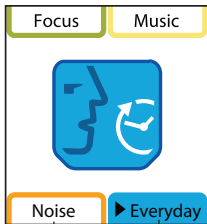


- 1 Press and hold both the lower left and lower right soft keys at the same time from the Home screen.



- 2 Press lower right soft key to confirm reset.


How to . . .

Turn on auto telecoil*

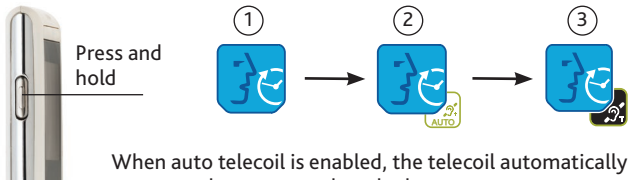

When auto telecoil is enabled, the telecoil automatically turns on when you use the telephone or enter a room loop environment. ③

Switch from auto telecoil to telecoil on and off

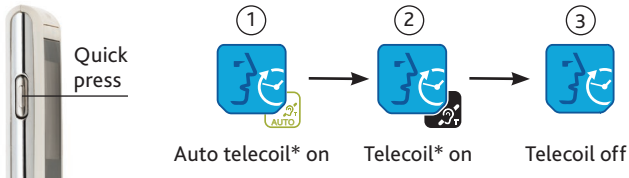

* Your clinician must enable this feature.

How to . . .


Use Demo mode

Demo mode allows you to familiarise yourself with the features on your remote assistant.

To enter Demo mode when your remote assistant is already paired with your processor:

- 1 Press and hold the top right and bottom left soft keys at the same time from the Home screen. The Demo mode screen displays.
- 2 To select bilateral advanced functions, press the lower left soft key. To select unilateral simple functions, press the lower right soft key.

Access bilateral
advanced functions

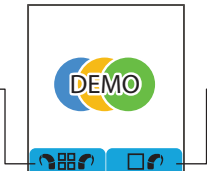

Access unilateral
simple functions

To stop using Demo mode, turn off your remote assistant and turn it back on.

Checking the status indicator

Indicator	What it means
-----------	---------------



Your processor is within operating distance of your remote assistant and they are communicating with each other.



Your processor is not communicating with your remote assistant. For example, they are not within operating distance of each other, or they are not paired.



Your remote assistant is charging.



Flashing orange could mean:

- A visual alert needs your attention.
- Your remote assistant's battery is low or empty.
- Your remote assistant's software may need an update. Contact your clinician.



Your remote assistant is fully charged and the USB cable is still connected.

How to . . .

Access additional features (Advanced mode only)

Press the Cochlear key from the Home screen to access the Processor Status screen.

Enter processor settings menu

Processor Status screen



Enter remote assistant settings menu

Enter the interactive troubleshooting

Adjusting processor settings (Advanced mode only)

Icon

What it means



Adjust the telecoil and microphone mixing ratio.



Adjust the accessory and microphone mixing ratio.



Change your processor's sound ok indicator preferences.



Change your processor's alert and key press indicator preferences.



Enable or disable private tones.

For more information about your remote assistant, refer to the CR110 Remote Assistant User Guide.



Cochlear Ltd (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas

13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

ECREP Cochlear Deutschland GmbH & Co. KG

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

www.cochlear.com

Cochlear, Hear now. And always, Nucleus, and the elliptical logo
are either trademarks or registered trademarks of Cochlear Limited.

© Cochlear Limited 2015

Hear now. And always

471823 ISS2 APR15