

Exercise 3.7 Topic of conversation – unrelated sentences with clue words (set A)

Speaker Copy

Instructions to the Speaker

- 1 Explain to the listener that they should use the clue word to help them repeat the sentence, but that the clue word does not appear in the sentence.
- 2 Say the clue word to the listener with visual cues, or provide the listener with a written list of all the clue words (see next page).
- 3 Say the sentence slowly, but with normal intonation and expression.
- 4 Ask the listener to repeat the sentence.
- 5 Encourage the listener to guess, and give feedback about any correct words.
- 6 If the listener is having difficulty, give one or two key words from the sentence with visual cues, and try again.

Clue word	Sentence
A Greeting	How are you?
B Storm	The weather is sometimes hard to predict.
C Books	Reading is a pleasant pastime.
D Fruit	Apples are good for you.
E Shop	Prices are high these days.
F Meal	The table has to be set for breakfast.
G Bed	When I am tired, I go to sleep.
H Jewellery	Are diamonds very expensive?

Exercise 3.7 Topic of conversation – unrelated sentences with clue words (set A)

Listener Copy

Instructions for the Listener

The speaker will say a sentence. Please try to repeat what they have said. The clue word may help.

Clue word

- A Greeting
- B Storm
- C Books
- D Fruit
- E Shop
- F Meal
- G Bed
- H Jewellery