Kanso[®] Halo Accessory

User Guide

EN

KO

ZH-CN

ZH-TW

MS

ID

TH

Contents

EN	Kanso® Halo Accessory User Guide	1
KO	Kanso® Halo 액세서리 사용 안내서	17
ZH-CN	Kanso® Halo 附件用户指南	33
ZH-TW	Kanso® Halo 配件使用指南	49
MS	Panduan Pengguna Aksesori Kanso® Halo	65
ID	Panduan bagi Pengguna Aksesoris Kanso® Halo	81
TH	Kanso® อุปกรณ์เสริม Halo คู่มือผู้ใช้	97

Contents

Introduction 2
Kanso Halo Accessory 3
Use the Halo Accessory 5
Remove the Halo Accessory 9
Regular care10
Cautions10
Warnings.11
Other information13

Introduction

About this guide

This guide is intended for hearing implant recipients and their carers using the Kanso® Halo Accessory.

Note

- Refer to the relevant sections for cautions and warnings relating to the use of the Halo Accessory.
- Please also refer to your **Important Information** document for essential advice that applies to Cochlear implant systems.

Symbols used in this guide

Note

Important information or advice.

Tip

Time saving hint.

Caution (no harm)


Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

Warning (harmful)

Potential safety hazards and serious adverse reactions. Could cause harm to person.

Kanso Halo Accessory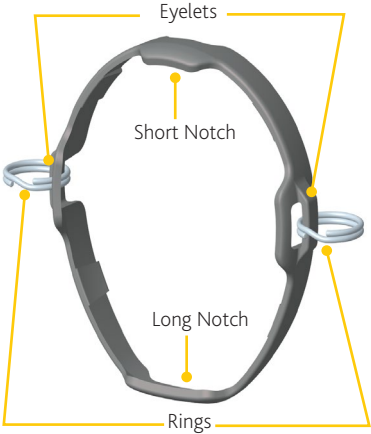

The **Halo Accessory** is a loss prevention device for your sound processor. It provides users of the Off-the Ear (OTE) Kanso 2 Sound Processor with improved security against dropping the device. Hairclips will be required to use with the Halo Accessory.


**Note**

There is an option to use double-loop safety lines to attach hairclips to the Halo Accessory after removing the rings.

The Halo Accessory should be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

Use the Halo Accessory


1. Align the eyelets and notches of the Halo Accessory with the Kanso 2 Sound Processor.
 - i. Shorter notch aligns with the top
 - ii. Longer notch aligns with the base

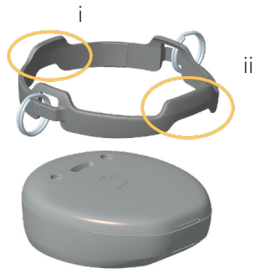
2. Ensure that the Halo Accessory is aligned parallel to the sound processor.


-
3. Push the Halo Accessory until it slides into place on the sound processor. Ensure the sound processor is secure.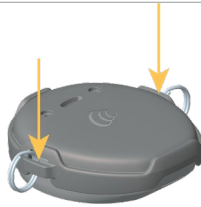

-
4. Thread the hairclips to the rings attached to the Halo Accessory.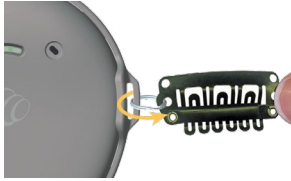

-
5. Mount the sound processor on to the implant, with hairclips hanging from the rings.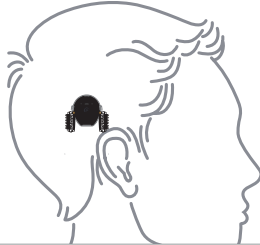

6. Attach the hairclips to hair on either side of the sound processor. For each of the two hair clips:


a. Hold the hairclip from the edges and press in the middle to open it.

b. Attach the hair clip firmly to the hair.


Note
Teeth facing up may be a more secure attachment. However teeth facing down may be suitable for your type of hair.

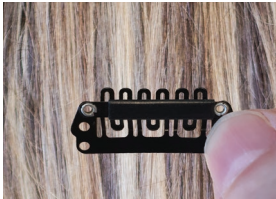c. Press the edges of the hairclip to close.

7. Check if the hairclips are secure and orient in V-shape with the sound processor.

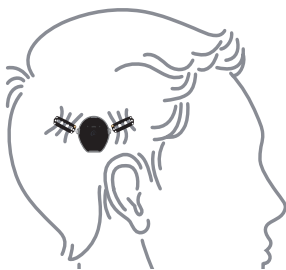**Caution**

To avoid damage to your sound processor, make sure the Halo Accessory is properly secured.

**Note**

When the Halo Accessory is on your sound processor, a portable charger can be attached.

Remove the Halo Accessory

⚠ Caution

Always remove the Halo Accessory before charging your sound processor in the Home Charger. If the Halo Accessory is attached to your sound processor in the Home Charger, charging could be slow or may not work.

-
1. Detach hairclips from your hair.

 2. Remove your sound processor from the implant.

 3. Support the Halo Accessory on either side with your fingers.

 4. Push the sound processor through with your thumbs.

Regular care

Caution

Do not use cleaning agents or alcohol to clean your processor or Halo Accessory.

Caution

To avoid deformation or breakage of your Halo Accessory, do not wring or twist it.

- Check the Halo Accessory for dirt or moisture regularly. Wipe the Halo Accessory with a soft dry cloth.
- Check if the Halo Accessory is becoming loose or showing signs of wear. Replace as needed.

Cautions

- The Halo Accessory is a loss prevention device. Do not use it for any other purpose.
- Always use the Halo Accessory with a compatible sound processor.
- If your Halo Accessory appears damaged or worn, replace it with a new one.

Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Sound processors and related accessories contain small parts that alone or in combination may pose a hazard of inhalation, choking or ingestion. Swallowing or inhaling small parts can cause severe or fatal injuries. Use the tamper-resistant locks to keep small parts attached to the processing unit where available. Always supervise children under 3 years of age and others who may be at risk of inhalation, choking or ingestion of small parts when they use the sound processor and related accessories. When not in use, keep small parts, and combinations of small parts, out of reach from children. If small parts are swallowed or inhaled, seek immediate medical attention.
- The Halo Accessory and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Halo Accessory and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.

- Exposure of the sound processor and Halo Accessory to temperatures above the operating range (+5° C/+41° F to +40° C/+104° F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.
- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

Other information

ITEM	MATERIAL
Halo Accessory	Nylon
Rings	316 Stainless Steel

Product component measurements

Typical values.

ITEM	WEIGHT	WIDTH	LENGTH
Halo Accessory	1 g	38.9 mm	40.1 mm

Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage and transport temperature	-10° C (+14° F)	+55° C (+131° F)
Storage and transport humidity	0% RH	90% RH
Operating temperature (sound processor)	+5° C (+41° F)	+40° C (+104° F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

Disposal

Help protect the environment by not disposing of your Halo Accessory with your unsorted household waste. Please recycle your Halo Accessory according to your local regulations.

Warranty

The use of the Halo Accessory will not impact your Nucleus Global Limited Warranty for your sound processor, if used in accordance with the Halo Accessory instructions.

Labelling symbols

The following symbols may appear on your sound processor or accessories and/or packaging:

Consult instructions for use

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Authorised representative in the European Community

Authorised representative in Switzerland


Unique Device Identification

Catalogue number

Date of manufacture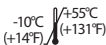

Temperature limits

IP68

Protected against failure from dust penetration
Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

Recyclable packaging

Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

© Cochlear Limited 2022

Trademark legal notice

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield and Xidium are either trademarks or registered trademarks of the Cochlear group of companies.

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© Cochlear Limited 2022

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, 콘트
 ㄱ아, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, 타원형 로고, Vistafix, Whisper, WindShield 및 Xidium은 코클리어 그룹의 상표 또는 등록 상표입니다.

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© Cochlear Limited 2022

Notis undang-undang tanda dagangan

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, logo elips, Vistafix, Whisper, WindShield dan Xidium ialah tanda dagangan atau tanda dagangan berdaftar syarikat kumpulan Cochlear.

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© Cochlear Limited 2022

Pemberitahuan hukum merek dagang

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, logo elips, Vistafix, Whisper, WindShield, dan Xidium merupakan merek dagang atau merek dagang terdaftar dari grup perusahaan Cochlear.

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, โลโก้รูปวงรี, Vistafix, Whisper, WindShield และ Xidium เป็นเครื่องหมายการค้าหรือเครื่องหมายการค้าจดทะเบียนอย่างหนึ่งอย่างใดของกลุ่มบริษัท Cochlear

Hear now. And always

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove, NSW 2066, Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, 30625 Hannover, Germany
Tel: +49 511 542 770 Fax: +49 511 542 7770

Cochlear Americas 10350 Park Meadows Drive, Lone Tree, CO 80124, USA
Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 8204 Fax: +41 61 205 8205

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Cochlear Benelux NV Schaliënhoefreef 20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

Cochlear France S.A.S. 135 Route de Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National) Fax: +33 5 34 63 85 80

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Cochlear (HK) Limited Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong
Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 2nd Floor, Yongsan Centreville Asterium, 25, Hangang-daero 30 gil, Yongsan-gu, Seoul, Korea (04386)

Cochlear Medical Device (Beijing) Co., Ltd
Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Medical Device Company India Pvt. Ltd.
Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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Cochlear Middle East FZ-LLC
Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

Cochlear Latinoamérica S.A.
International Business Park, Building 3835, Office 403, Panama Pacifico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

Cochlear NZ Limited
Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand
Tel: + 64 9 914 1983 Fax: 0800 886 036

www.cochlear.com

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Cochlear Italia S.r.l. Via Trattati Comunitari Europei 1957-2007 n.17, 40127 Bologna (BO), Italy
Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.