# Nucleus<sup>®</sup> Aqua Accessory

User Guide

EN

ID

KO

MS

TH

VI

ZH-TW

# Contents

<b>EN</b>	Nucleus® Aqua Accessory User Guide	1
<b>ID</b>	Panduan bagi Pengguna Aksesoris Nucleus® Aqua	18
<b>KO</b>	Nucleus® 아쿠아 액세서리 사용 안내서	35
<b>MS</b>	Panduan Pengguna Aksesori Nucleus® Aqua	52
<b>TH</b>	คู่มือผู้ใช้อุปกรณ์เสริม Nucleus® Aqua	69
<b>VI</b>	Phụ kiện Nucleus® Aqua Hướng dẫn sử dụng	86
<b>ZH-TW</b>	Nucleus® Aqua 配件使用指南	103

# Contents

Introduction.....	2
Nucleus Aqua Accessory instructions.....	8
Removing the Aqua Accessory .....	10
Using the Aqua Accessory .....	11
Cautions .....	12
Warnings .....	13
Other information.....	15

# Introduction

## About this guide

This guide is intended for hearing implant recipients and their carers using the Nucleus® Aqua Accessory.

### Notes

- Refer to the relevant sections for cautions and warnings relating to the use of the Nucleus Aqua Accessory.
- Please also refer to your **Important Information** document for essential advice that applies to Cochlear implant systems.

## Symbols used in this guide

Note

Important information or advice.

Tip

Time saving hint.

Caution (no harm)


Special care to be taken to ensure safety and effectiveness.  
Could cause damage to equipment.

Warning (harmful)

Potential safety hazards and serious adverse reactions.  
Could cause harm to person.

# Nucleus Aqua Accessory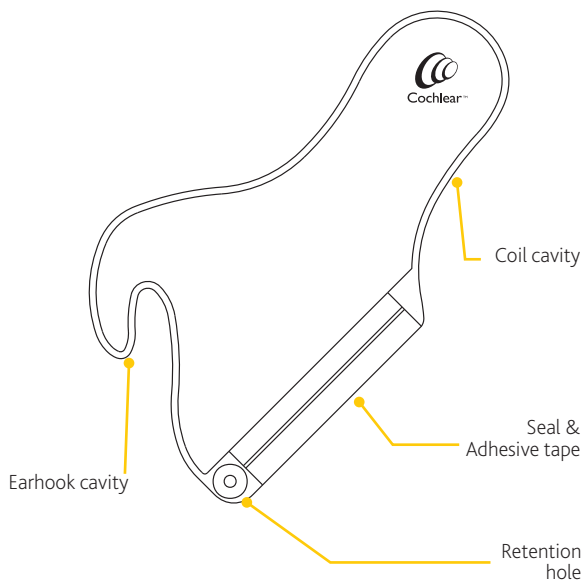

The **Nucleus Aqua Accessory** is a single-use sealable cover that keeps your sound processor dry during use in or around water (e.g. when swimming or in the bath). It is worn behind the ear with your sound processor sealed inside.

The Nucleus Aqua Accessory should be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

**You must use a Cochlear Rechargeable Battery Module for your sound processor to work inside the Aqua Accessory.** Zinc air batteries will not work as they need air and there is no air inside the Aqua Accessory when it's sealed.

#### Note

- You cannot use your sound processor in Hybrid™ mode with the Aqua Accessory.
- You cannot use your sound processor with any of the Cochlear retention accessories (e.g. Snugfit, Hugfit) with the Aqua Accessory.

## Intended purpose

The Nucleus Aqua Accessory is intended to provide improved water and dust resistance to the sound processor.

## Indications

The Nucleus Aqua Accessory is indicated for use with a compatible Cochlear sound processor. Compatible Cochlear sound processors are as follows (when used with a rechargeable battery module):

- Nucleus 7 Sound Processor
- Nucleus 7 SE Sound Processor
- Nucleus 7 S Sound Processor
- CP910 Sound Processor
- CP920 Sound Processor
- CP802 Sound Processor
- CP810 Sound Processor

## Contraindications

The Nucleus Aqua Accessory is not indicated for use with sound processors other than those listed in Indications. The Aqua Accessory is not indicated for use with the Cochlear Battery Cover (with zinc air batteries).

## Intended patient population

The sound processor is intended for patients implanted with a compatible Cochlear Nucleus implant. There are no restrictions for the intended patient population of the sound processor and accessories in terms of age, weight, health or other condition.

## Benefits

The sound processor operates in conjunction with a compatible Cochlear Nucleus implant system.

Potential benefits of receiving a Cochlear Nucleus implant system include:

- Better understanding of speech in quiet.
- Better understanding of speech in noise.
- Increased satisfaction based on hearing capabilities.

## Intended users

The devices listed in Indications are intended for the following users:

- Recipients of a compatible Cochlear Nucleus implant.
- Carers of recipients, who carry out recipient functions as needed. Carers may include parents of paediatric recipients who are less than 12 years old, and nurses or other carers of otherwise dependant recipients.

# Nucleus Aqua Accessory instructions


Connect the the coil cable to the processing unit before using the Aqua accessory.

## Tip

Don't forget you must be using a rechargeable battery for your sound processor to work inside the Aqua Accessory.

1. Insert the coil towards the coil cavity of the Aqua Accessory.

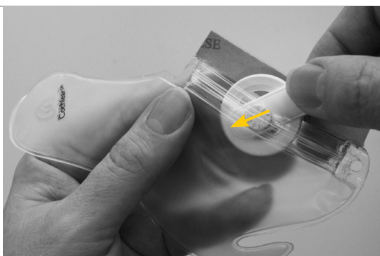
2. Seal the accessory using the plastic zipper.


3. Peel off the adhesive tape from the edge of the accessory.

4. Fold the edge over the adhesive part.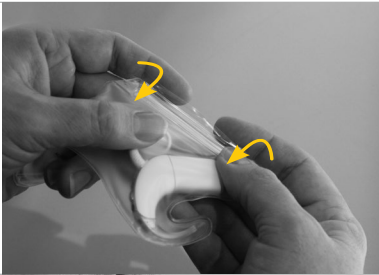

5. Run your thumb to secure the edge on to the adhesive.

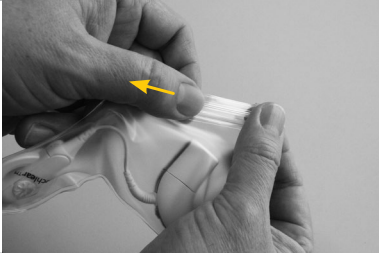6. Wear the sound processor secured in the Aqua accessory.

**Caution**

Make sure your Aqua Accessory is properly sealed, so water doesn't leak in and damage your sound processor.

## Removing the Aqua Accessory

The Aqua Accessory is a single-use device. Dispose of after use.

1. Tear open the Aqua Accessory along the sealed edge.

2. Remove the sound processor.

## Using the Aqua Accessory

The Aqua Accessory can be secured with a swimming cap.

Use the retention hole of the Aqua Accessory with a lanyard or a retention clip.

## Cautions

- Aqua Accessory is a single-use device, do not reuse.
- Always use the Aqua Accessory with a compatible sound processor.
- Avoid pressing the processor button with your fingernails as this can damage your Aqua Accessory.
- If your Aqua Accessory appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.

## Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Small parts (e.g. magnets, Aqua Accessory) may be a choking or suffocation hazard.
- The Aqua Accessory and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Aqua Accessory and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.
- Exposure of the sound processor and Aqua Accessory to temperatures above the operating range (+5° C/+41° F to +40° C/+104° F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.

- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

**Note**

If you want to use your sound processor around water frequently, ask your clinician about the Cochlear Nucleus 7 Aqua+.

## Other information

### MATERIALS

Aqua Accessory

Thermoplastic Polyurethane (TPU)

## Product component measurements

Typical values.

ITEM	WEIGHT	WIDTH	LENGTH
Aqua Accessory	4 g	88.8 mm	153 mm

## Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage and transport temperature	-10° C (+14° F)	+55° C (+131° F)
Storage and transport humidity	0% RH	90% RH
Operating temperature (sound processor)	+5° C (+41° F)	+40° C (+104° F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

## Warranty

Your Nucleus Global Limited Warranty will not be void in circumstances where the Aqua Accessory is used in water with a compatible sound processor, in accordance with the Aqua Accessory instructions.

## Labelling symbols

The following symbols may appear on your sound processor or accessories and/or packaging:

Consult instructions for use

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Do not re-use

Compatible sound processors

Authorised representative in the European Community

Unique Device Identification


Catalogue number

Medical device

Date of manufacture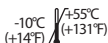

Temperature limits

**IP68**

Protected against failure from dust penetration

Protected against failure from continuous immersion in water up to 4 metres deep for up to 2 hours

CE registration mark with notified body number

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**Cochlear Canada Inc** 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada  
Tel: +1 416 972 5082 Fax: +1 416 972 5083

**Cochlear AG** EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland  
Tel: +41 61 205 8204 Fax: +41 61 205 8205

**Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom  
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

**Cochlear Benelux NV** Schaliënhoeverdreef 20 i, B-2800 Mechelen, Belgium  
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

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**Cochlear (HK) Limited** Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong  
Tel: +852 2530 5773 Fax: +852 2530 5183

**Cochlear Korea Ltd** 1st floor, Cheongwon Building 33, Teheran-ro B gil, Gangnam-gu, Seoul, Korea  
Tel: +82 2 533 4450 Fax: +82 2 533 8408

**Cochlear Medical Device (Beijing) Co., Ltd**

Unit 2608-2617, 26th Floor, No.9 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China  
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

**Cochlear Medical Device Company India Pvt. Ltd.**

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India  
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

**Cochlear Middle East FZ-LLC**

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates  
Tel: +971 4 818 4400 Fax: +971 4 361 8925

**Cochlear Latinoamérica S.A.**

International Business Park, Building 3835, Office 403, Panama Pacifico, Panama  
Tel: +507 830 6220 Fax: +507 830 6218

**Cochlear NZ Limited**

Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand  
Tel: + 64 9 914 1983 Fax: 0800 886 036

[www.cochlear.com](http://www.cochlear.com)

ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Codacs, Contour, 콘트오아, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hugfit, Hybrid, Invisible Hearing, Kanso, MET, MicroDrive, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Profile, Slimline, SmartSound, Softip, SPrint, True Wireless, the elliptical logo, and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCDrive, DermalLock, EveryWear, Human Design, Piezo Power, SoundArc, Vistafix, and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB.

© Cochlear Limited 2021

D1858104 V3

Multilingual version of D1911360 V1 2021-09

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**Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.**