Nucleus[®] 7 Aqua+

User Guide

EN ID KO MS ZH-TW TH VI

CP1000

Hear now. And always

Contents

EN	Cochlear™ Nucleus® 7 Aqua+ — Instructions	1
ID	Cochlear™ Nucleus® 7 Aqua+ — Instructions— Petunjuk	21
KO	Cochlear™ Nucleus® 7 Aqua+ — 사용 지침	41
MS	Cochlear™ Nucleus® 7 Aqua+ — Arahan	61
ZH-TW	Cochlear™ Nucleus® 7 Aqua+ — 說明	81
TH	Cochlear™ Nucleus® 7 Aqua+ — คำแนะนำ	101
VI	Cochlear™ Nucleus® 7 Aqua+ — Hướng dẫn	121

Nucleus[®] 7 Aqua+

The **Cochlear™ Nucleus® 7 Aqua+** is a re-usable sealable cover that, when used with the Cochlear Nucleus 7 Aqua+ Coil, keeps your Cochlear Nucleus 7 Sound Processor (model number: CP1000) dry during use in or around water (e.g. when swimming, surfing, or just having fun splashing around). It is worn behind the ear with your sound processor sealed inside.

The Nucleus 7 Aqua+ is intended for use by recipients using the Nucleus 7 Sound Processor with rechargeable batteries. It should only be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

You must use a Cochlear Standard Rechargeable Battery Module or Cochlear Compact Rechargeable Battery Module for your sound processor to work inside Aqua+.

Zinc air batteries will not work as they need air and there is no air inside Aqua+ when it's sealed.

The **Nucleus 7 Aqua+ Coil** is used in place of your usual coil while your sound processor is inside your Aqua+. It has a special plug that creates a seal when connected to your sound processor's coil socket through the coil plug hole in Aqua+. As with your usual coil, a coil magnet screws into the Aqua+ coil and holds it against your implant.

NOTES

- You cannot use your Nucleus 7 Sound Processor in Hybrid™ mode with Aqua+.
- You cannot use your Nucleus 7 Sound Processor with any of the Cochlear retention accessories (e.g. Snugfit, Hugfit) with Aqua+.

The **Cochlear Nucleus Mic Lock-Stirrup** is a retention option to hold Aqua+ more securely behind your ear. The stirrup connects to the base of Aqua+ and the Mic Lock tube (when cut to size) passes under the ear lobe and connects to the Aqua+ earhook.

The **Nucleus Safety Line** is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

Remotes, wireless accessories and compatible iOS devices can be used with Aqua+, however they should be protected from water and kept dry.

You may notice a difference in sound quality when using Aqua+.

Testing shows Aqua+ can be used at depths of up to 3 metres for up to 2 hours.

Testing shows Aqua+ can be re-used up to 50 times, but this may vary depending on your particular use of it.

There have been no reported undesirable side effects from using Aqua+.

For more information, refer to the *Nucleus 7 Sound Processor User Guide*.

Nucleus 7 Aqua+ instructions

Before you get wet...

TIP

Don't forget you must be using rechargeable batteries for your sound processor to work inside Aqua+.

1. Carefully unplug your usual coil from the sound processor. Hold the coil cable grip and firmly pull it straight out of the processor.

CAUTION

Do not pull on the coil cable or twist when you pull it out.

2. Insert your sound processor into Aqua+ (leave your earhook on).

CAUTION

Be careful to avoid bending the earhook, or detaching it from the sound processor.

3. Insert the plug into the base of Aqua+. To make a good seal, push the plug in as far as it will go.

4. Plug the blue Aqua+ coil (with magnet) straight into your sound processor through the hole in Aqua+. **Grip the black ring on the coil connector. Do not twist.** The coil plug should feel securely connected.

(See page 10 for magnet instructions)

5. If you're concerned about losing your Aqua+, use one of these retention options:

- **Safety Line** clips to your clothing.

(See page 11 for instructions)

- **Mic Lock-Stirrup** holds your processor on your ear.

(See page 12 for instructions)

6. Finally, do the **Aqua+ Checklist** to make sure you're ready to get wet!

7. Have fun!

After you get wet...

Care and storage

1. Rinse Aqua+ and the Aqua+ coil with clean fresh water and dry with a soft cloth.
2. Unplug the blue Aqua+ coil. **Grip the black ring on the coil connector.** Pull the plug straight out.

CAUTION

Do not pull on the coil cable or twist when you pull it out.

3. Open Aqua+ and remove your sound processor.

4. Re-attach your usual coil. Push your coil cable into the processor until it clicks. **Do not twist.**

5. Place Aqua+ and the blue Aqua+ coil in your dry aid kit overnight.


6. Once they're thoroughly dry, store Aqua+, the blue Aqua+ coil and any accessories in the Nucleus 7 Activity Kit's waterproof case.

Aqua+ coil magnet instructions

1. Insert the Cochlear Magnet into the hole in the blue Aqua+ coil.

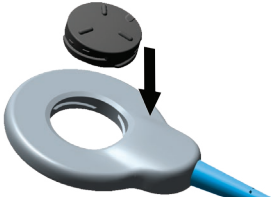2. Turn the magnet clockwise until it stops.

3. Turn the magnet a little more until you feel a click.

TIP

The tamper-resistant lock marker aligns with the coil cable when locked.

Safety Line instructions

WARNING

Retention lines longer than the Safety Line (standard length) are not recommended for use by children as they may present a risk of strangulation.

1. Thread the Safety Line loop end through the Aqua+ ring.

2. Create a knot by passing the clip end of the line through the line loop. Pull it tight.


3. Lift the lever to open the clip. Place the clip over your clothing and press the lever down to close the clip.

Mic Lock-Stirrup instructions

Before using Mic Lock-Stirrup for the first time, cut it to the right length to suit your ear.

1. Snap the stirrup into the dimples at the base of Aqua+.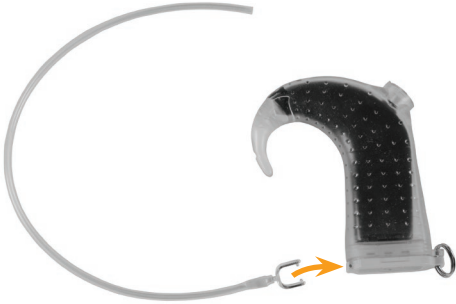

2. Find the right length of tubing to hold Aqua+ in place comfortably behind your ear. Cut the tubing to the required length.

3. Push the cut end of the tubing over the earhook, as far as you can.

4. Wear your Aqua+ with Mic Lock-Stirrup.

**TIP**

Once fitted, you can leave your Mic Lock-Stirrup on your Aqua+.

Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Small parts (e.g. magnets, Safety Line) may be a choking or strangulation hazard.
- The Aqua+ and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Aqua+ and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.
- Exposure of the sound processor and Aqua+ to temperatures above the operating range (+5 °C – +40 °C, +41 °F – +104 °F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.

- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- Store spare magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

Cautions

- Always use Aqua+ with the blue Aqua+ coil, never with your usual coil.
- When connecting the coil cable, always push the plug straight in; do not screw or twist the plug.
- Where possible, avoid pressing the processor button with your fingernail as this can damage your Aqua+.
- If your Aqua+ appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.

Other Information

Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage & transport temperature	-10 °C (+14 °F)	+55 °C (+131 °F)
Storage & transport humidity	0% RH	90% RH
Operating temperature	+5 °C (+41 °F)	+40 °C (+104 °F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

Environmental protection

Your Aqua+ coil contains electronic components subject to the Directive 2002/96/EC on waste electrical and electronic equipment. Help protect the environment by not disposing of your Aqua+ coil with your unsorted household waste. Please recycle your Aqua+ coil according to your local regulations.

Warranty

Your Nucleus Global Limited Warranty will not be void in circumstances where Aqua+ is used in water with Nucleus 7 Sound Processors, in accordance with the Aqua+ instructions.

Labelling symbols

The following symbols may appear on Aqua+ components and/or packaging.

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Authorised representative in the European Community

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark with notified body number

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

Ingress Protection Rating:

- Protected against failure from dust penetration
- Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

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Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 8204 Fax: +41 61 205 8205

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Cochlear Benelux NV Schaliënhoedreef 20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

Cochlear France S.A.S. 135 Route de Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National) Fax: +33 5 34 63 85 80

Cochlear Italia S.r.l. Via Larga 33, 40138 Bologna, Italy
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

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Cochlear (HK) Limited Room 1204, 12/F, CRE Building, No 303 Hennessy Road, Wanchai, Hong Kong SAR
Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea
Tel: +82 2 533 4450 Fax: +82 2 533 8408

Cochlear Medical Device (Beijing) Co., Ltd

Unit 2208-2212, Tower B, Gemdale Building, 91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Medical Device Company India Pvt. Ltd.

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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Cochlear Middle East FZ-LLC

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

Cochlear Latinoamérica S.A.

International Business Park, Building 3835, Office 403, Panama Pacifico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

Cochlear NZ Limited

Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand
Tel: +64 9 914 1983 Fax: 0800 886 036

www.cochlear.com

ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Button, Carina, Cochlear, コクレア, Codacs, Contour, Contour Advance, Custom Sound, ESprit, Freedom, Hear now. And always, Hugfit, Hybrid, inHear, Invisible Hearing, Kanso, MET, MP3000, myCochlear, NRT, Nucleus, 科利耳, Off-Stylet, Slimline, SmartSound, Softip, SoftWear, SPrint, True Wireless, the elliptical logo and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCDrive, DermaLock, EveryWear, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Bluetooth is a registered trademark of Bluetooth SIG.

© Cochlear Limited 2021

D1074443 V5 2022-06

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Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.