Cochlear™ Baha®

Softband

USER MANUAL

EN-GB	FI	LT	SK
AR	FR	LV	SL
BG	HE	MS	SR
CS	HR	NL	SV
DA	HU	NO	TR
DE	IS	PL	UK
EL	IT	PT	ZH-CN
ES	JA	PT-BR	ZH-TW
ET	KK	RO	
FA	KO	RU	

Contents

EN-GB.....	6	KO.....	69
AR.....	9	LT.....	72
BG.....	12	LV.....	75
CS.....	15	MS.....	78
DA.....	19	NL.....	81
DE.....	22	NO.....	85
EL.....	25	PL.....	88
ES.....	29	PT.....	91
ET.....	32	PT-BR.....	95
FA.....	35	RO.....	98
FI.....	39	RU.....	102
FR.....	42	SK.....	106
HE.....	45	SL.....	109
HR.....	49	SR.....	112
HU.....	52	SV.....	116
IS.....	55	TR.....	119
IT.....	58	UK.....	122
JA.....	62	ZH-CN.....	126
KK.....	65	ZH-TW.....	129

EN-GB Baha® Softband User manual

Intended use: The Cochlear Baha System uses bone conduction to transmit sounds to the cochlea (inner ear) with the purpose of enhancing hearing. The Baha Softband is intended to be used as part of the Cochlear Baha System to function as retention and coupling mechanism for the Baha Sound Processor.

Indications: The Baha System is indicated for patients with conductive hearing loss, mixed hearing loss and single-sided sensorineural deafness (SSD).

Clinical Benefit: Most recipients of a bone conduction hearing solution will experience improved hearing performance and quality of life compared to unaided listening.

The Baha Softband is for long-term use as well as for test situations.

It is not made with natural rubber latex. It is fitted with one or two connector disc(s) and a safety release which is designed to open if the Baha Softband should catch on an object (Fig.1).

The Baha Softband is available in unilateral (Fig.1) and bilateral (Fig.2) versions.

Getting started: If delivered with the Baha Softband, attach the Soft pad(s) to the disc(s) (Fig.3) and adjust the length of the band using the slider(s) (Fig.4). The bilateral version has two sliders: one for adjusting the total length and one for adjusting the distance between the discs (Fig.4).

The Baha Softband should be fitted firmly but comfortably (it should be possible to insert one finger underneath the band).

1. Adjust the length of the Baha Softband
2. Attach the Baha sound processor(s) to the disc(s)
3. Place the Baha Softband on the wearer's head and adjust it to fit comfortably. For best performance the disc(s) should be placed flat behind the ear or at any other bony location of the skull i.e. forehead (Fig.5).
4. Turn the sound processor on.

For the first few weeks the Baha Softband can be worn for shorter periods of time until the wearer has become accustomed to it. To help avoid any tenderness use the Soft pad(s) and, vary the position of the disc on the wearer's head each day.

Washing instructions: The Baha Softband can be washed by hand at 40 °C (104 °F) or cooler. Use a mild soap but avoid fabric softeners. Do not tumble dry or iron. The disc(s) can be cleaned with a small brush. Never use any strong chemical solutions.

Caution :

- Remove the sound processor(s) and the Soft pad(s) before washing or cleaning.
- Children should not be left unattended and should always have adult supervision when wearing the Baha Softband.
- If the wearer has an implant, do not place the Baha Softband directly over the implant/abutment during the healing period as this may hinder the osseointegration.
- If any skin irritation is noted the Baha Softband should be removed to let the skin rest. If irritation persists, contact your hearing care professional.

Serious incidents: Serious incidents are rare, any serious incident in relation to your device should be reported to your Cochlear representative and to the medical device authority in your country, if available.

Note:

- The Soft pad(s) can be used to improve comfort.
- Replace the Soft pad(s) when worn or damaged.
- Never use the safety release to open or close the Baha Softband.
- Always check that the Baha Softband is functioning properly.
Never try to modify or replace the elastic material yourself.

Warranty: The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processor. See "Cochlear Baha Global Limited Warranty card" for more details.

	Medical device
	Keep dry
	Unique device identifier
Store at room temperature	Store at room temperature

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www.cochlear.com

This content is meant for professional use. If you are a consumer, please seek advice from your medical practitioner or health professional about treatments for hearing loss. They will be able to advise you on a suitable solution for your hearing loss condition. All products should be used only as directed by your medical practitioner or health professional. Not all products are available in all countries. Please contact your local Cochlear representative.

Contains small parts
that may present a
choking hazard.

Medical
device

Rx Only

Cochlear, Baha, 科利耳, コクレア, 코클리어, Hear now. And always, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB or Cochlear Limited (unless otherwise noted).

© Cochlear Bone Anchored Solutions AB
2020. All rights reserved. 2020-06

630669-5

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Cochlear AG, EMEA Headquarters,