05830

Manufacturer:
Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:
Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405


Local offices:
Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co LtdUnit 2208-2212, Tower B, No.9 Gemdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022
P.R. CHINA Tel: +86 10 5909 7800 Fax: (+86 10) 5909 7900
Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schaliënhoevedreef 20 1, B - 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India (P) Ltd Platina Building, Ground Floor, Plot No. C-59, G-Block, BKC, Bandra - 400 0051, India
Tel: +91 22 6112 1111 Fax: +91 22 61121100
Cochlear Italia S.r.l. Via Larga n°33, 40138 Bologna, Italia Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. Route de l'Orme aux Merisiers, Z.I. Les Algorithmes - Bât. Homère, 91190 Saint Aubin, France Tel: +33 811 111 993 Fax: +33 160 196 499
Cochlear Nordic AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad., Bogazici Plaza No: 6/1, Kavacik,TR - 34805 Beykoz-Istanbul, Turkey
Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1 Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Cochlear™

Baha® Connect

User manual part B

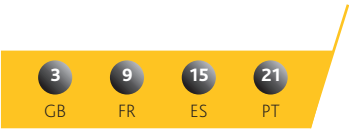
English Français Español Português **ZONE 8**

610129-00

1

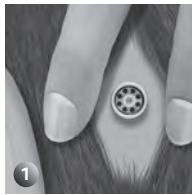
2

3

4

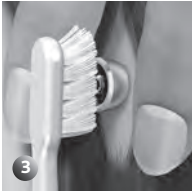5

6

Welcome

This guide is full of tips and advice on how to best use and care for your Baha sound processor when using an abutment to connect to your implant. By reading this manual and then keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

NOTE: This manual is to be seen as a complement to the sound processor manual part A and does not replace it.

Introduction

This guide includes instructions on:

- Connecting your sound processor to an abutment
- Care, cleaning and maintenance
- Using accessories
- Important information & troubleshooting

Note on figures: The figures included on the front cover correspond to sections of information in this manual which is general for all Baha sound processors. Please reference the appropriate figure when reading. The sound processor images shown are representative only and not to scale. The appearance of the sound processor included in your kit may vary.

Your kit includes the following items:

- Cochlear Baha sound processor
- Abutment cleaning brush
- Abutment cover
- Safety line
- Share the Experience test rod
- Document pack (user manual, warranty card, registration card, MRI card)
- Pack of batteries

Connecting the sound processor

See figure 1

Use the tilt technique to safely and comfortably connect and disconnect your sound processor, thus reducing any undue pressure on the snap connector and abutment. By tilting the sound processor onto or off of the abutment, you will avoid any discomfort.

To connect your sound processor:

- 1 Move hair from the abutment.
- 2 Hold the sound processor with the buttons positioned upwards, tilt it and gently snap it into place.
- 3 To release, place one finger under the sound processor and gently tilt until it releases.
- 4 **Note:** To avoid feedback (whistling), ensure that the sound processor is not touching any other items, such as glasses or a hat.

Supporting the processor

See figure 2

Once you are familiar with your processor, you can change the program and the volume level while the sound processor is attached to the abutment. Make sure you support the processor from the bottom with your thumb as you press the buttons with your index finger.

Using the abutment cover

See figure 3

The abutment cover protects the abutment when the sound processor is not in place. This prevents dirt from collecting in the abutment.

To attach the cover:

Gently press it onto the abutment. Lift the cover to remove.

Care of abutment and sound processor

Daily and weekly tasks are required to care for your abutment and sound processor properly.

Cleaning the abutment

Skin debris or other dirt in the abutment can affect the sound quality and lead to damage of your sound processor. Therefore it is very important to clean the abutment area thoroughly. Use the soft abutment cleaning brush provided in your Baha sound processor kit for cleaning.

Should you experience persistent irritation, soreness or other signs of inflammation around the abutment, contact your doctor or hearing care professional.

Daily care

See figure 4

- 1 Clean the area around the abutment daily to avoid the build up of debris. Bathing or showering is a great opportunity to do this. Use plenty of mild soap and warm water.
- 2 Brush on the side of the abutment rather than directly on your skin. Your focus should be to remove crust and debris from around the base of the abutment rather than brushing your skin. Clean your soft brush with mild soap and water after caring for your abutment.

Note: During the first few weeks after dressing removal, cleaning with the soft brush may feel a little rough. Use an alcohol free baby wipe (or similar) until your skin becomes less sensitive and you can use the brush comfortably.

Once your abutment is clean, gently dry the area with a clean towel or tissue. Remove any hair that may have wrapped around the abutment. If you use a hair dryer avoid overheating the abutment area.

Note: Ask a friend or relative to help you clean the abutment. You can also use a hand mirror or pictures from a digital camera to view the abutment area from all angles to make sure it gets cleaned properly.

Note: The abutment cleaning brushes should be changed at regular intervals. Cochlear recommends every 3 months.

Weekly care See figure 4

- 3 Clean the inside of the abutment on a weekly basis, using the cleaning brush as described previously.

If you use the abutment cover make sure it is cleaned at least once a week. Use warm soap and water and your soft brush to clean the cover.

Long-term care

You will have regular check-ups as determined by your hearing health care professional. These checkups help ensure that the processor works and fits properly and that your implant area is healthy.

Cleaning the sound processor

See figure 5

Clean the sound processor's housing with an alcohol-free wipe. Do not use water to clean your sound processor.

Brush the snap coupling clean using the soft brush. Avoid using excessive cleaning force on the snap coupling as this could damage the processor. Your hearing care professional can provide you with a special aftercare kit to keep your sound processor in good condition.

Connecting accessories via the Direct Audio Input (DAI) *See figure 6*

- 1 Match the accessory connector with the sound processor DAI connector.
- 2 Holding the sound processor gently at the corners, carefully push the accessory connector into the DAI port.

Dedicated DAI program and microphone mixing

An optional DAI program that is set by your hearing care professional allows you to listen exclusively to sounds through the DAI.

In all other programs, input from the DAI is combined with input from the microphone.

Using the telecoil

The telecoil unit improves sound when using some landline (fixed) and mobile telephones. It also gives you access to loop facilities in homes and public buildings. Attach the telecoil to the DAI port.

Using the Audio adapter

The Audio adapter allows direct input from computers, personal stereos, TVs, MP3 players and other external audio equipment. One end of the Audio adapter is connected to the sound processor at the DAI. The other end is connected to the audio device.

Caution: Connecting electrical equipment directly to the sound processor is extremely dangerous.

Always use the Audio adapter when connecting an external audio source to your sound processor.

Cochlear is not liable for any damage arising as a result of connecting incorrect equipment or incorrect use.

Using an FM receiver

An FM receiver enables the use of FM systems that are commonly used in school classrooms.

Your hearing care professional can let you know what FM receivers are available. Attach your FM receiver to the DAI port. Keep extra batteries available if using an FM receiver as they have high energy consumption rates.

Warnings

If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.

Caution: Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children, those with mental challenges, and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency center.

Troubleshooting

Battery function can be related to a number of problems, such as no sound, intermittent sound and crackling/buzzing. Often a new battery is the solution.

If the possible solutions in this chart do not solve your problem, please see your hearing care professional for further assistance.

Problem	Cause	Possible Solution
No sound/ weak sound	Weak or dead battery.	Replace battery.
	Volume too low.	Turn up volume.
Feedback (whistling)	Hat, glasses or other object in contact with sound processor.	Relocate or remove the object.
	Battery door in wrong position.	Close battery door.
Sound is distorted or intermittent	Too high volume.	Decrease volume.
	Weak battery.	Replace battery.
	Loose abutment.	Contact your hearing care professional.
Sound processor does not work	Sound processor not turned on.	Turn on sound processor.
	Battery door is not completely closed.	Gently close the battery door completely.
	Dead battery.	Replace battery.
	Battery inserted wrong way.	Make sure battery is inserted correctly.

Additional Information

Instructions for use in the USA

Caution: Federal law (USA) restricts this device to sale or by the order of a medical practitioner.

Warning to hearing aid dispensers

A prospective user of Baha® must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have an implant placed in the skull bone before a Baha sound processor can be dispensed. The Baha sound processor can be used with a softband which does not require an implant. However, a medical evaluation is still warranted.

Important notice for prospective users of a Baha® sound processor

- A prospective user of Baha must always have a thorough medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear, otolaryngologist, otologist, or otorhinolaryngologist) and have an implant placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for Baha. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.

- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children, pets, or people of diminished capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency center, or call the National Button Battery Hotline collect at (202) 625 – 3333.

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