

Cochlear™

Nucleus® CI500 Series implants

Important Information: Warnings, Precautions and
Electromagnetic Compatibility

Hear now. And always



Cochlear®

Symbols



Note

Important information or advice.



Caution (no harm)

Special care to be taken to ensure safety and effectiveness.
Could cause damage to equipment.



Warning (harmful)

Potential safety hazards and serious adverse reactions.
Could cause harm to person.

This document contains important information such as warnings and precautions that apply to cochlear implant systems.

Contents

- Introduction..... 5
- Warnings 6
 - Medical treatments generating induced currents, heat and vibration 6
 - Magnetic Resonance Imaging (MRI) 8
 - Small parts hazard..... 13
 - Adverse environments..... 13
 - Overheating..... 14
 - Long-term effects of electrical stimulation 14
 - Head trauma 14
 - Batteries and battery chargers..... 15
- Precautions 18
 - Theft and metal detection systems..... 19
 - Mobile telephones..... 19
 - Air travel..... 19
 - Scuba diving..... 20
 - Pressure..... 20
 - Electromagnetic interference with medical devices 20
 - Electrostatic discharge..... 21
- Privacy and the collection of personal information 22
- Electromagnetic compatibility (EMC) 23
 - Guidance and manufacturer’s declaration 23
 - Electromagnetic emissions 23
 - Electromagnetic immunity 24
 - Recommended separation distances..... 26
- Implant identification 28

Introduction

This document contains important information that applies to Cochlear™ implant systems in the Nucleus® CI500 Series.

Read this document carefully to ensure that you understand the care of your system.

Discuss this information with your physician before undergoing any major medical procedure.

Warnings

Medical treatments generating induced currents, heat and vibration

Some medical treatments generate induced currents that may cause tissue damage or permanent damage to the implant. Before initiating any of the following treatments deactivate the device.

Warnings for specific treatments are provided below.

Electrosurgery

Electrosurgical instruments are capable of inducing radio frequency currents that could flow through the electrode array. Monopolar electrosurgical instruments must not be used on the head or neck of an implant patient as induced currents could cause damage to cochlear/neural tissues or permanent damage to the implant. Bipolar electrosurgical instruments may be used on the head and neck of patients; however, the cautery electrodes must not contact the implant and should be kept more than 1 cm (½ in.) from the extracochlear electrodes.

Diathermy

Do not use therapeutic or medical diathermy (thermopenetration) using electromagnetic radiation (magnetic induction coils or microwave). High currents induced into the electrode lead can cause tissue damage to the cochlea/brainstem or permanent damage to the implant. Medical diathermy using ultrasound may be used below the head and neck.

Neurostimulation

Do not use neurostimulation directly over the implant. High currents induced into the electrode lead can cause tissue damage to the cochlea/brainstem or permanent damage to the implant.

Electroconvulsive therapy

Do not use electroconvulsive therapy on an implant patient under any circumstances. Electroconvulsive therapy may cause tissue damage or damage to the implant.

Ionizing radiation therapy

Do not use ionizing radiation therapy directly over the implant. It may cause damage to the implant.

Therapeutic ultrasound

Do not use therapeutic levels of ultrasound energy directly over the implant. It may inadvertently concentrate the ultrasound field and cause tissue damage or damage to the implant.

Magnetic Resonance Imaging (MRI)



Cochlear Nucleus implants are MR conditional. MRI is contraindicated except under the circumstances described below. If the patient is implanted with other implants, consult the manufacturer's instructions before performing MRI. Do not allow a patient with an implant to be in a room where an MRI scanner is located except under the following special circumstances.

The patient must take off the sound processor before entering a room where an MRI scanner is located.

The quality of MRI will be affected by the implant. Image shadowing may extend from the implant as far as:

- 6 cm (~2.5 in.) with the magnet removed (at 3 tesla for 3 minutes)
- 12 cm (~4.3 in.) with the magnet in place (at 1.5 tesla for 2 minutes).

Shadowing results in loss of diagnostic information in the vicinity of the implant.

Indications for MRI safety depend on the model of the implant. To verify the model refer to the *Implant identification* section of this document. The Cochlear Nucleus CI500 series implants have a removable magnet but, unlike earlier Cochlear implants, do not have radiopaque lettering.

Implant type	MRI field strength (T)	Maximum spatial gradient field (G/cm)	Max Head SAR (W/kg)	Average Whole Body SAR (W/kg)	
				Landmark location above sternum	Landmark location below sternum
CI512, CI513	0.2	360	1.0	1.0	2.0
	1.5	360	1.0	1.0	2.0
	3.0	700	1.0	1.0	2.0

Table 1: SAR levels during MRI (non-clinical testing)

Cochlear Nucleus CI512 and CI513 cochlear implant

Non-clinical testing according to the international standard ASTM F2182 has demonstrated that the above implants can be scanned safely in 0.2 tesla, 1.5 tesla and 3.0 tesla static magnetic fields at a maximum head averaged Specific Absorption Rate (SAR) of 1 W/kg for 15 minutes of scanning. In non-clinical testing, the above implants produced a temperature rise of less than 2 °C (3.6 °F) at a maximum local SAR of 1 W/kg under specific test conditions stated above.

Note

MRI machine manufacturers may claim that the scanning of patients with implanted devices is generally contra-indicated. This is a general precautionary claim, as MRI machine manufacturers are unable to ensure safety for all types of implantable devices.

Cochlear has performed specific testing for the above implants and established the necessary SAR safety limits as outlined. MRI machines are able to monitor SAR levels. The MRI machine manufacturer should be able to provide advice on how to maintain SAR levels with their machine.

CI512 and CI513 cochlear implants

More than 1.5 tesla (T), up to and including 3.0 T	Surgically remove the magnet for MRI. Tissue damage may occur if the magnet is in place during MRI.
More than 0.2 T, up to and including 1.5 T	Leave the magnet in place for MRI. Bandaging and splint are required.
0.2 T or less	Leave the magnet in place for MRI. Bandaging is optional.

Table 2: MRI in Australia and all other countries in the Asia Pacific region

For more information about magnet removal, refer to the Surgeon's Guide or contact Cochlear.

Performing an MRI scan with the magnet in place

The magnet can only be left in place for some implants at certain field strengths. See Table 2 to determine if the magnet can be left in place.

Warning

Although unlikely with the recommended bandaging, it is possible for the magnet to move during MRI and dislodge from the implant magnet pocket. In this case a surgical intervention to reposition or replace the magnet would be required.


1. Inform the patient that they may feel a slight pulling sensation during the scan. See Patient comfort below.
2. Remove the patient's external equipment (processor and coil) before they enter the MRI room.

Note

The patient cannot hear without the external equipment.

If the scan is at 0.2 T or less, bandaging is not required but acceptable to do so. Proceed to step 3.

If the scan is at more than 0.2 T, up to and including 1.5 T (the magnet must be removed at over 1.5 T), bandage around the head to reduce the likelihood of the magnet moving. Although unlikely with the recommended bandaging, it is possible for the magnet to move during MRI and dislodge from the implant magnet pocket.

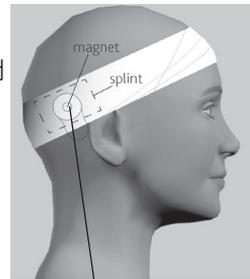

Implant magnet site

Bandage around the head as follows:

- Use an elasticised compression bandage with a maximum width of 10 cm or 4 in. Generic bandages are suitable. No special bandage is required.
- Ensure the centreline of the bandage is over the implant magnet site.
- Use a minimum of two layers at or near full stretch to apply firm pressure to the implant site. 'Full stretch' = no elasticity remaining in bandage.
- Use a splint (see table below) placed against the skin over the site of the magnet to maximise magnet stability.

Splint material	Instruction
A4, 80 gsm sheet of printer/copy paper	Fold five times along the longer edge and place between the implant magnet site and the bandage.
Plastic card (similar to a credit or ID card) without magnetic strip or SIM chip	Place between the implant magnet site and the bandage.
Re-usable adhesive e.g. Bostik Blu-Tack®	Flatten a 1.5 cm to 2 cm diameter ball of Blu-Tack into a disc approximately 0.5 cm thick and place between the implant magnet site and the bandage.

Table 3: Options for stabilising the implant magnet during MRI

3. Conduct the MRI scan. There is no need to position the patient in a particular way because of the implant.

Patient comfort

Explain to the patient that the compression bandage (for MRI above 0.2 T) will reduce the likelihood of the implant magnet moving. However, the patient may still sense the resistance to movement as pressure on the skin. The sensation will be similar to pressing down firmly on the skin with the thumb.

If the patient experiences pain with the bandage in place, check that it is not too tight, and if necessary, consider performing an MRI scan at 0.2 T (no bandaging required). Alternatively, consult the patient's physician to determine if the magnet should be removed or if a local anaesthetic may be applied to reduce discomfort.



Caution

If administering local anaesthetic, take care not to perforate the implant silicone.

Small parts hazard

The external implant system contains small parts and accessories that may be hazardous if swallowed or may cause choking if ingested or inhaled.

Adverse environments

The operation of the cochlear implant system may be adversely affected in environments of high magnetic field strength and high electric field strengths (e.g. close to high power commercial radio transmitters).

Seek medical advice before entering any environment that may adversely affect the operation of your implant (including areas protected by a warning notice preventing entry by patients fitted with a pacemaker).

Overheating

Remove your processor immediately if it becomes unusually warm or hot, and seek advice from your clinician. Parents and caregivers should touch their child's or recipient's processor to check for heat if the child or recipient is showing signs of discomfort.

Long-term effects of electrical stimulation by the implant

Most patients can benefit from electrical stimulation levels that are considered safe, based on animal experimental data. The long-term effects of such stimulation in humans are unknown.

Head trauma

Young children who are developing their motor skills are at greater risk to receive an impact to the head from a hard object (e.g. a table or chair).

A blow to the head in the area of the cochlear implant may damage the implant and result in its failure.

Impact to external components (e.g. sound processor, acoustic component) while being worn could result in damage to the device or injury.

Batteries and battery chargers

Dispose of used batteries promptly and carefully, in accordance with local regulations. Keep away from children.

Wash hands after handling disposable batteries.

Do not recharge disposable batteries.

Do not disassemble, deform, immerse in water or dispose of batteries in fire.

Do not mix old and new batteries or batteries of different types or brands.

Replace batteries with those recommended in the user instructions supplied with your processor.

Only use rechargeable batteries and battery chargers supplied or recommended by Cochlear. Use of another battery or battery charger type and brand may present risk of harm or injury. Do not touch battery charger contacts or allow children to use the battery charger without adult supervision.

Do not allow children to replace batteries without adult supervision.

Do not short circuit batteries, e.g. do not let terminals of batteries contact each other or do not carry batteries loosely in pockets.

Store unused batteries in original packing in a cool dry place. When processor is not in use, disconnect disposable or rechargeable batteries and store separately in a cool dry place.

Do not expose batteries to heat, e.g. Never leave batteries in sunlight, behind a window or in a car.

Do not use damaged or deformed batteries. If skin or eyes come into contact with battery fluid or liquid, wash out with water and seek medical attention immediately.

Never put batteries in mouth. If swallowed contact your physician or local poison information centre.

 Warnings

In certain circumstances, rechargeable batteries can become very hot and could cause injury. In the case of overheating, remove the device immediately and notify your clinician.

Use of the rechargeable battery is contraindicated in patients who cannot remove the device by themselves or are unable to notify a caregiver that the device has become hot.

Precautions

If you experience a significant change in performance or the sound becomes uncomfortable, turn off your processor and contact your implant centre.

Use the implant system only with the approved devices and accessories listed in the user guide.

Your processor and other parts of the system contain complex electronic parts. These parts are durable but must be treated with care. The opening of your processor by anyone other than Cochlear's qualified service personnel invalidates the warranty.

Each processor is programmed specifically for each implant. Never wear another person's processor or lend yours to another user. If you have two processors (one for each ear), always wear the processor programmed for your left ear on the left, and the processor programmed for your right ear on the right. Using the wrong processor could result in loud or distorted sounds that, in some instances, may cause extreme discomfort.

Do not operate or store your processor at temperatures other than those recommended in the user instructions supplied with your processor.

Your processor sound quality may be intermittently distorted when you are within approximately 1.6 km (~1 mile) of a radio or television transmission tower. The effect is temporary and will not damage your processor.

Theft and metal detection systems

Turn off your processor when in the vicinity of or passing through any theft and metal detection system devices. Recipients should carry the Cochlear Implant Patient Identification Card with them at all times.

Devices such as airport metal detectors and commercial theft detection systems produce strong electromagnetic fields. The materials used in the cochlear implant may activate metal detection systems.

Some cochlear implant recipients may experience a distorted sound sensation when passing through or near one of these devices.

Mobile telephones

Some types of digital mobile telephones, e.g. Global System for Mobile communications (GSM) as used in some countries, may interfere with the operation of the external equipment. As a result, implant recipients may perceive a distorted sound sensation when in close proximity, 1–4 m (~3–12 ft), to a digital mobile telephone in use.

Air travel

Some airlines request that passengers turn off portable electrical devices, such as laptop computers and electronic games, during take-off and landing or whenever the seat belt sign is illuminated. Your processor is considered to be a medical portable electronic device, so you should notify airline personnel that you are using an implant system. They can then alert you to safety measures which may include the need to switch the processor off.

Transmitting devices such as mobile/cell phones are required to be switched off on aircraft. If you have a remote control (remote assistant) for your processor, it should also be switched off because it is transmitting high frequency radio waves when switched on.

Scuba diving

For Cochlear Nucleus CI500 Series implants the maximum diving depth when wearing an implant is 40 m (~131 ft).

Recipients should seek medical advice before participating in a dive for conditions that might make diving contraindicated, e.g. middle ear infection, etc. When wearing a mask, avoid pressure over the implant site.

Pressure

Do not apply continued pressure to the coil when in contact with the skin (e.g. sleeping/lying on coil or using tight fitting headwear) as this may result in pressure sores.

If the coil magnet is too strong or is in contact with the skin, pressure sores may develop at the coil site. If this happens or if you experience any discomfort in this area, contact your clinician.

Electromagnetic interference with medical devices

Cochlear Nucleus Remote Assistants and Cochlear Nucleus Sound Processors meet defined international Electromagnetic Compatibility (EMC) and emission standards. However, because the remote assistant and sound processor radiate electromagnetic energy, it is possible that they could interfere with other medical devices such as cardiac pacemakers and implantable defibrillators when used nearby.

It is recommended that the remote assistant and sound processor are kept at least 15 cm (~6 in.) away from devices which could be subject to electromagnetic interference. For added assurance, please also consult the recommendations provided by the device manufacturer.

Electrostatic discharge

Prior to engaging in activities that create extreme electrostatic discharge, such as playing on plastic slides, the processor should be removed.

A discharge of static electricity can in rare cases damage the electrical components of the cochlear implant system or corrupt the program in your processor.

If static electricity is present, e.g. when putting on or removing clothes over the head or getting out of a vehicle, cochlear implant recipients should touch something conductive, e.g. a metal door handle, before the cochlear implant system contacts any object or person.

Privacy and the collection of personal information

During the process of receiving a Cochlear device, personal information about the user/recipient or their parent, guardian, care giver and hearing health professional will be collected for use by Cochlear and others involved in care with regard to the device.

For more information please read Cochlear's Privacy Policy on www.cochlear.com or request a copy from Cochlear at the address nearest you.

Electromagnetic compatibility (EMC)

Guidance and manufacturer's declaration

The Nucleus range of sound processors, remote assistants and remote controls are intended for use in the electromagnetic environments specified in this document.

They have been tested and found to be in compliance as shown. You should take care to use your equipment as described.



Note

The programming configuration, including the programming pod, is Class A. If used in a home environment, the programming configuration may cause interference with electronic devices.

Electromagnetic emissions

Emission test	Compliance	Guidance
RF emissions CISPR 11	Group 1	RF energy is only used for its internal function. The RF emissions are very low and not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The device is suitable for use in all establishments, including domestic establishments and those directly connected to public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3		

Table 4: Electromagnetic emissions

Electromagnetic immunity

Immunity test	IEC 60601 test level	Compliance level	Guidance
Electrostatic discharge IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Refer to <i>Electrostatic discharge</i> section
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable	Not applicable
Surge IEC 61000-4-5			
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11			
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields be at levels characteristic of a typical location in a typical commercial or hospital environment
Conducted RF IEC 61000-4-6	Not applicable	3 V/m	Refer to <i>Warnings</i> and <i>Precautions</i> sections, and <i>Guidance</i> below
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz		

Table 5: Electromagnetic immunity

Guidance

Portable and mobile RF communications equipment should be used no closer to any part of the devices, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

Recommended separation distance (d):

$$d = 1.2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$$

$$d = 2.3 \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$$

where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,^a should be less than the compliance level in each frequency range^b.

Interference may occur in the vicinity of equipment marked with the following symbol:



Note

1. At 80 MHz and 800 MHz, the higher frequency range applies.
2. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Explanatory notes:

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the processor is used exceeds the applicable RF compliance level above, the processor should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the processor.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances

Your processor is intended for use in an electromagnetic environment where the radiated RF disturbances are controlled.

To prevent electromagnetic interference, maintain a minimum distance between the portable and mobile RF communications equipment (transmitters) and the device as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.5 GHz $d = 2.3 \sqrt{P}$
0.01	Not applicable	0.12	0.23
0.1		0.38	0.73
1		1.2	2.3
10		3.8	7.3
100		12	23

Table 6: Recommended separation distances

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note

1. At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.
2. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.


Implant identification

If required, the implant type and model can be identified without the need of surgical intervention, using X-ray or Cochlear programming software.

When interpreting a sagittal X-ray image of a Cochlear implant, the device series can be identified by the electronic assembly layout.

Cochlear CI500 Series implants have:

- a round shape at the coil exit end
- four large components at the electrode exit end.

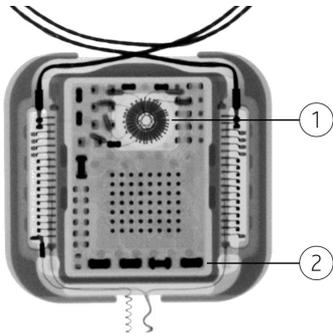

- 1 Round shape at coil exit end
- 2 Four large components at electrode exit end

Figure 1: Plain X-ray of CI500 Series implant

Cochlear™



Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada

Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland

Tel: +41 61 205 0404 Fax: +41 61 205 0405

ECREP Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom

Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Cochlear Benelux NV Schaliënhoedvedreef 20 i, B-2800 Mechelen, Belgium

Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

Cochlear France S.A.S. Route de l'Orme aux Merisiers, Z.I. Les Algorithmes – Bât. Hornère, 91190 Saint-Aubin, France

Tel: +33 805 200 016 Fax: +33 160 196 499

Cochlear Italia S.r.l. Via Larga 33, 40138 Bologna, Italy

Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden

Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.

Çubuklu Mah. Boğaziçi Cad., Boğaziçi Plaza No: 6/1, Kavacak, TR-34805 Beykoz-Istanbul, Turkey

Tel: +90 216 538 5900 Fax: +90 216 538 5919

Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR

Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea

Tel: +82 2 533 4663 Fax: +82 2 533 8408

Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585

Tel: +65 6553 3814 Fax: +65 6451 4105

Cochlear Medical Device (Beijing) Co Ltd

Unit 2208 Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China

Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

Cochlear Medical Device Company India Pvt. Ltd.

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India

Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan

Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

www.cochlear.com

Cochlear implant systems are protected by one or more international patents.

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

Advance Off-Stylet, AutoNRT, Beam, Clinicnet, Cochlear, Contour, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hybrid, Invisible Hearing, NRT, Nucleus, Off-Stylet, SmartSound, SPrint, the elliptical logo, Nucleus in Chinese characters, Codacs, and myCochlear are either trademarks or registered trademarks of Cochlear Limited. Baha, Baha Caleido, Baha Divino, Baha Intenso and Vistafix are registered trademarks of Cochlear Bone Anchored Solutions AB. Blu-Tack is a registered trademark of Bostik Limited.

© Cochlear Limited 2013

Hear now. And always

Printed in Australia
453340 ISS2 DEC13