

Quick Guide for Baha® 5 Sound Processor Users


Cochlear™ Wireless Phone Clip

Benefits of your Cochlear Wireless Phone Clip

- Enjoy hands-free calling direct to your Baha sound processor.
- Hear your Bluetooth®-enabled navigation system directly in your Baha sound processor.
- Discreetly adjust your Baha sound processor by using the Phone Clip as a remote control.
- Stream music from a Bluetooth-enabled device.
- The Phone Clip can be easily attached to clothing. The built-in microphone will pick up and transmit your voice, even if your phone is several meters away.



Product Overview

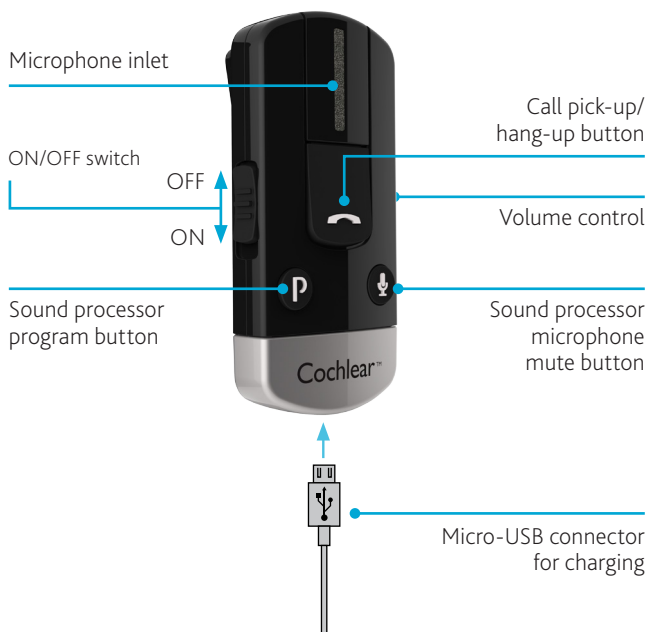

Getting Started

Check with your hearing healthcare professional whether your sound processor has been paired with your Phone Clip. Pre-paired sound processors do not need to be paired again. If your sound processor is not pre-paired, instructions on how to easily pair it with your Phone Clip are on the last page of this guide.

Tip! Make sure that your sound processor volume is comfortably adjusted before you change the settings with the volume control on the Phone Clip.

Charging the Battery








Plug one end of the included micro-USB charging cable into the Phone Clip and the other into a computer, or into a power outlet via the supplied adapter.

When the red LED indicator on top of the Phone Clip turns green with a red blink every 2 seconds charging is complete.

The first time you charge your Phone Clip allow it to charge for at least 3 hours, even if the indicator light shows a full charge. The battery cannot be overcharged.

Light indicators (LED)

The LED indicator at the top of your Phone Clip functions as a multi-purpose user interface, providing information on the status of the device.

Blinks	Meaning
	Normal operation (green)
	Battery low (yellow)
	Charging (red)
	Fully charged – still connected to charger (red/green)
	Ready for pairing with sound processor (yellow)
	Ready for pairing with Bluetooth device (blue)
	Bluetooth call active (blue)

Connecting with Your Mobile Phone

To receive or place phone calls with the Phone Clip, the Bluetooth® functionality must be activated in your mobile phone. Your phone might ask you to confirm whether you would like to connect with the Phone Clip after it is successfully paired.

If you have to connect manually, locate a list of “Paired Devices” in your phone’s Bluetooth menu. From this list, highlight “Hearing Aid Phone” and select “Connect.”

You are now ready to use your Phone Clip.

On some phones you can also enable “Automatic Reconnection.” Refer to your mobile phone’s user guide for instructions on how to do this.



Tip! In most instances, a headset symbol on the main screen of your mobile phone will confirm that you have successfully connected.

Using the Phone Clip

Managing calls with the call pick-up / hang-up button



Incoming calls:

To answer a call, press once.

To decline a call, double-click.

During a call:

To transfer a call started on your phone to the phone clip, press once.

To put a call on hold, press and hold for 2 seconds.

To make a call:

To redial the last number, double-click.

To activate voice dialling, press and hold for 2 seconds.

Using the Phone Clip as a remote control



When you’re not on the phone, the Phone Clip can also be used as a simple remote control for your sound processor(s).


Adjust the sound processor’s volume.

Press once to mute and unmute the sound processor’s microphones.

Press one or more times to toggle through the sound processor’s programs.

Pairing with Your Baha Sound Processor

- 1 Switch off your Baha sound processor by opening the battery door.

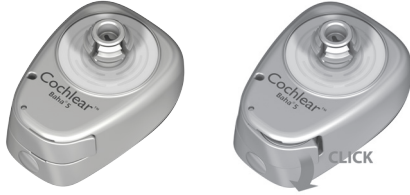

- 2 Switch on your Phone Clip and remove the silver cap.



- 3 Press the white pairing button on the Phone Clip once using the tip of a pen or similar object.



- 4 The LED will flash yellow every two seconds and the Phone Clip will now be in pairing mode for 20 seconds.



- 5 While pairing mode is active, switch on your sound processor by closing the battery door.



- 6 Successful pairing will be indicated by an audible melody played in the sound processor.



Pairing with Your Mobile Phone

- 1 Switch on your Phone Clip and remove the silver cap.
- 2 Make sure your mobile phone's Bluetooth® function is switched on. Refer to your mobile phone user guide or visit www.jabra.com for instructions.



- 3 Press the blue Bluetooth pairing button on the Phone Clip once to activate the pairing mode for 2 minutes. During pairing the LED will turn blue.



- 4 On your mobile phone search for new Bluetooth devices and select "Hearing Aid Phone" from the list.
- 5 If your mobile phone asks for a passcode, enter "0000" (four zeros).
- 6 Your Phone Clip is now paired with your mobile phone. If your mobile phone asks which Bluetooth service you wish to enable, select "Headset".

Support

For full details about the Cochlear Wireless Phone Clip, please refer to the User Manual.

Need help Pairing? Text **"PAIRING"** to (720) 606-6970.

A collection of yellow icons representing various mobile devices and accessories: a smartphone, a tablet, a mouse, a smartphone with a speech bubble, a smartphone with a dial pad, a smartphone with a play button, and a smartphone with a Wi-Fi signal icon.

Visit www.Cochlear.com/US to learn about our full range of apps and wireless accessories.

www.Cochlear.com/US

Cochlear Americas
13059 East Peakview Avenue
Centennial, CO 80111 USA
Telephone: 1 303 790 9010
Support: 1 800 483 3123

Cochlear Canada Inc.
2500-120 Adelaide Street West
Toronto, ON M5H 1T1 Canada
Telephone: 1 416 972 5082
Fax: 1 416 972 5083

©Cochlear Limited 2015. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.

Apple, the Apple logo, FaceTime, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.



BUN349 ISS1 APR15

