

Cochlear[™] Nucleus[®]

Implants MRI

Safety Checklist

This form guides you through the critical aspects of performing an MRI scan safely for patients with a Cochlear[™] Nucleus[®] implant. Before using this form, review the Cochlear Nucleus MRI Guidelines, available on the website:

www.cochlear.com/mri

Ensure the implant site has healed before an MRI scan is performed. The cochlear implant physician should be consulted if there are any concerns.

Follow these steps prior to inviting the patient into the MRI room and before performing the MRI scan.

- 1 Identify and record the Cochlear Nucleus Implant model using the Cochlear Patient Implant Card, X-ray or surgical/clinical notes in the spaces provided.
- Implant model number for left ear:
 - Implant model number for right ear:

- 2 Determine if implant magnet removal is required or if a Cochlear MRI Kit (bandage and splint) is necessary.
- Implant magnet removal may be mandatory at certain field strengths for specific implant types. Refer to **Table 2: MRI safety information and recommended SAR limits**.
 - Implant magnet removal may be necessary to reduce the artefact. Refer to the **MRI Guidelines** for artefact sizes. Metal Artefact Reduction Sequences (MARS) are recommended for optimal results.
 - Implant magnet removal may be avoided at certain field strengths with use of a Cochlear MRI Kit. The intent of an MRI Kit is to provide pressure over the implant magnet – not the implant body. Refer to **Table 2: MRI safety information and recommended SAR limits** .



Note

If there is no implant magnet present, then an MRI Kit is not required.

Circle the applicable option:

Implant magnet will be in place for scan / Implant magnet will not be in place for scan


Circle the applicable option:

MRI Kit is required for scan / MRI Kit is not required for scan

- 3 Record the MR parameters and transmit coil to be used for this scan in the spaces provided.
- MRI field strength:
 - Maximum spatial gradient the patient will encounter entering the MRI bore:
- Circle the applicable transmit coil to be used:**
Whole body coil / Head coil / Other local cylindrical transmit coil
-

- 4 Identify and record the SAR limit and maximum allowable spatial gradient for the implant type and MRI RF transmit coil in the spaces provided. Refer to **Table 1** and **Table 2**.
If bilaterally implanted, a patient may have two different cochlear implant types. Comply with the lowest SAR limit and lowest allowable spatial gradient of the two devices.
- Lowest SAR limit for recipient's implant types and scan conditions:
.....
 - Lowest allowable spatial gradient for recipient's implant types:
.....
-

- 5 Counsel the patient on sensations and risks.
- For patients where an implant magnet is in place, explain that they may sense the implant magnet moving. The MRI Kit will reduce the likelihood of the implant magnet moving. However, they may still sense resistance to movement as pressure on the skin. The sensation will be similar to pressing down firmly on the skin with the thumb.
 - Refer to the **MRI Guidelines** for a complete list of Warnings and Cautions.
-

- 6 Remove the sound processor and any accessories before entering the MRI room.
The sound processor is MR Unsafe.
-  **Note**
The patient may no longer be able to hear instructions with the sound processor removed.
-

- 7 Prior to entering the MRI room, apply the MRI Kit, if identified as required in step 2.
- Ensure you have the contents of the MRI Kit available and within easy reach.
 - Contents include: Two (2) splints and one (1) compression bandage.
 - Full instructions are supplied with every MRI Kit, and are also listed in the MRI Guidelines.
 - An instructional video for applying the MRI Kit is available on the Cochlear website.
 - Apply the splint and bandage to the implant site or sites in accordance with the information in **Table 2**.
-

- 8 Comply with patient positioning requirements.
- For safety, the patient should be in a supine position (lying flat on back, face upward) prior to entering the MRI bore.
 - Align the patient's head with the bore axis of the MRI machine (max 15° deviation allowed from the z-axis).
 - Advise the patient to lie as still as possible and to not move their head during the MRI scan.
 - Correct patient positioning prior to the MRI scan will minimise discomfort and reduce the risk of implant magnet demagnetisation.

Scan conditions and SAR limits

- The MRI safety information in the tables below only applies to 1.5 T and 3 T MRI horizontal scanners (closed bore or wide bore) with a circularly polarised (CP) RF field.
- MRI scans at 3 T must be performed in quadrature mode or CP mode for the radio frequency (RF) transmit coil.
- Using a multichannel mode may result in localised heating above safe levels.
- Maximum active scan time of 60 minutes with the SAR limits in **Table 2** below.
- The SAR limit depends on the selection of MRI RF transmit coil.
Refer to the Specific conditions for RF transmit coil in **Table 1** below.

RF transmit coil	Specific conditions
Main scanner body coil	<ul style="list-style-type: none"> • Keep local planar (flat linearly polarised) receive only RF coils more than 10 cm away from cochlear implant. • Local cylindrical RF receive only coils can be placed anywhere, with respect to the implant. • Comply with the whole body average SAR limit for the relevant implant type, field strength and landmark location. See Table 2.
Transmit/receive head coil	<ul style="list-style-type: none"> • Comply with the head SAR limit for the relevant implant model and field strength. See Table 2.
Other local transmit/receive coils (e.g. knee)	<ul style="list-style-type: none"> • Ensure distance between coil and implant is greater than the coil radius. • There are no added SAR restrictions due to the presence of the implant. Limit SAR as you would for a typical patient who does not have an implantable device.

Table 1: Specific conditions for RF transmit coil

Implant type	MRI field strength (T)	Remove implant magnet Yes/No	MRI Kit required Yes/No	Maximum allowable spatial gradient field (T/m)	Head average SAR limit (W/kg) Using transmit / receive head coil	Whole body average SAR limit (W/kg) Landmark location	
						<40 cm from top of head	≥40cm from top of head
CI600 Series implants							
CI612, CI622, CI624, CI632	1.5	No	No	20	<2	<1	<2
CI612	3	No	No	20	<1	<0.5	<1
CI622, CI624, CI632	3	No	No	20	<1	<0.4	<1
CI500 Series implants							
CI512, CI522, CI532, ABI541	1.5	No	Yes	20	<2	<1	<2
CI512, ABI541	3	Yes	No	20	<1	<0.5	<1
CI522, CI532	3	Yes	No	20	<1	<0.4	<1
CI24RE Series implants							
CI422, CI24REH (Hybrid L24) CI24RE (CA), CI24RE (ST)	1.5	No	Yes	20	<2	<1	<2
	3	Yes	No	20	<1	<0.5	<1
CI24R & CI24M Series implants							
CI24R (CA), CI24R (CS), CI24R (ST), CI24M, ABI24M	1.5	No	Yes	20	<2	<1	<2
	3	Yes	No	20	<1	<0.5	<1
CI 11+11+2M	1.5	No	Yes	20	<1	<0.5	<1
	3	MRI is contraindicated					
CI22M Series implants							
CI22M with removable magnet	1.5	No	Yes	20	<2	<1	<2
	3	MRI is contraindicated					
CI22M without removable magnet	1.5	MRI is contraindicated					
	3	MRI is contraindicated					

Table 2: MRI safety information and recommended SAR limits*

* Not all products are available in all countries.

Please contact your local Cochlear representative for product information.

Hear now. And always

As the global leader in implantable hearing solutions, Cochlear is dedicated to helping people with moderate to profound hearing loss experience a life full of hearing. We have provided more than 600,000 implantable devices, helping people of all ages to hear and connect with life's opportunities.

We aim to give people the best lifelong hearing experience and access to innovative future technologies. We collaborate with the industry's best clinical, research and support networks.

That's why more people choose Cochlear than any other hearing implant company.

AU Cochlear Ltd (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

ECREP DE Cochlear Deutschland GmbH & Co. KG

Mailänder Straße 4 a, 30539 Hannover, Germany
Tel: +49 511 542 770 Fax: +49 511 542 770

CHREP CH Cochlear AG

Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 8204 Fax: +41 61 205 8205

US Cochlear Americas

10350 Park Meadows Drive, Lone Tree, CO 80124, USA
Tel: +1 303 790 9010

CA Cochlear Canada Inc

2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 (800) 483 3123 Fax: +1 416 972 5083

GB Cochlear Europe Ltd

6 Dashwood Lang Road, Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

BE Cochlear Benelux NV

Schaliënhoeverdreef 20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

FR Cochlear France S.A.S.

135 Route de Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National)
Fax: +33 5 34 63 85 80

IT Cochlear Italia S.r.l.

Via Trattati Comunitari Europei 1957-2007 n.17,
40127 Bologna (BO), Italy
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

SE Cochlear Nordic AB

Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

www.cochlear.com

TR Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.

Küçükbakkalköy Mah, Defne Sok, Büyükhanlı Plaza No:3 Kat:3
Daire: 9-10-11-12, 34750, Ataşehir, İstanbul, Türkiye
Tel: +90 216 538 5900 Fax: +90 216 538 5919

HK Cochlear (HK) Limited

Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road,
Causeway Bay, Hong Kong
Tel: +852 2530 5773 Fax: +852 2530 5183

KR Cochlear Korea Ltd

2nd Floor, Yongsan Centreville Asterium, 25,
Hangang-daero 30 gil, Yongsan-gu, Seoul, Korea (04386)
Tel: +82 2 533 4450 Fax: +82 2 533 8408

CN Cochlear Medical Device (Beijing) Co., Ltd

Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road,
Chaoyang District, Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

IN Cochlear Medical Device Company India Pvt. Ltd.

Ground Floor, Platina Building, Plot No C-59, G-Block,
Bandra Kurla Complex, Bandra (E), Mumbai - 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

JP 株式会社日本コクレア(Nihon Cochlear Co Ltd)

〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル
Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

AE Cochlear Middle East FZ-LLC

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor,
Offices IR1 and IR2, Dubai, United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

PA Cochlear Latinoamérica S.A.

International Business Park, Building 3835, Office 403,
Panama Pacifico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

NZ Cochlear NZ Limited

Level 4, Takapuna Towers, 19-21 Como St, Takapuna,
Auckland 0622, New Zealand
Tel: + 64 9 914 1983 Fax: 0800 886 036

This material is intended for health professionals. If you are a consumer, please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

Cochlear, 科利耳, コクレア, 코클리어, Hear now. And always, Nucleus, Kanso, Advance Off-Stylet, AutoNRT, コントウア, Contour Advance, Custom Sound, Freedom, NRT, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of Cochlear Group of companies (unless otherwise noted).