94386

 Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 8204 Fax: +41 61 205 0405
Cochlear Latinoamérica, S. A. International Business Park Building 3835, Office 403, Panama Pacifico, Panama Tel: +507 830 6220 Fax: +507 830 6218

Local offices:

Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, 30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom Tel: +44 1932 26 3400 Fax: +44 1932 26 3426
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1204, 12/F, 297-307 Hennessy Road, Wan Chai, Hong Kong Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co Ltd Unit 2208-2212, Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, PR China Tel: +86 10 5909 7800 Fax: +86 10 5909 7900
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schaliënhoeverdreef 201, 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India PVT Ltd Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051, India Tel: +91 22 6112 1111 Fax: +91 22 61121100
Cochlear Italia SRL Via Larga 33, 40138 Bologna, Italy Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. 135 route de Saint Simon, 31100 Toulouse, France Tel: +33 5 34 63 85 85 (international) Tel: 0805 200 016 (national) Fax: +33 5 34 63 85 80
Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad. Bogazici Plaza No: 6/1 Kavacik, 34805 Beykoz-Istanbul, Turkey Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Ardium, Baha, Baha Divino, Baha Intenso, Baha PureSound, Baha SoftWear, BCDrive, DermaLock, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Cochlear, Hear now. And always and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited. © Cochlear Bone Anchored Solutions AB 2015. All rights reserved. JAN15.

0413

613697-03

Cochlear™

Baha® 4 Sound Processor

User manual part A

English Français Deutsch Nederlands Italiano **ZONE 1**

3

GB

11

FR

19

DE

27

NL

35

IT

Sound processor type designations for models included in this User Manual are:

FCC ID QZ3BAHA4, IC: 8039C-BAHA4, IC model: Baha4.

Statement:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications can void the user’s authority to operate the equipment .

Intended use

The Cochlear™ Baha® 4 Sound Processor uses direct bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

List of countries:

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

Specification of restrictions: You are not allowed to operate the equipment within 20 km of the center of Ny Ålesund, Norway.

The products are in compliance with the following regulatory requirements:

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Equipment includes RF transmitter.

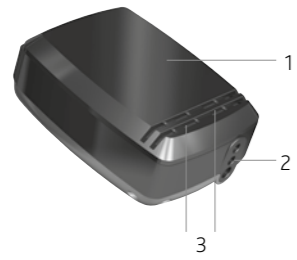
Hear now. And always

Cochlear®

1

2

3

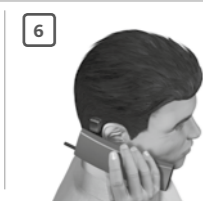
4

5

6

7

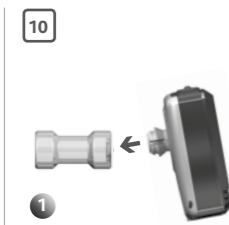
8

9

10

11

Series of horizontal lines for writing or notes.

Welcome

Congratulations on your choice of the Cochlear™ Baha® 4 Sound Processor. You are now ready to use Cochlear's highly advanced bone conduction sound processor, featuring sophisticated signal processing and wireless technology.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

Key to device *See figure* **1**

1. Battery compartment door
2. Direct Audio Input
3. Microphones
4. Volume up/down & Power on/off button
5. Plastic snap connector
6. Program button, Wireless audio streaming button
7. Volume up/down & Power on/off button
8. Attachment point for the safety line

Note on figures: The figures included on the cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

Introduction

You should have received your Baha sound processor from your hearing care professional along with the contents of this kit. To ensure optimal performance, your hearing care professional will fit the processor to suit your needs based on your connection configuration. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional.

Be sure to follow all applicable instructions in this manual based on your specific connection configuration.

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

Contacting customer service

We strive to provide the best possible products and services to our clients. Your views and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

Customer Service – Cochlear Americas

13059 E. Peakview Avenue,
Centennial, CO 80111, USA
Toll free (North America) 1800 523 5798

Tel: +1 303 790 9010, Fax: +1 303 792 9025
E-mail: customer@cochlear.com

Customer Service – Cochlear Europe

6 Dashwood Lang Road
Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom

Tel: +44 1932 87 1500, Fax: +44 1932 87 1526
E-mail: info@cochlear.co.uk

Customer Service – Cochlear Asia Pacific

1 University Avenue, Macquarie University
NSW 2109, Australia

Toll free (Australia) 1800 620 929
Toll free (New Zealand) 0800 444 819

Tel: +61 2 9428 6555, Fax: +61 2 9428 6352
or Toll free 1800 005 215

E-mail: customerservice@cochlear.com.au

Key to symbols

The following symbols will be used throughout these documents. Please refer to the list below for explanations:

	"Consult instructions for use" or "Operating instructions"
	"Caution" or "Caution, consult accompanying documents"
	Audible signal
	CE-mark
	Temperature limitation
	Interference risk
	1 Product name 2 Date of manufacture 3 Serial number
	Batch Code
	Serial Number
	Refer to instructions/booklet
	Wireless enabled Baha sound processor
	ACMA symbol

Using your sound processor

The sound processor is equipped with buttons for turning the sound processor on or off, selecting different listening programs, adjusting the volume level, and enabling wireless audio streaming. Audio indicators can be activated to alert you to the different settings and processor status.

Your sound processor is programmed to be used specifically on either the right or left side of your head. Your audiologist has marked your processor for left or right side usage.

On/off *See figure 2*

Turn on your sound processor by pressing and releasing either outer button. To turn off your sound processor, press and hold the volume down button for 2 seconds.

When your sound processor is turned off and then back on, it will return to the default setting (program one).

Audible status indicators *See figure 3*

Your sound processor is equipped with audible indicators (beeps).

Note: Your hearing care professional can disable the beeps if you prefer.

Changing the program *See figure 4*

Your hearing care professional has, after discussions with you, selected up to four programs for your Baha sound processor:

Program 1: _____

Program 2: _____

Program 3: _____

Program 4: _____

(These programs are suitable for different listening environments. Ask your audiologist to fill in your specific programs).

To switch programs, press and release the middle button. Audio signals let you know which program you have switched to:

Program 1: 1 beep

Program 2: 2 beeps

Program 3: 3 beeps

Program 4: 4 beeps

To activate wireless audio streaming, press and hold the middle button until an audio signal confirmation is heard. *See figure 4*

Adjusting volume *See figure 5*

Your hearing care professional has set the volume level for your sound processor. You can also manually adjust the volume control using the outer buttons. When the sound processor is placed on the head the front button is volume up and the rear button is volume down.

To control your listening comfort in different listening environments, you can adjust the volume:

- Press and release the volume up button to increase volume. Press and release the volume down button to decrease volume.
- A beep sounds each time a volume button is pressed. When you reach the minimum or maximum volume level, a long beep occurs.

Using the telephone *See figure 6*

For best results when talking on the telephone, place the receiver near the microphone inlet of the sound processor instead of near your ear canal. Make sure the receiver does not touch the sound processor as feedback may result.

Alternatively the optional Cochlear Wireless Phone Clip will allow you to stream phone calls and audio from your mobile phone to your sound processor.

Attaching the safety line *See figure 7*

Before connecting the sound processor, attach the safety line to the sound processor and clip the other end of the safety line to your shirt or jacket.

Cochlear recommends connecting the safety line when engaging in physical activities. Children should use the safety line at all times.

Accessories

Your sound processor can be used with the Cochlear Wireless Accessories. In addition, your sound processor has a DAI (Direct Audio Input) port that is used to connect via a cable directly to external audio sources, such as MP3 players (requires the use of the optional Cochlear Baha Audio adapter), or the Cochlear Baha Telecoil. Consult your hearing care professional for more information regarding the available accessories for your sound processor.

Changing batteries

Use one of the included batteries (zinc-air, non-rechargeable) as a replacement. The batteries included in the sound processor kit reflect Cochlear's latest recommendations. Contact your hearing care professional for additional batteries.

Battery tips

- Battery life decreases as soon as the battery is exposed to air (plastic strip is removed).
- Battery life depends on daily usage, volume setting, use of wireless audio streaming, sound environment, program setting, and battery strength.
- To maximize battery life, switch off the sound processor when not in use.
- If a battery leaks, immediately replace it.

Note: The inside of the battery door is lined with a moisture barrier that prevents dust and water from entering the sound processor. To ensure an effective seal, Cochlear recommends replacing the battery door every 6 months. In humid environments, replace the battery door more frequently.

Contact your hearing care professional for additional battery doors.

To replace the battery in Baha 4:

See figure 8

- 1
- 2 Gently slide the battery door downwards until you feel a 'click'. You can now remove the door.
- 3 With a fingernail, gently lift the battery hatch open to expose the battery.
- 4 Remove the old battery. Use the included magnetic removal tool if needed. Dispose of battery according to local regulations.
- 5 Remove the sticker on the + side of the new battery. Insert the new battery with the + sign facing the + marked on the processor's battery compartment.
- 6 Let the battery breathe for one full minute to build up necessary power.
- 7 Gently press the battery hatch closed.
- 8 Attach the battery door and slide it gently back into its locked position.

Flight mode

When boarding a flight, wireless functionality must be deactivated because radio signals may not be transmitted during flights. To disable wireless operation:

1. Turn off the sound processor by pressing and holding the volume down button for 2 seconds
2. Press and hold the program button and turn on the sound processor by pressing and releasing either volume button. An audio signal will confirm that the flight mode has been activated.

When in flight mode, wireless operation can be re-enabled by turning the processor off and on normally.

If the sound processor is turned off because the battery has been removed, it can be turned on in flight mode by holding down the program button and reinserting the battery.

Optional tamper-proof battery door *See figure 9*

To prevent the user from accidentally opening the battery door and exposing the battery, an optional tamper-proof battery door is available. This is particularly useful for children. The tamper-proof battery door should only be locked and unlocked using the specially designed tool included in the tamper-proof battery door kit to avoid accidental damage to the door. Contact your hearing care professional for tamper-proof battery doors.

To use the tamper-proof battery door:

1. Make sure the screw slot is positioned vertically.
2. Attach the battery door and slide it to its closed position.
3. Turn the screw a quarter-turn clockwise to lock the battery door in the horizontal position.
4. To unlock the battery door, turn the screw a quarter-turn anti-clockwise to vertical. The battery door can now be removed.

 **Caution:** Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children, those with mental challenges, and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency center.

General processor care

Your Baha sound processor is a delicate electrical instrument. Follow these guidelines to maintain its proper working order:

- If you will not be using your sound processor for any length of time, remove the battery door to let the sound processor ventilate.
- If you will not be using the sound processor for a long period, remove the battery.
- When not in use, turn off your sound processor and store it free from dust and dirt.
- During physical activities, secure your sound processor using the safety line. If the physical activity involves contact, Cochlear recommends removing your sound processor and protecting the implant site with a sweat band or similar.
- Remove your sound processor before applying hair spray, gel or using hair dye.

Using Share the Experience

See figure 10

Family members and friends can “share the experience” of bone conduction hearing. The test rod is used by others to simulate hearing with the Baha sound processor.

To use the test rod:

- 1 Snap the sound processor onto the test rod using the tilt technique. Hold the rod against the skull bone behind an ear. Plug both ears and introduce sound.
- 2 To avoid feedback (whistling), the sound processor should not touch anything other than the test rod.

Warnings

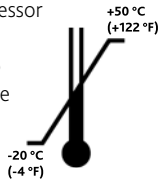
General advice

A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.

- Infrequent use of a sound processor may not permit a user to attain full benefit from it.
- The use of a sound processor is only part of hearing rehabilitation and may need to be supplemented by auditory training and instructions in lip-reading.

Warnings

- The sound processor is a digital, electrical, medical instrument designed for specific use. As such, due care and attention must be exercised by the user at all times.
- The sound processor is not waterproof!
- Never wear it in heavy rain, in the bath or shower!
- Do not expose the sound processor to extreme temperatures. The sound processor is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). In particular, battery performance deteriorates in temperatures under +5°C. The processor should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).
- Contains small parts that may present a choking hazard
- This product is not suitable for use in flammable and/or explosive environments.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.
- Portable and mobile RF (radio frequency) communications equipment can affect the performance of your sound processor.

- The sound processor is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.

- Interference may occur in the vicinity of equipment with the symbol to the right.
- Dispose of batteries and electronic items in accordance with your local regulations.
- Do not dispose of your sound processor in an incinerator or into fire.
- When wireless function is activated, the device uses low-powered digitally coded transmissions in order to communicate with other wireless devices. Although unlikely, nearby electronic devices may be affected. In that case, move the sound processor away from the affected electronic device.
- When using wireless functionality and the devices are affected by electromagnetic interference, move away from the source.
- Be careful when boarding flights to deactivate the wireless functionality.
- Turn off your wireless functionality by using the flight mode in areas where radio frequency emission is prohibited.
- Cochlear Baha wireless devices include a RF transmitter that operates in the range of 2.4 GHz - 2.48 GHz.
- For use of wireless functionality only use Cochlear Wireless accessories. For further guidance regarding e.g. pairing, please refer to the user guide of the relevant Cochlear Wireless accessory.
- No modification of this equipment is allowed.

If the sound processor becomes soaked with water:

1. Immediately open the battery door and remove the battery.
2. Put the sound processor in a container with drying capsules such as a Dri-aid kit or similar. Leave the processor to dry out overnight. Drying kits are available from most hearing care professionals.

Feedback (whistling) problems

See figure **11**

- 1 Check that items such as hats and glasses do not come in contact with the sound processor. Also, ensure that neither the head nor the ear come in contact with the sound processor.
- 2 Decrease the volume.
- 3 Check that the battery door is closed.

Sound processor type designations for models included in this User Manual are:

FCC ID QZ3BAHA4, IC: 8039C-BAHA4, IC model: Baha4.

Statement:

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

(1) this device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Changes or modifications can void the user's authority to operate the equipment .

Intended use

The Cochlear™ Baha® 4 Sound Processor uses direct bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

List of countries:

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

Specification of restrictions: You are not allowed to operate the equipment within 20 km of the center of Ny Ålesund, Norway.

The products are in compliance with the following regulatory requirements:

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