

## Pairing Guide (for Android™)




This guide tells you how to pair your Cochlear™ Nucleus 7 Sound Processor(s) with the Nucleus Smart App on a compatible Android smartphone\*, so you can use the app to control and monitor your sound processor(s).

**NOTE**  
To use the Android version of the app, your sound processor must have the latest firmware installed. Please check with your clinician.

## Pair your processor(s)

- NOTES**
- If you have other sound processor(s) already paired, you need to unpair them first. See page 4 for unpairing instructions.
  - Radio interference: when trying to pair or connect your sound processor with the app, you may experience interference from some devices (e.g. microwave ovens, wireless routers or TV Streamers). If this happens, move away from the device and try again.


- 1 Have your processor(s) ready.
- 2 Switch on Bluetooth on your Android smartphone.
- 3 Run **Nucleus Smart App**.
- 4 Swipe through the **Welcome** screens.
- 5 Create your Cochlear Account or login if you have an account.
- 6 The **Pair your processor(s)** screen displays. Tap **Continue**.
- 7 In the **Location** popup, tap **OK**.
- 8 In the **Allow Nucleus Smart** popup, tap **ALLOW**.

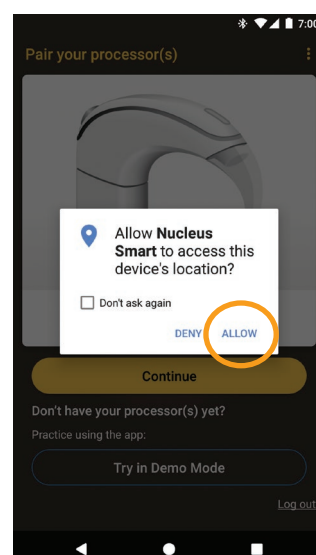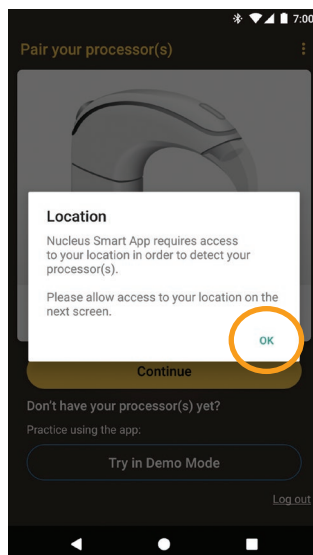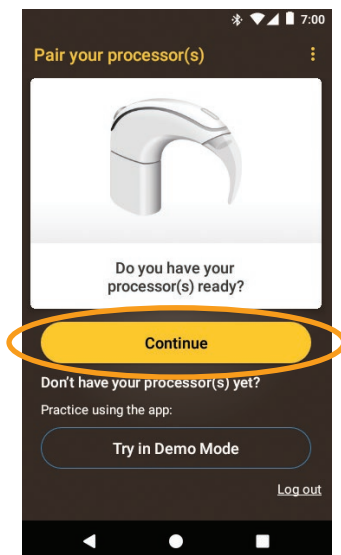


\* For compatibility information, refer the Nucleus Smart App user guide.

...CONTINUED OVER THE PAGE...


- 9 The **Detect your processor(s)** screen displays. Turn your processor(s) **OFF**, then **ON**. If you have two processors do both, one after the other. Tap **Continue**.

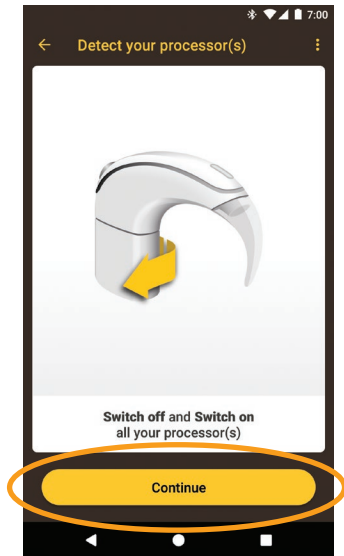


- 10 When your processor(s) display, tap once to start pairing.

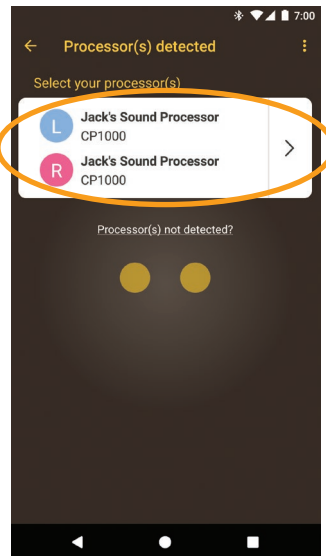


- 11 The **Paired processors** screen displays when pairing is complete. Tap **Continue**.

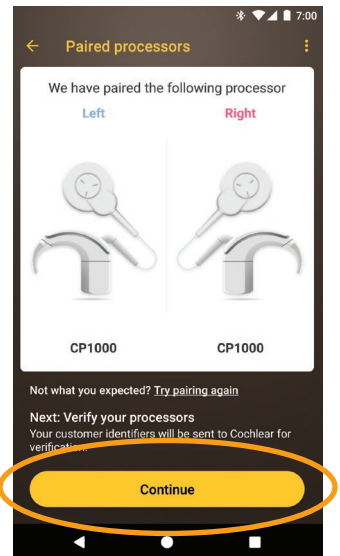


- 12 When the **Verify processor** screens display:
- Remove** your processor and **press** the processor button.
  - Place** your processor on your head and tap **Continue**.

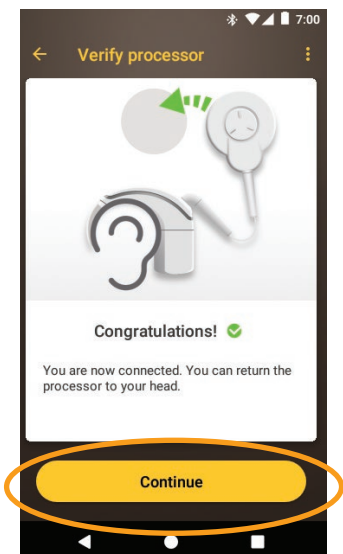


- 13 Tap an option in the **Sound Processor(s)** screen and if prompted, enter user details.

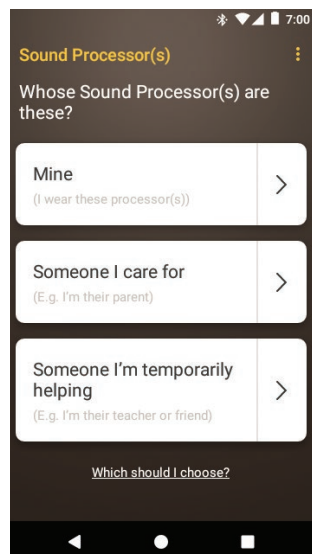

- 14 The **Enable data sync** screen displays. If you would like to use this feature, tap **Enable**.

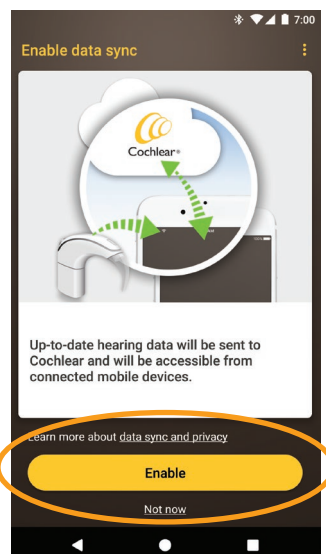

- 15 If the **Audio streaming** screen displays, see steps on the next page.  
or  
If the **Setup complete** screen displays, tap **Continue**.

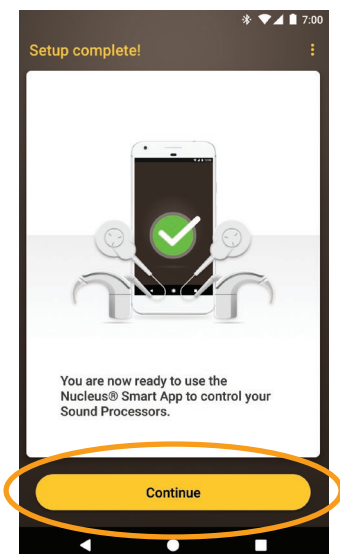

**NOTE**  
To pair additional compatible devices, you must first disable Bluetooth on existing paired devices.

# Smartphones with audio streaming for hearing aids


If your smartphone supports audio streaming for hearing aids, you can receive sound from your phone straight to your processor. To setup audio streaming:

- 1 When the **Audio streaming** screen displays, tap **Enable**. Your sound processor(s) will turn off for a few moments.




- 2 If other sound processor(s) are already paired or streaming to your smartphone you need to **Forget** them now. Follow the screen prompts to **Forget** the processor(s), then tap the **back button** to return to the Nucleus Smart App.


- 3 Wait while the **Connecting to processor** screen displays.

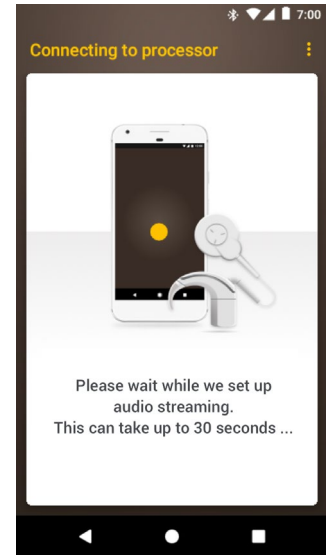

- 4 If you have **Touch sounds** enabled on your smartphone, the **Touch sounds** screen displays. Follow the screen prompts to **disable Touch sounds**, then tap the **back button** to return to the Nucleus Smart App.

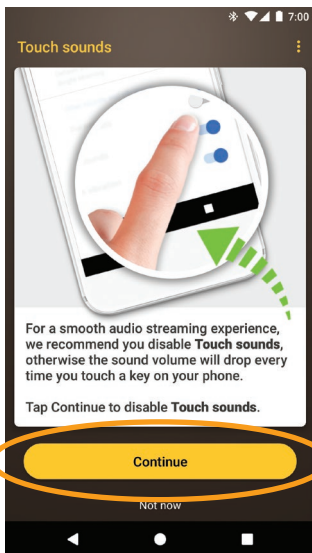

- 5 When the **Setup complete!** screen displays, tap **Continue**.

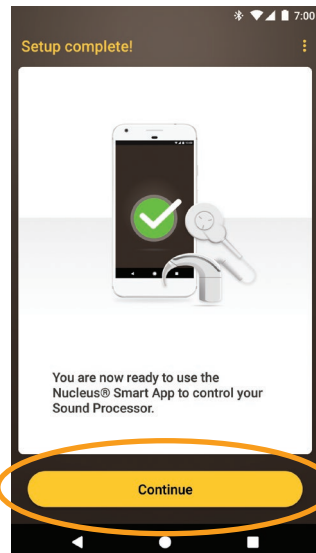

- 6 You can use the **Nucleus Smart App!** Refer to the *Nucleus Smart App User Guide* for details.

**NOTE**  
To pair additional compatible devices, you must first disable Bluetooth on existing paired devices.


# Unpair your processor(s)

If you change your sound processor(s), for example if you receive a new sound processor from the repair centre, you need to:


- **Unpair** the old sound processor(s), then
- **Pair** the new sound processor(s).

To unpair your processor(s):


1 Open the app's Menu.

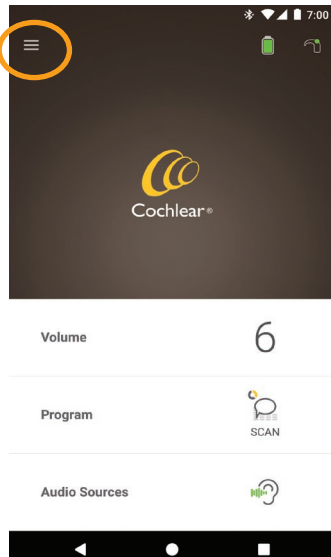


2 Tap Settings.

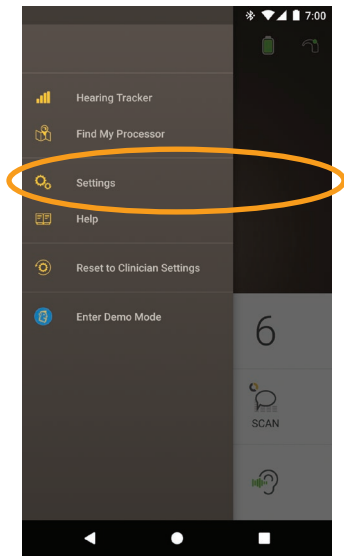


3 Tap Processor settings.

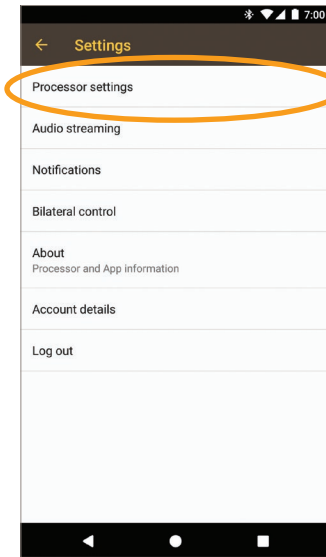


4 Tap Unpair your processor(s).

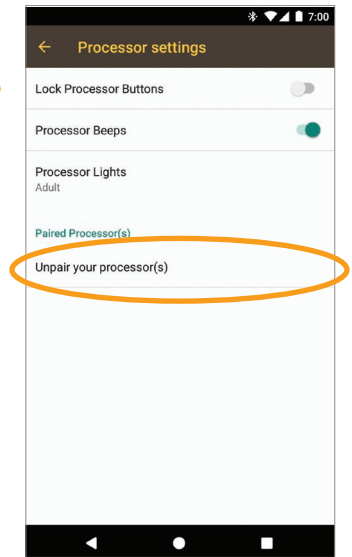


5 Tap YES.

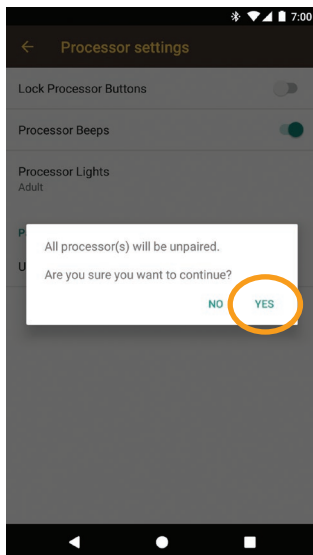

6 If streaming is enabled for your processor(s), a prompt to **disable streaming** displays.

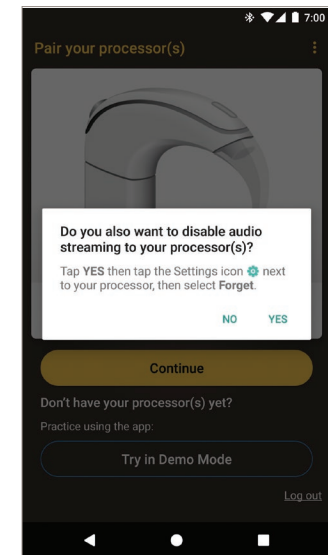


- A. Tap **YES** and follow the prompts to **Forget** your processor(s).
- B. Tap the **back button** to return to the Nucleus Smart App.

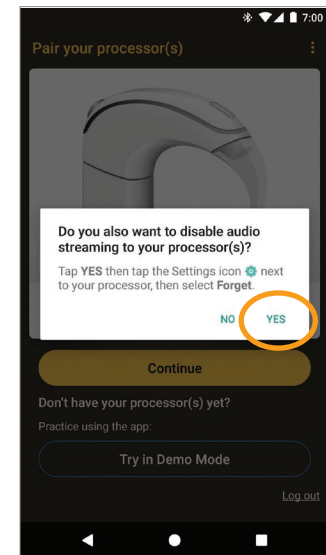


# Check streaming status

To check your streaming status:


1 Open the app's Menu.

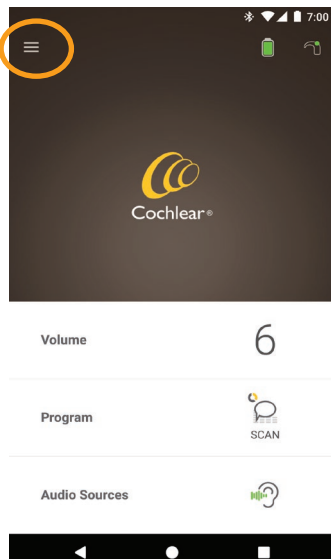

2 Tap Settings.

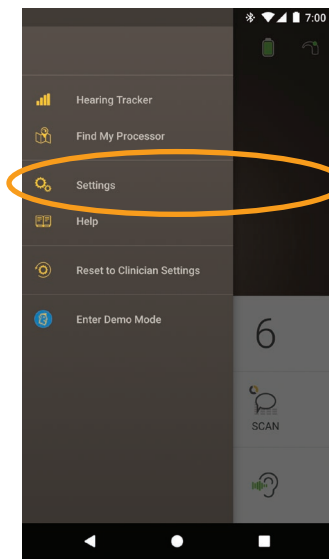

3 Tap Audio streaming.

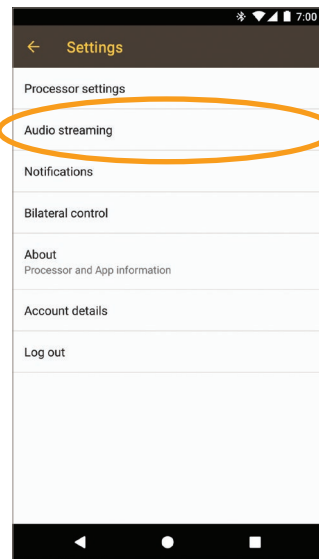

4 If streaming is **not active**, for more details read the information on screen and tap available links.

- For more information on the app, see the *Nucleus Smart App User Guide*.
- For more information on the sound processor, see the *Nucleus 7 Sound Processor User Guide*.
- Visit us online for tips, how-to videos and more: [www.cochlear.com/](http://www.cochlear.com/)

**Cochlear Ltd** (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia  
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352  
**Cochlear Ltd** (ABN 96 002 618 073) 14 Mars Road, Lane Cove, NSW 2066, Australia Tel: +61 2 9428 6555  
Fax: +61 2 9428 6352  
**CECOP Cochlear Deutschland GmbH & Co. KG** Karl-Wiechert-Allee 76A, 30625 Hannover, Germany  
Tel: +49 511 542 770 Fax: +49 11 542 7770  
**Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel:  
+44 1932 26 3400 Fax: +44 1932 26 3426  
[www.cochlear.com](http://www.cochlear.com)

Cochlear, Hear now. And always, Nucleus, and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited. Android is a trademark of Google Inc.