
Baha® SoundArc™

INSTRUCTIONS FOR USE

EN-GB	FI	LT	SK
AR	FR	LV	SL
BG	HE	MS	SR
CS	HR	NL	SV
DA	HU	NO	TH
DE	IS	PL	TR
EL	IT	PT	UK
ES	JA	PT-BR	ZH-CN
ET	KK	RO	ZH-TW
FA	KO	RU	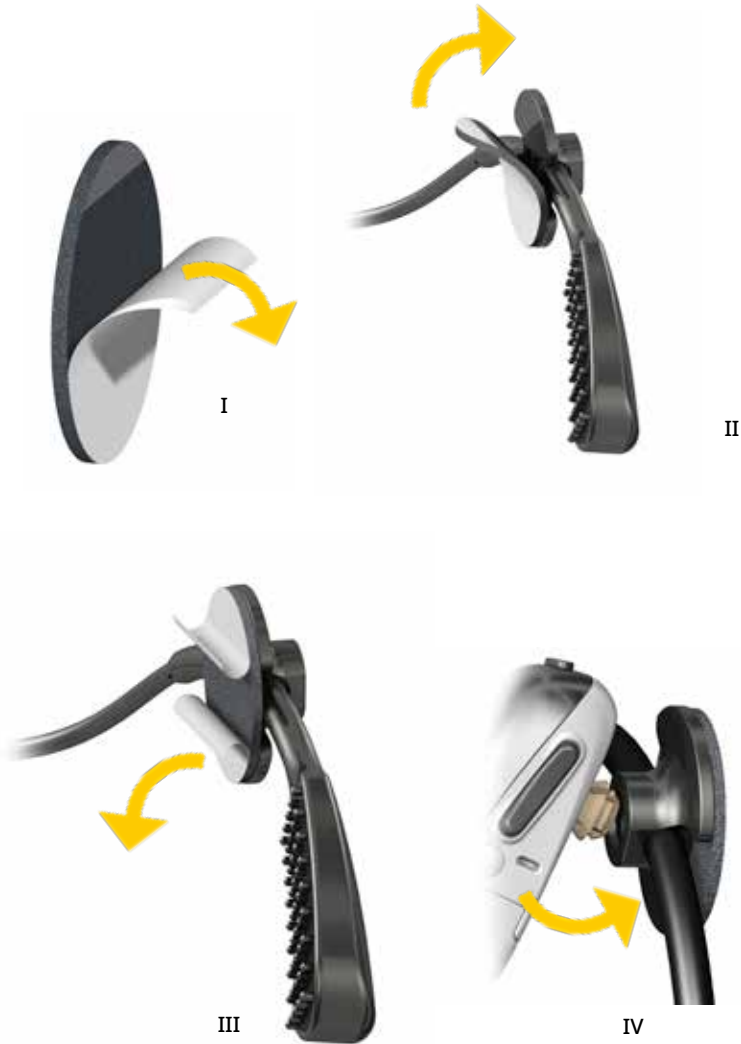

Contents

EN-GB.....	4	SK	29
AR	4	SL	30
BG.....	5	SR	31
CS	6	SV	32
DA	7	TH	33
DE	8	TR	33
EL	9	UK.....	34
ES	10	ZH-CN.....	35
ET	10	ZH-TW.....	36
FA	11		
FI.....	12		
FR	13		
HE.....	14		
HR.....	14		
HU.....	15		
IS.....	16		
IT.....	17		
JA.....	18		
KK	19		
KO.....	20		
LT	20		
LV	21		
MS.....	22		
NL.....	23		
NO.....	24		
PL	25		
PT	26		
PT-BR	26		
RO.....	27		
RU.....	28		

EN-GB Baha® SoundArc Instructions for use

Intended use: The Cochlear Baha System uses bone conduction to transmit sounds to the cochlea (inner ear) with the purpose of enhancing hearing. The Baha SoundArc is intended to be used as part of the Cochlear Baha System to function as retention and coupling mechanism for the Baha Sound Processor.

Indications: The Baha System is indicated for patients with conductive hearing loss, mixed hearing loss and single-sided sensorineural deafness (SSD).

Clinical Benefit: Most recipients of a bone conduction hearing solution will experience improved hearing performance and quality of life compared to unaided listening

The Baha® SoundArc is for long-term use as well as for test situations.

Notes: Soft pad(s) should always be used when wearing the Baha SoundArc. Replace the Soft pad(s) when worn or damaged. For the first few weeks the Baha SoundArc can be worn for shorter periods of time until the wearer has become accustomed to it. It is not made with natural rubber latex.

For bilateral use of the SoundArc a second connector disc is needed.

Cleaning: Use an alcohol wipe or similar germicidal wipe on the Baha SoundArc.

! Caution:

- Do not place the Baha SoundArc directly over the implant or abutment during the healing period, as this may interfere with the osseointegration process.
- Remove the sound processor(s) and the Soft pad(s) before washing or cleaning.
- Children should not be left unattended and should always have adult supervision when wearing the Baha SoundArc.

- If you experience persistent irritation or soreness of the skin, or if the Baha SoundArc gives you any discomfort, remove it and contact your hearing care professional.
- Contains small parts that may present a choking hazard or suffocation

Serious incidents: Serious incidents are rare, any serious incident in relation to your device should be reported to your Cochlear representative and to the medical device authority in your country, if available.

Warranty: The warranty does not cover defects or damage arising from, associated with, or related to the use of this Product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

	Medical device
	Keep dry
	Unique device identifier
Store at room temperature	Store at room temperature

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Tel: +46 31 792 44 00

Cochlear Ltd, (ABN 96 002 618 073), 1 University Avenue,

Macquarie University, NSW 2109 Australia

Tel: +61 2 9428 6555

Cochlear Americas, 10350 Park Meadows Drive, Lone Tree, CO 80124, USA

Tel: +1 303 790 9010

CHREP Cochlear AG, Peter Merian-Weg 4, 4052 Basel, Switzerland

Tel: +41 61 205 8204

Cochlear Latinoamerica, S. A., International Business Park

Building 3835, Office 403 Panama Pacifico, Panama

Tel: +507 830 6220

UK Responsible Person: Cochlear Europe Ltd, 6 Dashwood Lang Road,

Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom

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Rx Only

Medical
device

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information. In Australia, Baha bone conduction implant systems are intended for the treatment of moderate to profound hearing loss.

Cochlear, Baha, 科利耳, コクレア, 코클리어, Hear now. And always, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB or Cochlear Limited (unless otherwise noted).

© Cochlear Bone Anchored Solutions AB 2022. All rights reserved. 2022-06

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Cochlear Bone Anchored Solutions AB, Konstruktionsvägen 14,
Hizmetleri Ltd. Sti., Çubuklu Mah. Boğaziçi Cad., Boğaziçi Plaza No: 6/1,