



## Guide to enjoying television

Tips to help you tune into your favourite shows

### Get your technology to do the heavy lifting



#### Explore the benefits of the Cochlear™ Wireless TV Streamer

- Use the Cochlear Wireless TV Streamer to directly stream clear audio and media from your television to your sound processor. The TV streamer always remains paired. Simply plug into the power socket on your wall and directly into the audio input on your television. It is compatible with Nucleus® 7, Nucleus 6, Nucleus Kanso® and Baha® 5 Sound Processors.
- **Personalised listening:** You can set your own personal volume. You can also program the TV streamer so you can hear both the television and the environmental sounds around you\*.
- **Streaming tips:** If you or a family member have bilateral sound processors, set the setting on the TV Streamer to 'S' to stream in stereo. For one sound processor, use the 'M' setting to ensure that all the sound input is streamed to the processor.

\*Refer to your user guide for pairing instructions.



#### Put your other True Wireless™ accessories to work

- Did you know that other wireless accessories can help stream television audio directly to your sound processor too? Here's how you can make your other wireless accessories work with the TV:
  - **Mini Mic 2+:** Ensure that your Nucleus 7, Nucleus 6, Nucleus Kanso or Baha 5 Sound Processor is paired\* correctly. Then insert the 2.5 mm accessory cable that comes with your Mini Mic 2+ into the audio socket of your television and connect the other end to the Mini Mic 2+.
  - **Phone Clip:** If you are watching on a device that allows for Bluetooth® connectivity, you can use your Phone Clip to improve clarity. Ensure that your Nucleus 7, Nucleus 6, Nucleus Kanso or Baha 5 Sound Processor is paired\* correctly and that the Bluetooth functionality on your device is enabled. On your device, search for Bluetooth devices that are available and select "Hearing Aid Phone" from the list of Bluetooth devices.

# Practical advice for enjoying TV at home



## Optimise acoustics at home

- Check the acoustics of your home and consider introducing some soft furnishings into the room where you watch television. Remember, hard surfaces can create reverberation and distort sound quality.
  - For example, if the room you are in has tiled or timber floors you may find a rug will help reduce some of the reverberation. Or, if you have large windows in the room, adding curtains may help to improve your listening environment.



## Switch on ForwardFocus

- If you do not have wireless accessories, you can improve your viewing and listening experience by making simple adjustments to the program settings on your sound processor.
  - Use the Nucleus Smart App to enable Forward Focus on your Nucleus 7 Sound Processor. This will help to reduce environmental noise behind you, so you can better focus on the TV program in front of you.



## Turn on closed captioning

- Sometimes it can be hard for the brain to filter out speech from background music/noise in the program you are watching, especially if you have hearing loss. This can be exhausting, so make listening easier by enabling closed captioning. Streaming services offer closed captioning on most movies and TV shows.
  - **TIP:** Ensure that you have good lighting in the room, so that you can easily read the words on your TV screen.



## Stay connected with family and friends

- Stay connected with family and friends while you watch television. With **Netflix Party** you can watch movies and TV shows with your friends and family online by synchronising video playback and adding group chat to the shows you're watching. So, you don't need to even be in the same room, home or city to stay connected with your loved ones.

For more information contact your local customer service team.

## Hear now. And always

As the global leader in implantable hearing solutions, Cochlear is dedicated to helping people with moderate to profound hearing loss experience a life full of hearing. We have provided more than 600,000 implantable devices, helping people of all ages to hear and connect with life's opportunities.

We aim to give people the best lifelong hearing experience and access to innovative future technologies. We have the industry's best clinical, research and support networks.

That's why more people choose Cochlear than any other hearing implant company.

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use and refer to the relevant user guide for more information. Not all products are available in all countries. Please contact your local Cochlear representative for product information. Consult your health professional to determine if you are a candidate for Cochlear technology.

Cochlear, Hear now. And always, the elliptical logo, True Wireless, Nucleus, Kanso and Baha are either trademarks or registered trademarks of Cochlear Limited or Cochlear Bone Anchored Solutions AB (unless otherwise noted). © Cochlear Limited 2020.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Cochlear Limited is under license. The Cochlear Nucleus Smart App is available on App Store and Google Play. For compatibility information visit [www.cochlear.com/compatibility](http://www.cochlear.com/compatibility)

[www.cochlear.com](http://www.cochlear.com)

**Cochlear Ltd** (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555

**Cochlear NZ Limited** Level 4, Takapuna Towers, 19-21 Como Street, Takapuna, Auckland 0622 New Zealand Tel: +64 9 914 1983

**株式会社日本コクレア (Nihon Cochlear Co Ltd)** 〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-ku, Tokyo 113-0033 Japan Tel: +81 3 3817 0241

**Cochlear (HK) Ltd** Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong Tel: +852 2530 5773

**Cochlear Medical Device (Beijing) Co Ltd** Unit 2608-2617, Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing, 100022, P.R. China Tel: +86 10 5909 7800

**Cochlear Limited (Singapore Branch)** 238A Thomson Road, #25-06, Novena Square Office Tower A, Singapore 307684 Tel: +65 6553 3814

**Cochlear Korea Ltd** 1st floor, Cheongwon Building 33, Teheran-ro 8-gil, Gangnam-gu, Seoul, Korea Tel: +82 2 533 4450

**Cochlear Medical Device Company India Private Ltd** G Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai - 400 051, India Tel: +91 22 6112 1111



D1726789 APR20