

Cochlear™ Baha® Remote Control 2

Product description

The Cochlear™ Baha® Remote Control 2 is for patients who appreciate an easy overview of their sound processor and accessory settings. They can use it to change programs, adjust the volume and control streaming from the Cochlear Wireless TV Streamer and Cochlear Wireless Mini Microphone.

The Remote Control allows the patient to adjust the volume both monaurally and binaurally. When in streaming mode, they can individually adjust volume of the sound processor versus the streamed audio volume.

The Remote Control is more than just buttons. It has a high contrast LCD display where your patients can see what program they're in and the volume level. With the simple keypad, it's easy to choose programs, control volume and mute sound processors. This small remote makes it easy to stay in control.

Easy to connect

- To pair the Remote Control to the sound processor, simply press the pairing button on the Remote Control and turn on the sound processor within 20 seconds, and it automatically pairs with the sound processor.
- To pair bilateral sound processors, press the pairing button on the Remote Control and insert the batteries of the sound processors at the same time.
- Pairing can also be done through the Cochlear Baha Fitting Software.

Easy to use

- At the push of a button you can control volume, mute/unmute and change programs.
- Volume can be adjusted monaurally as well as binaurally.
- 'Home' button resets the sound processor to your default settings.
- The 'Streaming' button activates streaming from your TV Streamer or Mini Microphone (optional streaming devices).
- Built-in rechargeable battery with status indicator.
- Sound Processor low battery warning indication.
- Mechanical On/Off switch to avoid unintentional activation.

2.4 GHz wireless technology


- Communication between the Remote Control and the sound processor uses 2.4 GHz wireless technology; a smart, robust and reliable standard that has a strong, clear and stable signal.



Cochlear™ Baha® Remote Control 2

Description

Article number: 94791 Cochlear Baha Remote Control 2

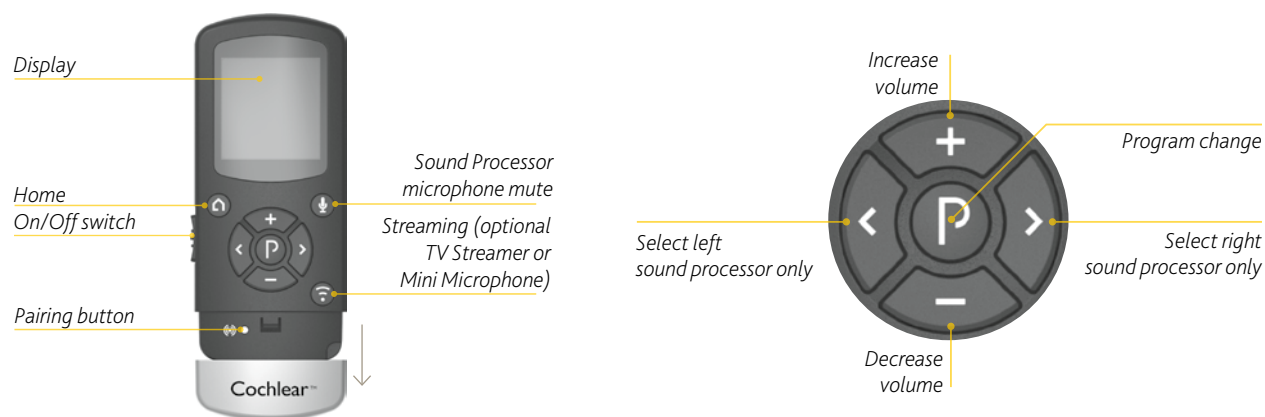

NOTE: Images are not to scale.

Technical information

Dimensions	98 x 42 x 15 mm
Weight	35 g
Display	128 x 128 pixel colour display
Batteries	Built-in rechargeable battery
Battery charge time	Approximately 2.5 hours when using the charger supplied with the device
Wireless connections	2 sound processors per Remote Control
Wireless frequency	2.4 GHz
Wireless range	Up to 9 feet
Communication speed	2 Mbit/s
Emission	Maximum +4 dBm. In compliance with all relevant standards
ESD tolerance	±4 kV contact ±8 kV air
Operating temperature	0 to +55 °C
Storage temperature	-20 to +60 °C

Not everyone with hearing loss is a candidate for a Baha. All surgical procedures include an element of risk, and it is impossible to guarantee success. For complete information regarding the risks and benefits of a Baha procedure, please refer to the Instructions for use for the Baha implant available at www.Cochlear.com/US/BahaIndications

www.Cochlear.com/US

Cochlear Americas
13059 East Peakview Avenue
Centennial, CO 80111 USA
Telephone: 1 303 790 9010
Support: 1 800 483 3123

Cochlear Canada Inc.
2500-120 Adelaide Street West
Toronto, ON M5H 1T1 Canada
Telephone: 1 416 972 5082
Fax: 1 416 972 5083

BUN041 ISS2 AUG14



©Cochlear Limited 2014. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.

Hear now. And always

