Cochlear[™] Aqua+

User Guide

EN

ID

KO

MS

TH

VI

ZH-TW

Contents

EN	Cochlear™ Aqua+ User Guide	4
ID	Panduan bagi Pengguna Aqua+ Cochlear™	32
KO	Cochlear™ Aqua+ 사용 안내서	60
MS	Cochlear™ Aqua+ Panduan Pengguna	88
TH	คู่มือผู้ใช้ Cochlear™ Aqua+	116
VI	Cochlear™ Aqua+ Hướng dẫn sử dụng	144
ZH-TW	Cochlear™ (科利耳®) Aqua+ 使用指南	172

Symbols used in this document

Note: Important information or advice.

Tip: Time saving hint.

Caution: Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

Warning: Potential safety hazards and serious adverse reactions. Could cause harm to person.

Contents

Symbols used in this document.....	4
About this guide	6
Cochlear Aqua+	7
Introduction	8
Intended purpose	10
Indications	11
Contraindications.....	13
Intended patient population.....	14
Benefits.....	14
Intended users	14
Cochlear Aqua+ instructions.....	15
Aqua+ coil magnet instructions	19
Safety Line instructions.....	20
Mic Lock-Stirrup instructions.....	21
Care and storage	23
Cautions.....	25
Warnings.....	25
Other information	27
Environmental conditions	27
Environmental protection	27
Labelling symbols	28
Serious incidents.....	30
Warranty.....	31
Legal statement	31
Trademark legal notice	31

About this guide

This guide is intended for hearing implant recipients and their carers using the Cochlear™ Aqua+.

Note

Refer to the relevant sections for cautions and warnings relating to the use of the Cochlear Aqua+.

Please also refer to your *Important Information* document for essential advice that applies to Cochlear implant systems.

Cochlear Aqua+

1. Cochlear Aqua+
2. Cochlear Nucleus 8 Aqua+ Coil
3. Nucleus Safety Line (optional)
4. Nucleus Mic Lock-Stirrup (optional)

Introduction

The Cochlear™ Aqua+ are re-usable sealable covers that, when used with the Cochlear™ Nucleus® 8 Aqua+ Coil, keep your Nucleus 8 Sound Processor (model number: CP11110) dry during use in or around water (e.g. when swimming or in the bath). It is worn behind the ear with your sound processor sealed inside.

The Nucleus 8 Aqua+ should only be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

The Nucleus 8 Aqua+ comes in two sizes:

- Cochlear Aqua+, Power Extend, to fit the Cochlear Power Extend Battery Module
- Cochlear Aqua+, Compact, to fit the Cochlear Compact Battery Module

You must use a Cochlear Power Extend Battery Module or Cochlear Compact Battery Module for your sound processor to work inside Aqua+. Zinc air batteries will not work as they need air and there is no air inside Aqua+ when it's sealed.

The Cochlear Nucleus 8 Aqua+ Coil is used in place of your usual coil while your sound processor is inside your Aqua+. It has a special plug that creates a seal when connected to your sound processor's coil socket through the coil plug hole in Aqua+. As with your usual coil, a coil magnet screws into the Aqua+ coil and holds it against your implant.

Note

You cannot use your Nucleus 8 Sound Processor in Hybrid™ mode with Aqua+.

You cannot use your Nucleus 8 Sound Processor with any of the Cochlear retention accessories (e.g. Snugfit, Hugfit) with Aqua+.

The Nucleus Mic Lock-Stirrup is a retention option to hold Aqua+ more securely behind your ear. The stirrup connects to the base of Aqua+ and the Mic Lock tube (when cut to size) passes under the ear lobe and connects to the Aqua+ earhook.

The Nucleus Safety Line is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

While using Aqua+ you can also use the following, provided they are protected from water and kept dry:

- Your smartphone (with the Nucleus Smart App)
- Cochlear CR310 Remote Control
- Cochlear wireless accessories.

You may notice a difference in sound quality when using Aqua+. Testing shows Aqua+ can be:

- Used at depths of up to 3 metres for up to 2 hours.
- Re-used up to 50 times, but this may vary according to how you use it.

There have been no reported undesirable side effects from using Aqua+.

For more information, refer to the *Nucleus 8 Sound Processor User Guide*.

Intended purpose

Device	Intended purpose
Cochlear™ Aqua+, Power Extend Cochlear™ Aqua+, Compact	The Aqua+ is intended to provide improved water and dust resistance to the sound processor.
Cochlear Nucleus 8 Aqua+ Coil Cochlear Nucleus 8 Aqua+ Coil (5(I))	The coil is intended to be used in combination with other devices as part of a hearing implant system to supply electrical signals and power between a processing unit and a hearing implant.

Indications

Cochlear Aqua+

The Cochlear™ Aqua+ is indicated for Cochlear Nucleus implant recipients with a compatible Cochlear sound processor.

Compatible Cochlear sound processors are as follows (when used with a rechargeable battery module):

- Nucleus 8 Sound Processor

Cochlear Nucleus 8 Aqua+ Coil

The Cochlear Nucleus 8 Aqua+ Coil is indicated for a recipient with a compatible Cochlear Nucleus® implant, a compatible Nucleus processing unit and compatible Cochlear magnet.

Compatible Cochlear Nucleus implants are:

- CI600 Series Implants : CI612, CI622, CI624, CI632
- CI500 Series Implants : CI512, CI513, CI522, CI532, CI551, ABI541
- CI24RE Series Implants: CI422, CI24REH (Hybrid L24), CI24RE (CA), CI24RE (ST), CI24RE(CS), CI24RES (Hybrid S8 (CI8REH), Hybrid S12 (CI12REH))
- CI24R Series Implants: CI24R (CA), CI24R (CS), CI24R (ST)
- CI24M Series Implants: CI24M, ABI24M, CI 11+11+2M, CI24MH (CI6+16+2M)

Compatible processing unit is the Cochlear Nucleus 8 Processing Unit.

Compatible magnets are Cochlear Magnet strength ½ to 6, ½(I) to 4(I) and Cochlear Magnet Reverse polarity strength ½ to 6.

Cochlear Nucleus 8 Aqua+ Coil (5(I))

The Cochlear Nucleus 8 Aqua+ Coil (5(I)) is indicated for a recipient with a compatible Cochlear Nucleus implant, a compatible Nucleus processing unit and compatible Cochlear magnet.

Compatible Cochlear Nucleus implants are:

- CI600 Series Implants : CI612, CI622, CI624, CI632

Compatible processing unit is the Nucleus 8 Processing Unit.

Compatible magnets are Cochlear Magnet strength 5(I).

Contraindications

Cochlear Aqua+

The Aqua+ is not indicated for use with processing units other than those listed in Indications. The Aqua+ is not indicated for use with the Cochlear Battery Cover (with zinc air batteries).

Cochlear Nucleus 8 Aqua+ Coil

The Cochlear Nucleus 8 Aqua+ Coil is not indicated for use with hearing implants or processing units other than those listed in Indications.

The Cochlear Nucleus 8 Aqua+ Coil is not indicated for use with Cochlear Magnet strength 5(I).

Cochlear Nucleus 8 Aqua+ Coil (5(I))

The Cochlear Nucleus 8 Aqua+ Coil (5(I)) is not indicated for use with hearing implants or processing units other than those listed in Indications.

The Cochlear Nucleus 8 Aqua+ Coil (5(I)) is not indicated for use with Cochlear Magnet strength $\frac{1}{2}$ to 6 and $\frac{1}{2}$ (I) to 4(I) and Cochlear Magnet Reverse Polarity strength $\frac{1}{2}$ to 6.

Intended patient population

The sound processor is intended for patients implanted with a compatible Cochlear Nucleus implant. There are no restrictions for the intended patient population of the sound processor and accessories in terms of age, weight, health or other condition.

Benefits

The sound processor operates in conjunction with a compatible Cochlear Nucleus implant system.

Potential benefits of receiving a Cochlear Nucleus implant system include:

- Better understanding of speech in quiet.
- Better understanding of speech in noise.
- Increased satisfaction based on hearing capabilities.

Intended users

Intended users of the Cochlear Aqua+ are:

- Recipients of a compatible Cochlear Nucleus implant.
- Carers of recipients, who carry out recipient functions as needed. Carers may include parents of paediatric recipients who are less than 12 years old, and nurses or other carers of otherwise dependant recipients.
- Audiologists/clinicians.

Cochlear Aqua+ instructions

Tip: Don't forget you must be using a rechargeable battery for your sound processor to work inside Aqua+.

1. Carefully unplug your usual coil from the sound processor. Hold the coil cable grip and firmly pull it straight out of the processor.

Caution: Do not pull on the coil cable or twist when you pull it out.

2. Pull up on the earhook to remove it.

3. Insert your sound processor into Aqua+.

4. Insert the plug into the base of Aqua+. To make a good seal, push the plug in as far as it will go.

5. Grip the black side ribs of the coil connector and push the blue Aqua+ coil (with magnet) straight into your sound processor through the hole in Aqua+ until it clicks into place. **Do not twist.**

(See page 19 for instructions)

6. If you're concerned about losing your Aqua+, use one of these retention options:

Safety Line clips to your clothing.

(See page 20 for instructions)

Mic Lock-Stirrup holds your processor on your ear.

(See page 21 for instructions)

7. Finally, do the **Aqua+ Checklist** to make sure you're ready!

- Using Blue Aqua+ coil

- Using rechargeable batteries

- Aqua+ tightly sealed


- Mic Lock-Stirrup or Safety Line attached if required

Caution: Remember, you can't go deeper than 3 metres.

Aqua+ coil magnet instructions

1. Insert the Cochlear Magnet into the hole in the blue Aqua+ coil.

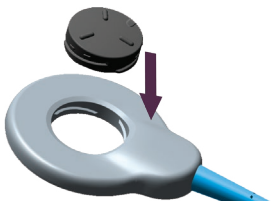2. Turn the magnet clockwise until it stops.

3. Turn the magnet a little more until you feel a click.

Tip: The tamper-resistant lock marker aligns with the coil cable when locked.

Safety Line instructions


Warning: Retention lines longer than the Safety Line (short length) are not recommended for use by children under 3 years as they may present a risk of strangulation.

1. Thread the Safety Line loop end through the Aqua+ ring.

2. Create a knot by passing the clip end of the line through the line loop. Pull it tight.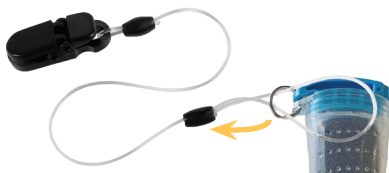

3. Lift the lever to open the clip. Place the clip over your clothing and press the lever down to close the clip.


Mic Lock-Stirrup instructions

Before using Mic Lock-Stirrup for the first time, cut it to the right length to suit your ear.

1. Snap the stirrup into the two dimples at the base of Aqua+.

2. Find the right length of tubing to hold Aqua+ in place comfortably behind your ear. Cut the tubing to the required length.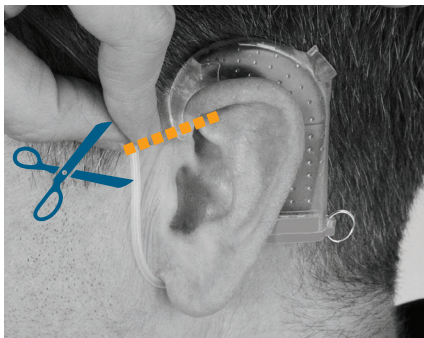

3. Push the cut end of the tubing over the earhook, as far as you can.

4. Wear your Aqua+ with Mic Lock-Stirrup.

Tip: Once fitted, you can leave your Mic Lock-Stirrup on your Aqua+.

Care and storage

1. Rinse Aqua+ and Aqua+ coil with clean fresh water and dry with a soft cloth.
2. Unplug the blue Aqua+ coil. Grip the black side ribs of the coil connector. Pull the plug straight out.

Caution: Do not pull on the coil cable or twist when you pull it out.

3. Open Aqua+ and remove your sound processor.

4. Re-attach your usual coil. Push your coil cable into the processor until it clicks. **Do not twist.**

5. Re-attach your usual earhook. Click it into place.

6. Place the Aqua+ and the blue Aqua+ coil in your dry aid kit overnight.

7. Once they're thoroughly dry, store Aqua+, the blue Aqua+ coil and any accessories in the Nucleus 8 Activity Kit case.

Cautions

- Always use Aqua+ with the blue Aqua+ coil, never with your usual coil.
- When connecting the coil cable, always push the plug straight in; do not screw or twist the plug.
- Where possible, avoid pressing the processor button with your fingernail as this can damage your Aqua+.
- If your Aqua+ appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.

Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Small parts (e.g. magnets, Safety Line) may be a choking or strangulation hazard.
- Sound processors and related accessories contain small parts that alone or in combination may pose a hazard of inhalation, choking or ingestion. Swallowing or inhaling small parts can cause severe or fatal injuries. Use the tamper-resistant locks to keep small parts attached to the processing unit where available, including parts that fit between the processing unit and battery module. Always supervise children under 3 years of age and others who may be at risk of inhalation, choking or ingestion of small parts when they use the sound processor and related accessories. When not in use, keep small parts, and combinations of small parts, out of reach from children. If small parts are swallowed or inhaled, seek immediate medical attention.

- The Aqua+ and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Aqua+ and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.
- Exposure of the sound processor and Aqua+ to temperatures above the operating range (+5° C – +40° C, +41° F – +104° F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.
- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- Store spare magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

Other information

Environmental conditions

Condition	Minimum	Maximum
Storage and transport temperature	-10° C (+14° F)	+55° C (+131° F)
Storage and transport humidity	0% RH	90% RH
Operating temperature (sound processor)	+5° C (+41° F)	+40° C (+104° F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

The temperature of the sound processor and its accessories may rise by up to 2.7° C (4.86° F) during normal operation, and could result in these components reaching a temperature of +42.7° C (+108.86° F) when operated at the maximum environmental temperature of +40.0° C (+104° F).

Environmental protection

Your Aqua+ coil contains electronic components subject to the Directive 2002/96/EC on waste electrical and electronic equipment. Help protect the environment by not disposing of your Aqua+ coil with your unsorted household waste. Please recycle your Aqua+ coil according to your local regulations.

Labelling symbols

The following symbols may appear on your sound processor or accessories and/or packaging:

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Authorised representative in the European Community

Unique Device Identification

Medical Device

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark with notified body number

CE registration mark

Rx Only

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

IP68

Ingress Protection Rating:

- Protected against access of solid foreign objects greater than or equal to 1.0 mm diameter.
- Protected against failure from dust penetration
- Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

Serious incidents

Whilst serious incidents in relation to medical devices are rare, it is acknowledged that incidents may happen. As an organisation, Cochlear recognises the potential for harm and will respond to any reported serious incident.

What is a serious incident?

A 'serious incident' means any event that directly or indirectly has caused or could have caused an unexpected or unwanted event including any of the following:

- The death of a patient, user or other person,
- The temporary or permanent serious deterioration of a patient's, user's or other person's state of health,
- A serious public health threat.

Reporting a serious incident

There is no definitive list of events/incidents that constitute a serious incident, however all serious incidents should be reported to:

- Your local Cochlear office
www.cochlear.com/intl/contact/global-offices

People within the European Union should also report all serious incidents to:

- Your National Competent Authority
http://ec.europa.eu/growth/sectors/medical-devices/contacts_en

People within Australia should also report all serious incidents to:

- Therapeutic Goods Administration
<https://www.tga.gov.au>

Warranty

Your Nucleus Global Limited Warranty will not be void in circumstances where Cochlear Aqua+ is used in water with Nucleus 8 Sound Processors, in accordance with the Aqua+ instructions.

Legal statement

The statements made in this version of the User Guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

© Copyright 2022 Cochlear Limited. All rights reserved.

Trademark legal notice

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, 콘트우아, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield and Xidium are either trademarks or registered trademarks of the Cochlear group of companies.

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- Administrasi Barang-Barang Terapi (Therapeutic Goods Administration) <https://www.tga.gov.au>

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© Hak Cipta 2022 Cochlear Limited. Hak cipta dilindungi undang-undang.

Pemberitahuan hukum merek dagang

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, logo elipsnya, Vistafix, Whisper, WindShield dan Xidium adalah merek dagang atau merek dagang terdaftar milik grup perusahaan Cochlear.

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, HugOfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, 타원형 로고, Vistafix, Whisper, WindShield 및 Xidium은 코클리어 그룹의 상표 또는 등록 상표입니다.

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© Hak cipta 2022 Cochlear Limited. Hak cipta terpelihara.

Notis undang-undang tanda dagangan

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, logo elips, Vistafix, Whisper, WindShield dan Xidium ialah tanda dagangan atau tanda dagangan berdaftar syarikat kumpulan Cochlear.

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield และ Xidium เป็นเครื่องหมายการค้า หรือเครื่องหมายการค้าจดทะเบียนของกลุ่มบริษัท Cochlear

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ACE, Advance Off-Stylet, AOS, Ardiium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, logo hình elip, Vistafix, Whisper, WindShield và Xidium là các nhãn hiệu thương mại hoặc nhãn hiệu thương mại đã đăng ký của tập đoàn Cochlear.

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Cochlear Canada Inc 2500-120 Adelaide
Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear Europe Ltd 6 Dashwood Lang
Road, Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

Cochlear Benelux NV Schaliënhoeverdreef
20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

Cochlear France S.A.S. 135 Route de
Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805
200 016 (National) Fax: +33 5 34 63 85 80

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Cochlear (HK) Limited Room 1404-1406,
14/F, Leighton Centre, 77 Leighton Road,
Causeway Bay, Hong Kong
Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Korea Ltd 2nd Floor, Yongsan
Centreville Asterium, 25, Hangang-daero
30 gil, Yongsan-gu, Seoul, Korea (04386)
Tel: +82 2 533 4450 Fax: +82 2 533 8408

Cochlear Medical Device (Beijing) Co., Ltd
Unit 2608-2617, 26th Floor, No.9 Building,
No.91 Jianguo Road, Chaoyang District,
Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

**Cochlear Medical Device Company India
Pvt. Ltd** Ground Floor, Platina Building, Plot
No C-59, G-Block, Bandra Kurla Complex,
Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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Cochlear Middle East FZ-LLC Dubai
Healthcare City, Al Razi Building 64, Block A,
Ground Floor, Offices IR1 and IR2, Dubai,
United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

Cochlear Latinoamérica S.A. International
Business Park, Building 3835, Office 403,
Panama Pacífico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

Cochlear NZ Limited Level 4, Takapuna
Towers, 19-21 Como St, Takapuna, Auckland
0622, New Zealand
Tel: + 64 9 914 1983 Fax: 0800 886 036

P2007047-D2007048 V1

Multilingual version of D1888143 V2 2022-04

Cochlear®

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© Copyright 2022 Cochlear Limited. สงวนลิขสิทธิ์