Cochlear™ Chargers User Guide

Cochlear™ Nucleus® Kanso 3® Nexa™
Sound Processor

Cochlear™ Nucleus® Kanso 3®
Sound Processor

Cochlear™ Nucleus® Kanso 2®
Sound Processor

Cochlear™ Surgical Processor

EN

ES

FR-CA

Contents

EN	Cochlear™ Chargers User Guide	1
ES	Guía de usuario de cargadores Cochlear™	33
FR-CA	Mode d'emploi des chargeurs Cochlear™	65

Symbols used in this document

Note: Important information or advice.

Tip: Time saving hint.

Caution (no harm):

Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

Warning (harmful):

Potential safety hazards and serious adverse reactions. Could cause harm to person.

Contents

Symbols used in this document.....	ii
About this guide	2
Cochlear Home Charger.....	3
Cochlear Portable Charger	3
Charger compatibility	4
Power	5
Charge your sound processor.....	5
Use the Home Charger	6
Use the Portable Charger	9
Regular care	17
Troubleshooting	18
Cautions.....	19
Warnings.....	20
Physical configuration	21
Home Charger	21
Portable Charger.....	23
Environmental conditions	25
Other information	26

About this guide

This guide is intended for people who use the:

- Cochlear™ Home Charger
- Cochlear™ Portable Charger.

Note

- Refer to the Cautions and Warnings sections for safety advice relating to the use of the chargers, accessories and components.
- Please refer to your *Important Information for Recipients* document for essential advice that applies to Cochlear implant systems.
- Please refer to the relevant sound processor user guide for more information about the sound processor.

Warning: Sound processors and related accessories contain small parts that alone or in combination may pose a hazard of inhalation, choking or ingestion. Swallowing or inhaling small parts can cause severe or fatal injuries.

Always supervise children under 3 years of age and others who may be at risk of inhalation, choking or ingestion of small parts when they use the sound processor and related accessories. When not in use, keep small parts, and combinations of small parts, out of reach from children. If small parts are swallowed or inhaled, seek immediate medical attention.

Cochlear Home Charger

Cochlear Portable Charger

Charger compatibility

Charger	Sound Processor
Cochlear Home Charger	• Cochlear™ Nucleus® Kanso® 3 Nexa™ Sound Processor (Model: CP1175) • Cochlear™ Nucleus® Kanso 3 Sound Processor (Model: CP1170) • Cochlear™ Nucleus® Kanso 2 Sound Processor (Model: CP1150) • Cochlear™ Surgical Processor (Model: CP1150S)
Cochlear Portable Charger	• Cochlear™ Nucleus® Kanso® 3 Nexa™ Sound Processor (Model: CP1175) • Cochlear™ Nucleus® Kanso 3 Sound Processor (Model: CP1170) • Cochlear™ Nucleus® Kanso 2 Sound Processor (Model: CP1150)

Power

Charge your sound processor

Your sound processor has an internal battery that needs regular charging.

Please charge your sound processor as soon as you receive it.

There are two methods to charge your sound processor's internal battery:

- Place it in the **Home Charger** (refer to page 6). The Home Charger is a dedicated charging dock. It integrates charging capability with the ability to dry the sound processor, or
- Attach it to the **Portable Charger** (refer to page 9). The Portable Charger provides a method of recharging the sound processor while you can continue to use the sound processor. The Portable Charger connects directly to the sound processor via a charger cable.

Before using the Portable Charger for the first time you must charge it. Refer to *Charge the Portable Charger* on page 11.

Caution: To charge your sound processor:

- use Cochlear equipment only
- do not use non-Cochlear equipment.

Caution: To optimise battery performance, avoid charging your sound processor in high ambient temperatures.

Use the Home Charger

The Home Charger will store, dry and charge your sound processor.

To fully charge a depleted sound processor takes approximately 4½ hours.

The Home Charger equipment includes:

- Home Charger unit
- Power adapter
- USB cable to connect the power adapter to the charger.

Note: For optimum performance, Cochlear recommends to connect the Home Charger to a power outlet using the power adapter provided.

To use the Home Charger to charge your sound processor:

1. **Plug** the USB cable connector into the power adapter.

2. **Plug** the other end of the cable into the power connector on the back of the Home Charger.

3. **Plug** the power adapter into a power outlet and **switch** the power on.

When power is on, the Home Charger power light displays **steady green**.

4. **Open** the lid and **place** your sound processor inside.

Caution: Do not press down on the sound processor.

Note: Safety lines and SoftWear Pads can remain attached to the sound processor while in the Home Charger.

5. **Close** the lid.

As your sound processor begins to charge, the light **pulses green**.

6. When fully charged, the sound processor light displays **steady green**.

Open the lid and remove the sound processor.

Indicator lights

What it means

Orange flashes

Problem with power or temperature.

Steady orange

Charger is faulty. Contact your clinician to arrange a replacement charger.

Use the Portable Charger

You can carry the Portable Charger¹ and use it as needed to charge your sound processor. The Portable Charger contains an internal battery that requires charging before use.

You must charge the Portable Charger before first use.

Refer to *Charge the Portable Charger* on page 11.

The Portable Charger takes approximately 3 hours to fully charge. It can be connected to the back of the Home Charger for charging overnight.

When fully charged, the Portable Charger provides one full charge to your sound processor. To fully charge a depleted sound processor takes approximately 3 to 4 hours.

¹ Optional accessory

Portable Charger equipment includes:

- Portable Charger unit
- USB cable for charging
- Charger cable for connecting to your sound processor.

Warning

- Remove the Portable Charger immediately if it becomes hot and contact your clinician.
- Do not place the Portable Charger under clothing of children or anyone who is unable to communicate to you if the charger becomes hot.

Caution: To optimise battery performance:

- Fully charge the Portable Charger immediately, even if you do not plan to use it.
- Every 6 months, fully charge the Portable Charger.

Charge the Portable Charger

1. **Plug** the small USB-C cable connector into the Portable Charger.

2. **Plug** the larger USB-A cable connector into a power source. Options include:
 - Home Charger DC 5V OUT socket
 - Home Charger power adaptor
 - Laptop USB port*

* USB ports must be high power USB 2.0 or higher. If you use a USB hub to connect more than one USB device to a port, we recommend you use a powered hub.

3. After approximately 3 hours, **press** the button to check the charge level. When fully charged, **4 lights** will display.
4. When fully charged, **unplug** the Portable Charger.

It is now ready to take with you in case you need to recharge your sound processor.

Make sure you also take the Portable Charger cable so you can connect the charger to the sound processor.

Indicator lights**What it means**

Green flashes

Power is connected for charging.
Number of lights = amount of charge.
Light display stops 30 seconds after charger
is connected to power.

Orange flashes

Problem with power or temperature.

Steady orange

Charger is faulty. Contact your clinician to
arrange a replacement charger.


Check the charge level

To check the charge level, **press the button** on the front of the Portable Charger.

Green lights display to indicate the available charge.

Indicator lights	What it means
	1 light – ¼ full
	2 lights – ½ full
	3 lights – ¾ full
	4 lights – fully charged
Green flashes	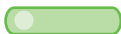

Charge your sound processor

To use the Portable Charger to charge your sound processor:

1. **Connect** the cable to the **Portable Charger**.
The grey plug corresponds to the grey socket.

2. If the socket cover is in place, **remove** the socket cover from the back of your sound processor.

3. **Connect** the other end of the cable to your **sound processor**.
As it begins to charge, the sound processor light **pulses green**.

4. **Wear** the charger by attaching to clothing (for example, pocket or collar) or on a lanyard around your neck.


5. Use the hole on the clip to **connect** a lanyard.

6. **Place** the sound processor on your head.

7. When fully charged, the sound processor light displays **steady green**.
8. **Disconnect** the cable by lifting the connector upwards.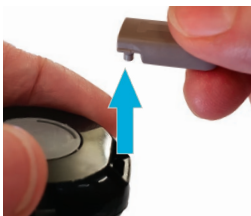

9. **Store** the charger and cable carefully to avoid damage.
10. If removing the cable, **hold** the connector on either side and **pull**.

Note: Two cable lengths are available for the Portable Charger: 30 cm and 50 cm.

For the Cochlear Headband

The Cochlear Headband is an optional accessory so you can charge your sound processor while you are wearing it in the headband, using the Cochlear Portable Charger.

For information on how to use the Headband, refer to the relevant sound processor user guide.

Regular care

**Caution:**

- Do not use cleaning agents or alcohol to clean the chargers.
- Turn your sound processor off before cleaning or performing maintenance.

Every day

Check your chargers for dirt and moisture. Wipe the chargers with a soft dry cloth.

Store your chargers, power adapter and cables in a safe place to avoid damage.

Every six months

If your Portable Charger has not been charged, charge the Portable Charger to optimise battery life.

Troubleshooting

An **orange light**, indicating an error, will flash on the chargers if they are:

- not receiving sufficient power, or
- used in temperatures outside the operating range.

What should you do?

- Check that the cables are connected properly.
- Try using the charger in a different location. The temperature may have been outside the charger's operating range.
- If using a USB port, try another port. The type of USB port may not be compatible.
- If using a USB hub:
 - Check you are using a powered hub. The type of hub may not be compatible.
 - Remove some devices. The hub may have too many devices connected.

Cautions

To charge your sound processor:

- use Cochlear equipment only.
- do not use non-Cochlear equipment.

Warnings

- Sound processors and related accessories contain small parts that alone or in combination may pose a hazard of inhalation, choking or ingestion. Swallowing or inhaling small parts can cause severe or fatal injuries. Always supervise children under 3 years of age and others who may be at risk of inhalation, choking or ingestion of small parts when they use the sound processor and related accessories. When not in use, keep small parts, and combinations of small parts, out of reach from children. If small parts are swallowed or inhaled, seek immediate medical attention.
- Removable parts of the system (for example, SoftWear pad, Safety Line) can be lost or may be a choking or strangulation hazard. Keep out of reach of children.
- Unsupervised use of cables (for example, Portable Charger cable) may present a risk of strangulation.
- Do not place the device or accessories inside any part of your body (for example, nose, mouth).
- Carers must routinely check devices that are worn on the body for signs of overheating (for example, Portable Charger). Remove the device immediately if it becomes hot and contact your clinician.
- Do not place the chargers or parts in any household devices (for example, microwave oven, dishwasher, dryer).
- Do not expose the chargers or parts to heat (for example, never leave them in sunlight, behind a window or in a vehicle).
- Your chargers and other parts of the system contain complex electronic parts. These parts are durable but must be treated with care.
- No modification of this equipment is allowed. Warranty will be void if modified.

Physical configuration

Home Charger

The Home Charger is the primary charging unit and includes capability to dry your sound processor. The Home Charger comprises:

- wireless charging coil, internal fan and custom electronics
- power input and error light indicators
- built-in temperature sensor
- power adaptor
- USB cable.

Materials

External housing	• Polyamide / acrylonitrile butadiene styrene (ABS) • Polycarbonate • Silicone
------------------	--

Dimensions	Length	Width	Depth
Home charger unit (typical values)	42 mm	85 mm	83 mm

Weight	Weight
Charging unit (typical value)	111 g

Operating characteristics	Value / range
Operating frequency (inductive charging link)	100 Hz to 160 kHz
Input operating voltage	4.75 V to 5.25 V DC
Output voltage	4.75 V to 5.25 V DC
Output current	200 mA
Power consumption	3.5 W

Compatible products

The Home Charger can be used to charge:

- Cochlear Kanso 3 Nexa Sound Processor
 - Cochlear Kanso 3 Sound Processor
 - Cochlear Kanso 2 Sound Processor
 - Cochlear Surgical Processor.
-

Portable Charger

The Portable Charger has an internal battery to provide charge to the sound processor battery using the charger cable. The Portable Charger comprises:

- Custom electronics with internal battery
- Battery charge level check button and light indicators
- Built-in temperature sensor
- Charger cable to connect to sound processor
- USB cable.

Materials

External housing	Polyamide
------------------	-----------

Dimensions

Dimensions	Length	Width	Depth
Portable charger unit (typical values)	59.5 mm	28 mm	13.7 mm (excludes clip)

Weight

Weight	Weight
Portable charger unit and cable (typical value)	22 g

Compatible products

The Portable Charger can be used to charge:

- Kanso 3 Nexa Sound Processor
- Kanso 3 Sound Processor
- Kanso 2 Sound Processor

Operating characteristics	Value / range
Input operating voltage	4.75 V to 5.25 V
Output voltage	4.90 V to 5.25 V
Output current	120 mA
Power consumption	1 W
Button functions	Check battery level
Battery type	Lithium ion
Battery capacity	1.18 Wh
Ingress Protection Rating when connected to the Kanso 3 or Kanso 2 Sound Processor	IP54 Ingress Protection: • Protected against access of solid foreign objects greater than or equal to 1.0 mm diameter. • Protected against excessive dust penetration that could interfere with satisfactory operation. • Protected against failure from splashing water.

Environmental conditions

Chargers

Conditions	Minimum	Maximum
Storage and transport temperature	-10° C (14° F)	+55° C (131° F)
Storage and transport humidity and Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa
Operating temperature	+5° C (41° F)	+40° C (104° F)

Note: Refer to your sound processor user guide for maximum operating temperature of your sound processor when used during charging with the portable charger.

Other information

Electromagnetic compatibility (EMC)

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 in.) to any part of the Cochlear Home Charger or Cochlear Portable Charger, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Interference may occur in the vicinity of equipment marked with the following symbol:

Environmental protection

Help protect the environment by not disposing of your chargers with your unsorted household waste. Please recycle your chargers according to your local regulations.

Equipment classification

Your chargers are internally powered equipment Type B applied parts as described in the international standard IEC 60601 1:2005/A1:2012+A2:2020, Medical Electrical Equipment – Part 1: General Requirements for Basic Safety and Essential Performance.

Labelling symbols

The following symbols may appear on your charger components, or packaging:

Consult instructions for use

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Compatible implants

Authorised representative in the European Community/European Union

Authorised representative in Switzerland

Unique Device Identifier

Medical Device

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

Model number

CE registration mark

CE registration mark with notified body number

Contains or presence of natural rubber latex.
May cause allergy.

Recyclable material

Dispose of electrical components in accordance with
your local regulations

Dispose of batteries separately from normal waste
and according to your local regulations

Rx Only

By prescription.

Caution: US law restricts this device to sale by,
or on the order of, a physician

IP21

Ingress Protection Rating

- Protected against solid objects over 12.5 mm, for example, fingers or similar objects
- Protected against dripping water (vertically falling drops).

Privacy and the collection of personal information

During the process of receiving a Cochlear device, personal information about the user, recipient or their parent, guardian, carer and hearing health professional will be collected for use by Cochlear and others involved in care with regard to the device.

For more information please read Cochlear's Privacy Policy on www.cochlear.com or request a copy from Cochlear at the address nearest you.

Trademark legal notice

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, Nexa, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, SoundBand, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield and Xidium are either trademarks or registered trademarks of the Cochlear group of companies.

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Aviso legal de marcas

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, Nexa, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, SoundBand, True Wireless, el logo elíptico, Vistafix, Whisper, WindShield y Xidium son marcas comerciales o marcas comerciales registradas del grupo de empresas de Cochlear.

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ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, Nexa, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, SoundBand, True Wireless, le logo en forme d'ellipse, Vistafix, Whisper, WindShield et Xidium sont des marques de commerce ou des marques déposées des sociétés du groupe Cochlear.

Notes

This image shows a full page of white paper with horizontal dotted lines, typical of primary school writing paper. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

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CA Cochlear Canada Inc 2500-120 Adelaide
Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 (800) 523 5798

GB Cochlear Europe Ltd 6 Dashwood Lang
Road, Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400

BE Cochlear Benelux NV
Schaliënhoevedreef 20 i, B-2800 Mechelen,
Belgium
Tel: +32 15 79 55 11

FR Cochlear France S.A.S. 135 Route de
Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International)
or 0805 200 016 (National)

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HK Cochlear (HK) Limited Room 1404-1406,
14/F, Leighton Centre, 77 Leighton Road,
Causeway Bay, Hong Kong
Tel: +852 2530 5773

KR Cochlear Korea Ltd 2nd Floor, Yongsan
Centreville Asterium, 25, Hangan-daero 30 gil,
Yongsan-gu, Seoul, Korea (04386)
Tel: +82 2 533 4450

**CN Cochlear Medical Device (Beijing) Co.,
Ltd** Unit 2608-2617, 26th Floor, No.9 Building,
No.91 Jianguo Road, Chaoyang District, Beijing
100022, P.R. China
Tel: +86 10 5909 7800

**IN Cochlear Medical Device Company
India Pvt. Ltd.** Ground Floor, Platina Building,
Plot No C-59, G-Block, Bandra Kurla Complex,
Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111

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AE Cochlear Middle East FZ-LLC Dubai
Healthcare City, Al Razi Building 64, Block
A, Ground Floor, Offices IR1 and IR2, Dubai,
United Arab Emirates
Tel: +971 4 818 4400

PA Cochlear Latinoamérica S.A. International
Business Park, Building 3835, Office 403,
Panama Pacifico, Panama
Tel: +507 830 6220

NZ Cochlear NZ Limited Level 4, Takapuna
Towers, 19-21 Como St, Takapuna,
Auckland 0622, New Zealand
Tel: + 64 9 914 1983

© Cochlear Limited 2023

P1704320 D1704300-V4

Multilingual version of D2077585-V1 2023-11

Cochlear® 