



Cochlear[™] Osia[®]

**Important information for
Osia System recipients**

About this document

This document applies to the Cochlear™ Osia® OSI300 Implant and the Osia 2(I) Sound Processor as part of the Cochlear Osia System. It is intended for Osia System recipients and their carers.

Read this document carefully

The information in this document contains important safety warnings and cautions relating to the device and its use. These warnings and cautions relate to:

- Implant recipient safety
- Device function
- Environmental conditions, and
- Medical treatments.

Before starting medical treatment, discuss the medical treatment warnings in this document with the recipient's physician.

Additional details on device use and care are included in the user guides and product information supplied with the device. Please read these documents carefully – they may contain additional warnings and cautions.

Symbols used in this document

 **Note**
Important information or advice.

 **Caution (no harm)**
Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

 **Warning (harmful)**
Potential safety hazards and serious adverse reactions.
Could cause harm to person.

Contents

- About this document 2
 - Symbols used in this document 2
- For implant recipients 4
 -  Warnings..... 4
 -  Cautions 6
- For parents and carers of Osia System recipients..... 9
- For discussion with physicians of Osia System recipients 11
 - Medical treatments generating induced currents, heat and vibration..... 11
 - MRI safety information 13
 - What is an MRI? 14
- Electromagnetic compatibility (EMC)..... 15
 - Recommended separation distances..... 17
- Privacy and the collection of personal information 19

For implant recipients

Cochlear devices are designed to be safe and effective. However, it is also essential that you take care when using them.

This section contains warnings and precautions for safe and effective use of your device. You should also refer to your user guide for specific warnings and cautions related to the use of external components.

Warnings

This section includes general warnings to ensure your personal safety.

Small parts hazard

Small parts and accessories could be hazardous if swallowed or cause choking if ingested or inhaled.

Overheating

Remove your processor immediately if it becomes unusually warm or hot, and seek advice from your clinician.

Uncomfortable sound levels

If the sound becomes uncomfortable, remove your sound processor immediately and contact your clinician.

If you have two sound processors (one for each ear), always wear the sound processor programmed for your left ear on the left and the processor programmed for your right ear on the right.

Using the wrong sound processor could result in loud or distorted sounds that, in some instances, could cause extreme discomfort.

Head trauma

Impact or injury to your head in the area of the Osia implant could damage the implant and result in its failure.

Impact to external components (e.g. sound processor) while being worn could result in damage to the device or injury.

Pressure

Do not apply continued pressure to the sound processor when in contact with the skin e.g. sleeping/lying on sound processor or using tight fitting headwear, as this may result in pressure sores.

If the magnet in the sound processor is too strong or is in contact with the skin, pressure sores may develop at the implant site. If this happens or if you experience any discomfort in this area, contact your clinician.

Use of batteries and battery ingestion

When using disposable batteries, only use battery types recommended by your clinician or Cochlear. Other types may not have sufficient energy to allow your sound processor to operate for a long time.

Batteries could be hazardous if used incorrectly. For information on safe battery use refer to your external component user guides.

Batteries can be harmful if swallowed. Ensure that batteries are kept out of reach of young children. If swallowed, seek prompt medical attention at the nearest emergency center.

Adverse environments

Operation of your Osia System could be adversely affected in environments of high magnetic field strength and high electric field strengths, e.g. close to high power commercial radio transmitters.

Seek medical advice before entering any environment that may adversely affect the operation of your implant, including areas protected by a warning notice preventing entry by patients fitted with a pacemaker.

Cautions

This section includes general cautions to ensure safe and effective use of your Osia System, and to avoid causing damage to system components.

General use

Use your Osia System only with cleared devices and accessories listed in the user guide. If you experience a significant change in performance, turn off your sound processor and contact your clinician.

Do not operate or store your sound processor at temperatures other than those recommended in the user instructions supplied with your sound processor.

Your processor and other parts of the system contain complex electronic parts. These parts are durable but must be treated with care.

No modification of external equipment is allowed. If your sound processor is modified or opened by anyone other than Cochlear's qualified service personnel, the warranty is invalid.

Sound processor

Each processor is programmed specifically for each implant.

Never wear another person's sound processor or lend yours to another person.

Your processor's sound quality could be intermittently distorted when you are within approximately 1.6 km (~1 mile) of a radio or television transmission tower. Additional sources of interference include, but are not limited to:

- Security systems
- Industrial machinery and power systems
- Mobile communications equipment, including cellular telephones and certain kinds of hand-held, two-way radios

To reduce or eliminate the interference, move away from the source. If your sound processor stops working, turn the power switch off and then back on.

Theft and metal detection systems

You could experience a distorted sound sensation when passing through or near one of these devices. Devices such as airport metal detectors and commercial theft detection systems produce strong electromagnetic fields.

The materials used in your Osia System may activate metal detection systems. For this reason, always carry the Implant Card with you at all times. Turn off your sound processor if near or passing through a theft and metal detection system.

Mobile telephones

Some types of digital mobile telephones, e.g. Global System for Mobile communications (GSM) as used in some countries, may interfere with the operation of your external equipment. You could perceive a distorted sound sensation when close, 1-4 m (~3-12 ft), to a digital mobile telephone in use.

Air travel

Some airlines request that passengers turn off portable electrical devices, such as laptop computers and electronic games, during take-off and landing or whenever the seat belt sign is illuminated. Your processor is considered to be a medical portable electronic device.

Notify airline personnel that you are using a hearing implant system that allows you to hear. They can then alert you to safety measures, which may include the need to switch your processor off. When boarding a flight, wireless functionality must be deactivated because radio signals must not be transmitted during flights.

TO ACTIVATE FLIGHT MODE:

1. Turn off your sound processor by opening the battery door.
2. Press the button and close the battery door at the same time.
3. If enabled, audio and visual signals will confirm that flight mode is activated.

TO DEACTIVATE FLIGHT MODE:

Turn the sound processor off and then on again (by opening and closing the battery door).

Scuba diving

The maximum diving depths with an Osia implant is 40 m (~131 ft).

Seek medical advice before participating in a dive to ensure you do not have any conditions that might make diving contraindicated, e.g. middle ear infection.

When wearing a mask, avoid pressure over the implant site.

Do not wear the sound processor under water unless it is inside a waterproof container, such as the Osia 2 Aqua+ accessory and it has batteries approved to be used in a waterproof case.

Electromagnetic interference with medical devices

The Osia sound processor meets defined international Electromagnetic Compatibility (EMC) and emission standards. However, because the sound processor radiates electromagnetic energy, it is possible that it could interfere with other medical devices such as cardiac pacemakers and implantable defibrillators when used nearby. It is recommended that you keep your sound processor at least 15 cm (~6 in.) away from devices which could be subject to electromagnetic interference. For added assurance, also consult the recommendations provided by the device manufacturer.

Electrostatic discharge (ESD)

Before engaging in activities that create extreme electrostatic discharge, such as playing on plastic slides, remove your sound processor. In rare cases, a discharge of static electricity can damage the electrical components of the Cochlear Osia system or cause your sound processor to shut down.

If static electricity is present (for example when removing or putting on clothes over your head, or getting out of a vehicle), before the Cochlear Osia system contacts any object or person you should touch something conductive, such as a metal door handle.

If you stop hearing and suspect your sound processor received a discharge of static electricity, turn it off and then on again. If the problem continues, contact your clinician or a Cochlear representative.

Magnetic fields

Magnetic fields that are very close to an Osia implant can affect the operation of the implant. These magnetic fields can be created by magnets that are stronger than the Osia Sound Processor coil magnets.

If you stop hearing and suspect that you have a strong magnetic field close to the location of the Osia implant, move away from the source of the magnetic field.

Hearing will then return. If the problem continues, contact your clinician or a Cochlear representative.

For parents and carers of Osia System recipients

Warnings

This section includes general warnings for parents and carers of Osia System recipients to ensure recipient safety. Please also read the user guide, which contains specific warnings on external component use, and the information earlier in this document.

Small parts hazard

Keep small parts and accessories out of reach of children.

Small parts and accessories could be hazardous if swallowed or cause choking if ingested or inhaled.

Strangulation

Parents and carers are advised that unsupervised use of long cables (such as the safety line) may present a risk of strangulation.

Overheating

Parents and carers should touch the processor to check for heat if the recipient is showing signs of discomfort.

Remove the processor immediately if it becomes unusually warm or hot, and seek advice from your clinician.

Uncomfortable sound levels

Carers should routinely check that the system is working at a comfortable volume level. If the sound becomes uncomfortable, remove the external equipment immediately and contact your clinician.

If the recipient has two processors (one for each ear), ensure they always wear the processor programmed for their left ear on the left and the processor programmed for their right ear on the right. Using the wrong sound processor could result in loud or distorted sounds that, in some instances, could cause extreme discomfort.

Head trauma

Young children who are developing motor skills are at greater risk of receiving an impact to the head from a hard object, e.g. table or chair.

Impact or injury to the head in the area of the Osia implant could damage the implant and result in its failure.

Impact to external components (e.g. sound processor) while being worn could result in damage to the device or injury.

Pressure

Carers should routinely check the skin over the implant site. Continued pressure to the sound processor when in contact with the skin may result in pressure sores, e.g. sleeping/lying on sound processor or using tight fitting headwear or headband.

If the magnet in the sound processor is too strong, pressure sores may develop at the implant site. If this happens, contact your clinician.

For discussion with physicians of Osia System recipients

Having a Cochlear Osia System means extra care must be taken when receiving some medical treatments. Before starting medical treatment, the information in this section should be discussed with the recipient's physician.

The sound processor must be removed before starting any of the medical treatments listed in this section.

Medical treatments generating induced currents, heat and vibration

Some medical treatments generate induced currents that may cause tissue damage or permanent damage to the implant. Before initiating any of the following treatments turn off the sound processor.

Warnings

Warnings for specific treatments are provided below.

Diagnostic and therapeutic ultrasound

The implant has been designed to withstand diagnostic ultrasound. It has been tested to the following parameters:

- centre frequency: $3.5 \text{ MHz} \pm 0.175 \text{ MHz}$
- duty cycle: $20 \% \pm 1 \%$
- intensity (ISPTA) $\geq A \times 1 \text{ 500 mW/cm}^2$

The sound processor must be turned off during diagnostic ultrasound.

Do not expose the sound processor to therapeutic ultrasound.

Diagnostic and therapeutic ionising radiation

The implant can be exposed to diagnostic ionising radiation (x-rays, CT scans).

The sound processor must be turned off during exposure to ionising radiation.

Do not expose the implant to a total dose greater than 70Gy of therapeutic ionizing radiation.

Electrosurgical equipment

Bipolar electrosurgical instruments can be used provided the electrodes are kept more than 1 cm from the device.

The sound processor must be turned off while bipolar electrosurgical instruments are used near the head or neck.

Monopolar electrosurgical instruments must not be used on the head or neck after the implant has been implanted.

Therapeutic or medical diathermy

Do not use therapeutic or medical diathermy on the head or neck.

Therapeutic or medical diathermy may be used below the neck.

Neurostimulation

Do not use neurostimulation over the implant.

Defibrillator

Do not place defibrillator electrodes in direct contact with the device.

Electroconvulsive therapy

Do not use electroconvulsive therapy on an implant patient under any circumstances.

Electroconvulsive therapy can cause tissue damage or damage to the implant.

MRI safety information



The Cochlear Osia OSI300 Implant is MR Conditional.

MRI examinations can be performed safely on a person with this implanted device only under specific conditions.

MRI examinations performed under different conditions may result in severe patient injury or device malfunction.

Full MRI safety information is available:

- In the Cochlear™ Osia® Magnetic Resonance Imaging (MRI) Guidelines
- By visiting www.cochlear.com/mri
- By calling your nearest Cochlear office – contact numbers are available on the back cover of this guide.

Refer to the MRI Guidelines for a complete list of Warnings and Cautions.



All external components of the Cochlear Osia System (e.g. sound processors, and related accessories) are MR unsafe.

The patient must remove all external components of their Cochlear Osia System before entering a room where an MRI scanner is located.

What is an MRI?

Radiologists and MR technologists are medical specialists experienced in diagnosing disease and injuries using a range of imaging techniques.

One of these imaging techniques is magnetic resonance imaging (MRI).

MRI is a diagnostic tool to obtain images of organs and tissues using a very powerful magnetic field measured in tesla (T). MRI scans can range in strength from 0.2 T to 7 T, with 1.5 T being the most common.

Safety concerns for medical device implants and MRI

Due to the powerful magnetic and radio-frequency fields, medical device implants with metallic or ferromagnetic components such as pacemakers, defibrillators, catheters, pumps and Osia implants can create problems for MRI scans. The risks include the potential for device repositioning, localised heating, unusual sounds or sensations, pain or injury and distortion of the MR image.

Cochlear Osia OSI300 Implant and MRI compatibility

To ensure MRI compatibility, Cochlear Osia OSI300 Implant contains a removable magnet cassette. The magnet cassette is easy to remove and replace if needed. A non-magnetic cassette is available to prevent fibrous tissue growing in the implant magnet recess.

The Cochlear Osia OSI300 Implant is cleared for MRI scans under specific conditions at 1.5 T and at 3 T with the magnet in place or removed.

Electromagnetic compatibility (EMC)

Guidance and manufacturer’s declaration

The Cochlear Osia OSI300 Implant is intended for use in the electromagnetic environments specified in this document.

Electromagnetic emissions

Emission test	Compliance	Guidance
RF emissions CISPR 11 / EN 55011, Group 1	Class A (programming mode) Class B (stand-alone mode)	The device is suitable for use in all establishments, including domestic establishments and those directly connected to public low-voltage power supply network that supplies buildings used for domestic purposes.
RTCA DO160G: 2010, Section 21, Category M	RTCA DO160G: 2010, Section 21, Category M	
Voltage fluctuations/ flicker emissions IEC 61000-3-3 Harmonic emissions IEC 61000-3-2	Not applicable	

Table 1: Electromagnetic emissions.

Electromagnetic immunity

Immunity	Compliance	Guidance
ESD, IEC 61000-4-2	+/- 8 KV contact, +/-15 KV air discharge	Remove the sound processor prior to undertaking activities that may create extreme electrostatic discharge. In rare cases, a discharge of static electricity can damage the electrical components of the hearing implant system or corrupt the program in the sound processor. If static electricity is present (for example when removing or putting on clothes over your head, or getting out of a vehicle), before the hearing implant system contacts any object or person, the static electricity should be discharged by touching something conductive such as a metal door handle.
Power frequencies: ISO 14708-7 / EN 45502-2-3: 50Hz and 60Hz	Test level 1200 A/m peak	Power frequency magnetic fields be at levels characteristic of a typical location in a typical commercial or hospital environment
Radiated RF: IEC 61000-4-3, 80MHz to 2.7GHz	10 V/m	See Warnings and Recommended separation distances
Radiated RF: EN 45502-2-3 Section 27.4	As specified in EN 45502-2-3:2010 Section 27.4	None
Magnetic field. EN45502-2-3: Section 27.3	As specified in EN 45502-2-3:2010 Section 27.3	None

Table 2: Electromagnetic immunity.

Recommended separation distances

 Warning

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 in.) to any part of your sound processor, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The Osia System is intended for use in an electromagnetic environment where the radiated RF disturbances are controlled. Keep a distance of at least 30 cm from common devices to reduce the likelihood of electromagnetic interference. Refer to Table 3 for some of the devices that can result in electromagnetic interference.

Frequency band (MHz)	Devices
380-390	Emergency Services - two way radio
430-470	Walkie Talkie
704-787	Mobile phones on LTE network
800-960	Special two way radio used by emergency services or in mines.
	Mobile phones - GSM / LTE /CDMA networks.
	Push to talk services.
1700-1990	Mobiles phones, cordless phones
2400-2570	Bluetooth devices e.g. Bluetooth speakers, Bluetooth headphones etc.
	Shopping centre barcode readers , RFID readers. 2.4 G Wireless modems/routers
5100-5800	5G Wireless modems/routers

Table 3: List of example devices that can result in electromagnetic interference.

 Warning

Use of your sound processor adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, your sound processor and the other electronic equipment should be observed to verify that they are operating normally.

Interference may occur in the vicinity of equipment marked with the following symbol:



 Note

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people

Privacy and the collection of personal information

During the process of receiving a Cochlear device, personal information about the user/recipient or their parent, guardian, carer and hearing health professional will be collected for use by Cochlear and others involved in care with regard to the device.

For more information please read Cochlear's Privacy Policy on www.cochlear.com or request a copy from Cochlear at the address nearest you.

AU Cochlear Ltd (ABN 96 002 618 073)
1 University Avenue, Macquarie University, NSW 2109,
Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

EC REP DE Cochlear Deutschland GmbH & Co. KG
Mailänder Straße 4 a, 30539 Hannover, Germany
Tel: +49 511 542 770 Fax: +49 511 542 7770

CH REP CH Cochlear AG
Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 8204 Fax: +41 61 205 8205

US Cochlear Americas
10350 Park Meadows Drive, Lone Tree, CO 80124, USA
Tel: +1 303 790 9010

CA Cochlear Canada Inc
2500-120 Adelaide Street West, Toronto, ON M5H 1T1,
Canada
Tel: +1 (800) 483 3123 Fax: +1 416 972 5083

GB UK Responsible Person: Cochlear Europe Ltd
6 Dashwood Lang Road, Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

BE Cochlear Benelux NV
Schaliënhoeverdreef 20 i, B-2800 Mechelen, Belgium
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

FR Cochlear France S.A.S.
135 Route de Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International) or 0805 200 016
(National)
Fax: +33 5 34 63 85 80

IT Cochlear Italia S.r.l.
Via Trattati Comunitari Europei 1957-2007 n.17,
40127 Bologna (BO), Italy
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

SE Cochlear Nordic AB
Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

TR Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.
Küçükbakkalköy Mah, Defne Sok, Büyükhanlı Plaza No:3 Kat:3
Daire: 9-10-11-12, 34750, Ataşehir, İstanbul, Türkiye
Tel: +90 216 538 5900 Fax: +90 216 538 5919

HK Cochlear (HK) Limited
Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road,
Causeway Bay, Hong Kong
Tel: +852 2530 5773 Fax: +852 2530 5183

KR Cochlear Korea Ltd
2nd Floor, Yongsan Centreville Asterium, 25,
Hangang-daero 30 gil, Yongsan-gu, Seoul, Korea (04386)
Tel: +82 2 533 4450 Fax: +82 2 533 8408

CN Cochlear Medical Device (Beijing) Co., Ltd
Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo
Road,
Chaoyang District, Beijing 100022, P.R. China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

IN Cochlear Medical Device Company India Pvt. Ltd.
Ground Floor, Platina Building, Plot No C-59, G-Block,
Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

JP 株式会社日本コクレア(Nihon Cochlear Co Ltd)
〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル
Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

AE Cochlear Middle East FZ-LLC
Dubai Healthcare City, Al Razi Building 64, Block A, Ground
Floor, Offices IR1 and IR2, Dubai, United Arab Emirates
Tel: +971 4 818 4400 Fax: +971 4 361 8925

PA Cochlear Latinoamérica S.A.
International Business Park, Building 3835, Office 403,
Panama Pacifico, Panama
Tel: +507 830 6220 Fax: +507 830 6218

NZ Cochlear NZ Limited
Level 4, Takapuna Towers, 19-21 Como St, Takapuna,
Auckland 0622, New Zealand
Tel: + 64 9 914 1983 Fax: 0800 886 036

www.cochlear.com

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always follow the directions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

Cochlear, 科利耳, コクレア, 코클리어, Hear now. And always, Osia, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of the Cochlear group of companies (unless otherwise noted).

© Cochlear Limited 2023. All rights reserved. 2023-01

P1906423 D1906424-V1