Getting Started with Smart Sync

Supplement

EN

ES

FR-CA

Contents

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About this supplement

This supplement describes the **Smart Sync** feature for hearing implant recipients and their carers, and health professionals.

The information in this supplement should be read together with:

- Cochlear™ Nucleus® 8 Sound Processor User Guide, or
- Cochlear™ Nucleus® Kanso® 3 Sound Processor User Guide.

Receiving your new sound processor

Your new sound processor comes with no programs.

Smart Sync will automatically copy your existing programs from your Cochlear™ Nexa™ (CI1000 Series) Implant to a new compatible sound processor.

Compatible Cochlear Nucleus implants are¹:

- Nexa (CI1000) Series Implants: CI1012, CI1022, CI1024, CI1032.

Compatible sound processors are:

- Cochlear Nucleus 8 Processing Unit with Cochlear Nucleus 8 Nexa Slimline™ Coil
- Cochlear Nucleus Kanso 3 Nexa Sound Processor.

¹ Not all products are approved or available in all countries. Please contact your local Cochlear representative for product information.

Smart Sync

Caution for bilateral recipients with two Cochlear Nuclea implants:

Ensure to place the sound processor on the intended side of the head, otherwise the programs will be programmed for the opposite ear and will need to be reset by your clinician.

1. **Charge** the sound processor battery when you receive your new sound processor¹.
2. **Assemble** and **turn on** your new sound processor¹.
3. **Place** your new sound processor on the intended side of the head. If your sound processor turns off after two minutes, turn it back on and re-attempt.
4. **Wait** approximately 20 seconds for the programs to transfer to your new sound processor.
5. When the Smart Sync process is **complete**, you will hear short high beeps. Your new sound processor is ready to use.

Note: You will need to re-pair your new sound processor to your mobile device, Nucleus Smart App, and any wireless devices. Please refer to the respective *Cochlear Pairing Guide* for your mobile device.

¹ For further instructions, please refer to:

- Cochlear Nucleus 8 Sound Processor User Guide, or
- Cochlear Nucleus Kanso 3 Sound Processor User Guide.

Lights and beeps

Smart Sync indication lights you may see, or alert beeps you may hear

Beep	What it means
 Short high beeps	Sound processor has completed set up, and is ready to use. The number of beeps indicates the number of the current program (1,2,3,...).
Light	What it means
 Flash of orange every second	Sound processor is turned on. Sound processor flashes while it is off your head.
 Steady green	Sound processor one-time-only copy of programs in progress. The program copy takes approximately 20 seconds.
 Green flashes	Sound processor is ready to use. The number of flashes indicates the number of the current program (1, 2, 3 ...).
 Steady orange	Fault. Contact your clinician. Stays on until the issue is resolved.

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CA Cochlear Canada Inc 2500-120 Adelaide
Street West, Toronto, ON M5H 1T1, Canada
Tel: +1 (800) 523 5798

GB Cochlear Europe Ltd 6 Dashwood Lang
Road, Bourne Business Park, Adlestone,
Surrey KT15 2HJ, United Kingdom
Tel: +44 1932 26 3400

BE Cochlear Benelux NV
Schaliënhoeverdreef 20 i, B-2800 Mechelen,
Belgium
Tel: +32 15 79 55 11

FR Cochlear France S.A.S. 135 Route de
Saint-Simon, 31035 Toulouse, France
Tel: +33 5 34 63 85 85 (International)
or 0805 200 016 (National)

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HK Cochlear (HK) Limited Room 1404-1406,
14/F, Leighton Centre, 77 Leighton Road,
Causeway Bay, Hong Kong
Tel: +852 2530 5773

KR Cochlear Korea Ltd 2nd Floor, Yongsan
Centreville Asterium, 25, Hangang-daero 30 gil,
Yongsan-gu, Seoul, Korea (04386)
Tel: +82 2 533 4450

CN Cochlear Medical Device (Beijing) Co.,
Ltd Unit 2608-2617, 26th Floor, No.9 Building,
No.91 Jianguo Road, Chaoyang District, Beijing
100022, P.R. China
Tel: +86 10 5909 7800

IN Cochlear Medical Device Company
India Pvt. Ltd. Ground Floor, Platina Building,
Plot No C-59, G-Block, Bandra Kurla Complex,
Bandra (E), Mumbai – 400 051, India
Tel: +91 22 6112 1111

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AE Cochlear Middle East FZ-LLC Dubai
Healthcare City, Al Razi Building 64, Block
A, Ground Floor, Offices IR1 and IR2, Dubai,
United Arab Emirates
Tel: +971 4 818 4400

PA Cochlear Latinoamérica S.A. International
Business Park, Building 3835, Office 403,
Panama Pacifico, Panama
Tel: +507 830 6220

NZ Cochlear NZ Limited Level 4, Takapuna
Towers, 19-21 Como St, Takapuna,
Auckland 0622, New Zealand
Tel: + 64 9 914 1983

Cochlear, Hear now. And always, Kanso, Nexa, Nucleus, Slimline, and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited.

© 2024 Cochlear Limited

P2292959 D2292967-V1

Multilingual version of D2291090-V1 2024-11

Cochlear® 