

Using the CR310 Remote Control

- 1 Turn the remote on by sliding the gray bar on the side up.



- 3 You can adjust your volume by using the buttons on the right side.

Turn up volume or sensitivity (if available)

Turn down volume or sensitivity



- 2 You can adjust your programs by using the arrow buttons on the left side.

Next program

Previous program



- 4 Turn the remote off by sliding the gray bar on the side down.

Note: The battery in your CR310 Remote could last up to 3 months.*

Note: Use only CR2032 or 5004LC 3V standard lithium coin cell batteries in your remote control.



For help, call Device Support at 800 483 3123 then select option 2.

* Battery life may vary by individual.

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

©Cochlear Limited 2023. All rights reserved. ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントウア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield and Xidium are either trademarks or registered trademarks of the Cochlear group of companies.