


Pairing your Cochlear® Nucleus® Sound Processor to the Nucleus Smart App with a compatible Android™ device


These instructions apply to all supported Nucleus sound processors*.




1.
Open the Nucleus Smart App.

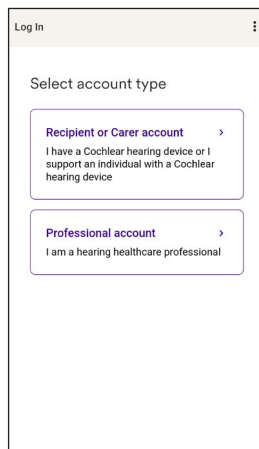


2.
Navigate through the welcome screens. Create or login to your Cochlear Account.

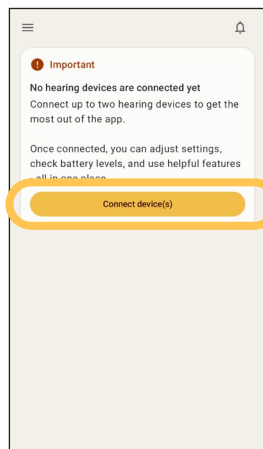


3.
Tap 'Connect device(s)'.

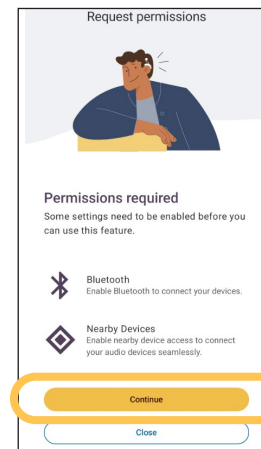

4.
Ensure permissions are allowed to continue.

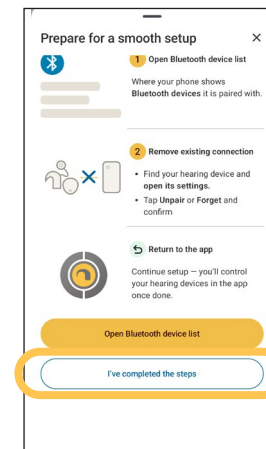


5.
Follow the instructions to remove your hearing device from your phone. Once removed, return to the Nucleus Smart App and tap 'I've completed the steps'.

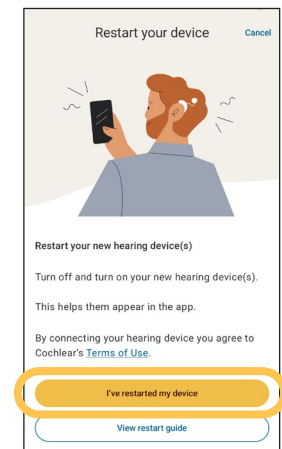


6.
Restart your hearing devices to connect them to the app. 'View restart guide' if required. Then confirm by tapping 'I've restarted my device' to proceed.


* The Cochlear Nucleus Smart App is available on App Store. For information regarding the sound processors, operating systems and devices that are compatible with the Nucleus Smart App, visit www.cochlear.com/compatibility.

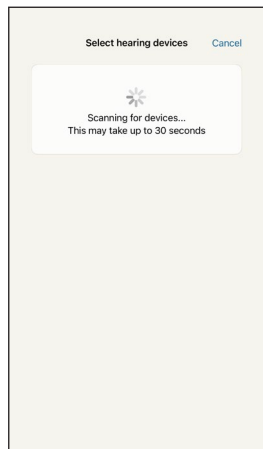

7.
The app will scan for hearing devices.

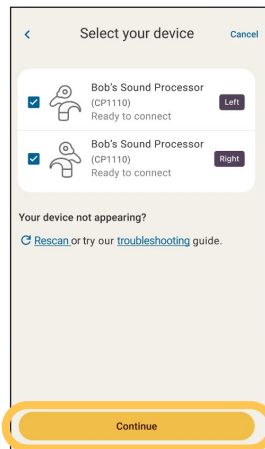


8.
Select your hearing devices and then tap 'Continue'.

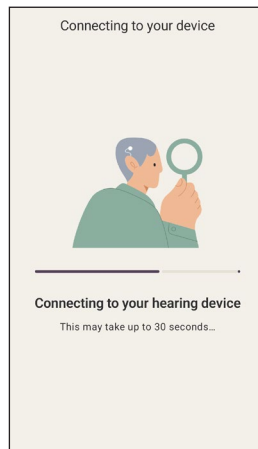

9.
Your device will commence verification.





10.
When instructed, remove your left sound processor from your head.

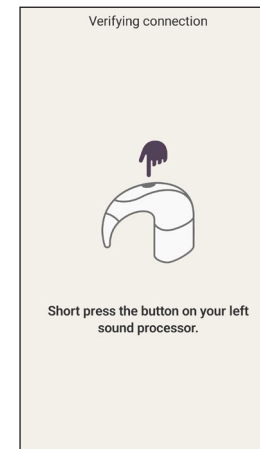


11.
Follow the instructions in the app to verify your sound processor.

OR

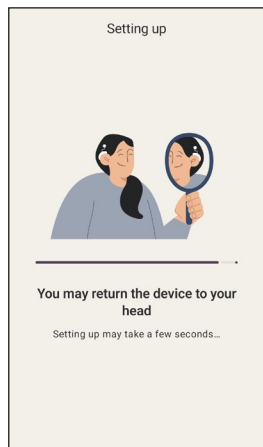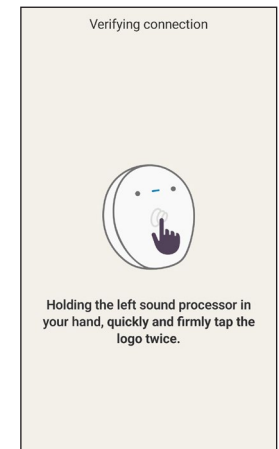


12.
Return your sound processor to your head.

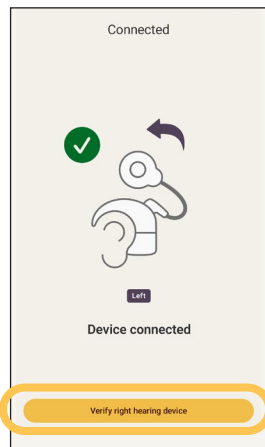

13.
Your sound processor is now connected. Tap 'Verify right hearing device'.




14.
When instructed, remove your right sound processor from your head.

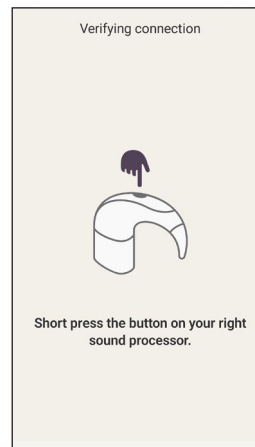



15.
Follow the instructions in the app to verify your sound processor.

OR

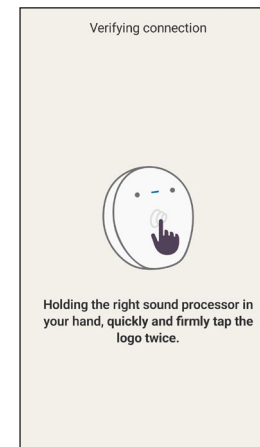


16.
Return your sound processor to your head

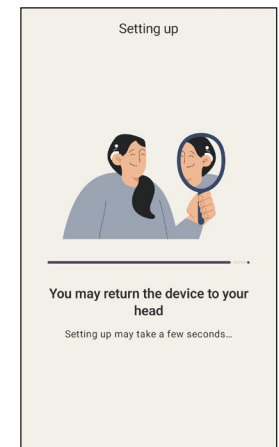

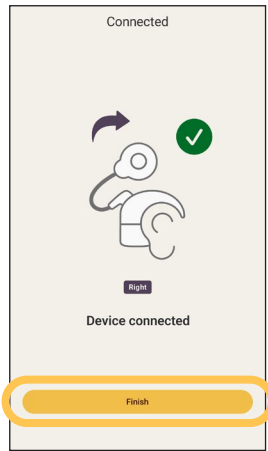

17.

Your sound processor is now connected. Tap 'Finish'.

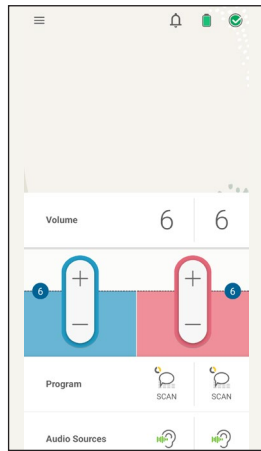

18.

You are now paired and ready to use the app.