


Pairing your Cochlear® Nucleus® Sound Processor to the Nucleus Smart App with a compatible Apple device


These instructions apply to all supported Nucleus sound processors*.




- 1.**
Open the Nucleus Smart App.

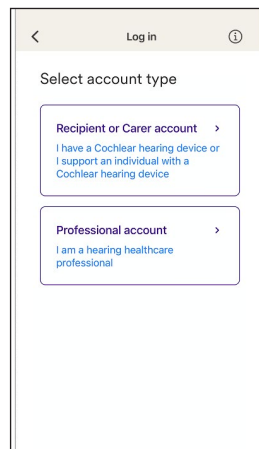


- 2.**
Navigate through the welcome screens. Create or login to your Cochlear Account.

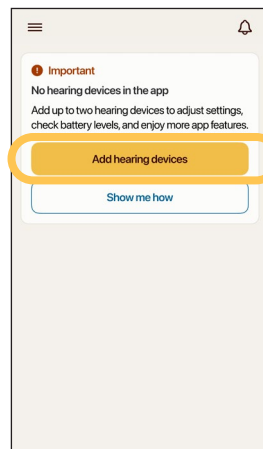


- 3.**
Tap 'Add hearing devices'.

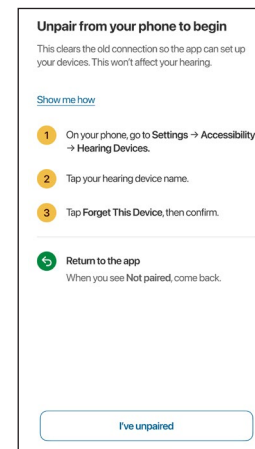


- 4.**
Follow the instructions to remove your hearing device from your phone.

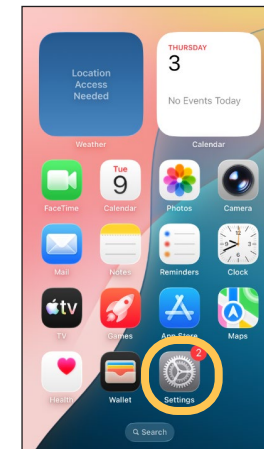


- 5.**
Go to 'Settings'.

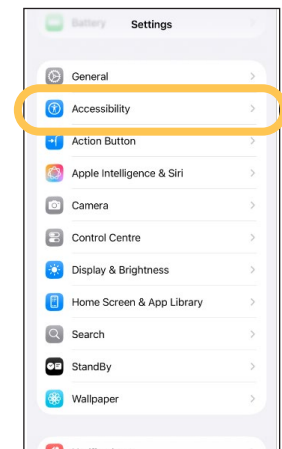


- 6.**
Tap 'Accessibility'.

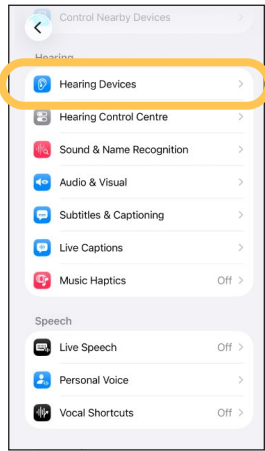


7.
Tap 'Hearing Devices'.

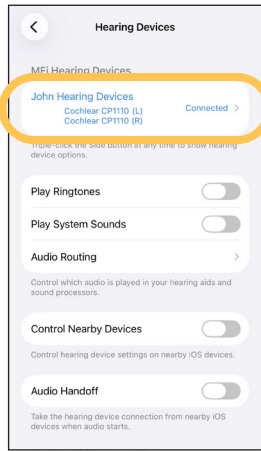



8.
Tap your hearing device.

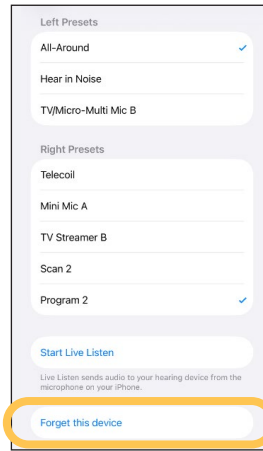


9.
Tap 'Forget this device'.

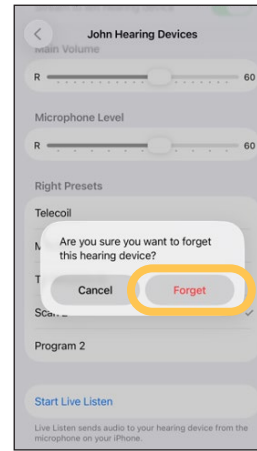


10.
Tap 'Forget'.

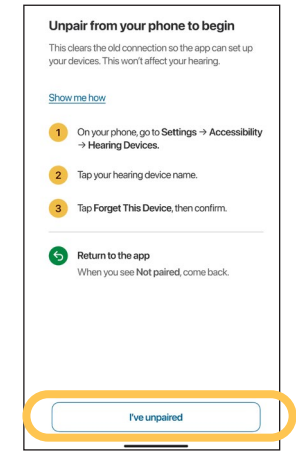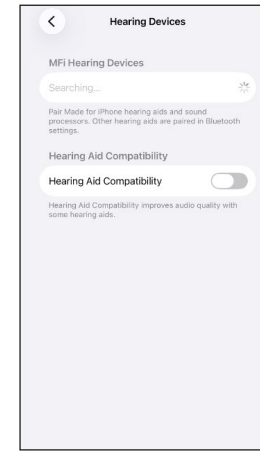


11.
Return to the Nucleus Smart App and tap 'I've unpaired'.

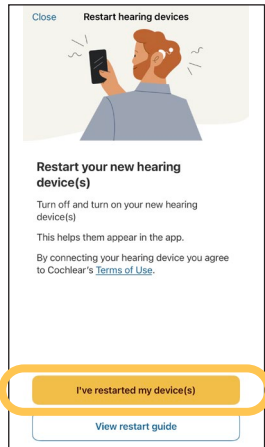


12.
Restart your hearing devices to connect it to the app. View the restart guide if required. Confirm by tapping 'I've restarted my device(s)'.

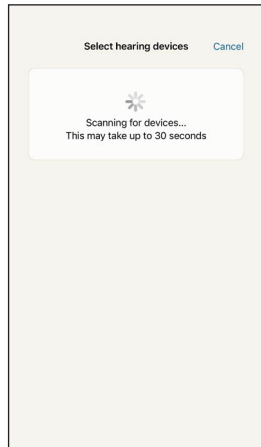


13.
The app will scan for hearing devices.

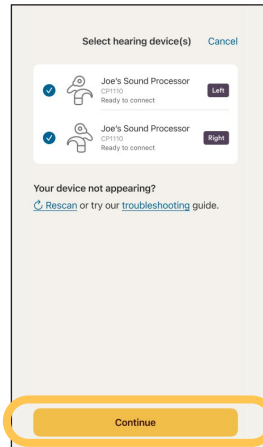


14.
Tap your hearing devices and then tap 'Continue'.

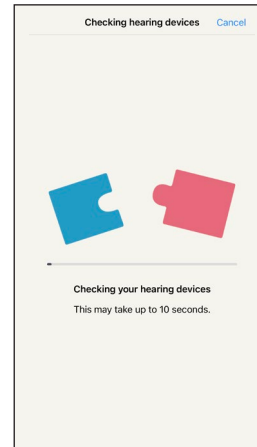


15.
The app will check your hearing devices.

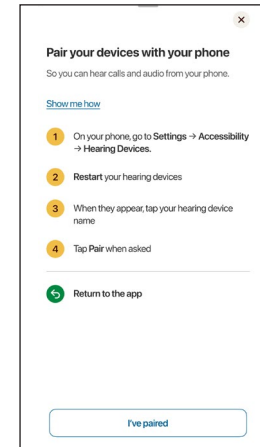


16.
Read the instructions and then open 'Settings' on your device.

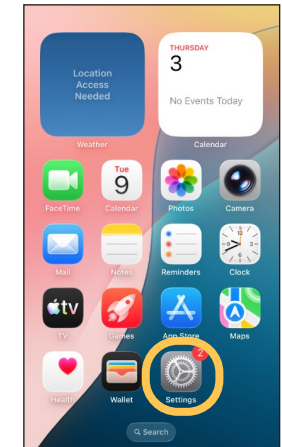


17.
Go to 'Settings'.

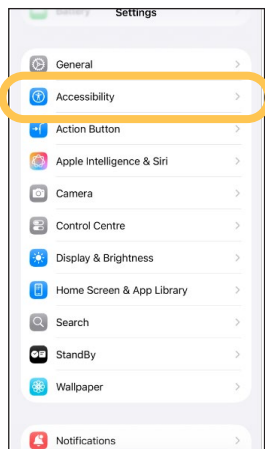


18.
Tap 'Accessibility'.

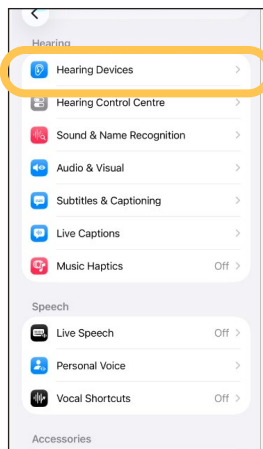


19.
Tap 'Hearing Devices'.

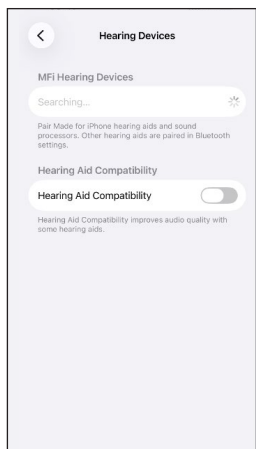



20.
Your phone will search for devices.

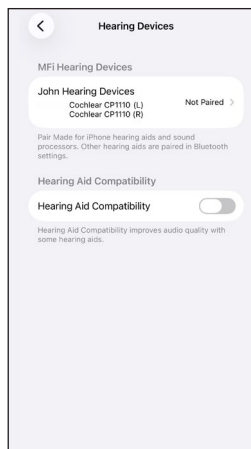


21.
Wait until your hearing devices appear and then select by tapping on them.

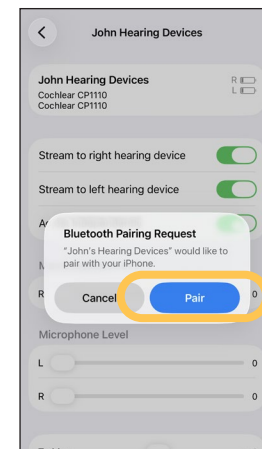

22.
Accept the pairing requests.

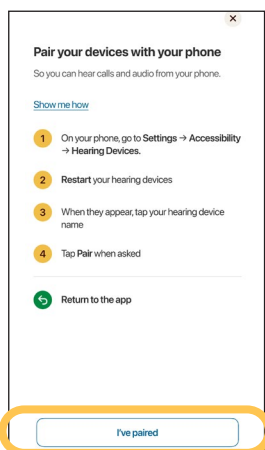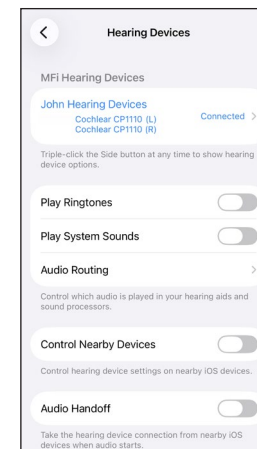


23.
Return to the Nucleus Smart App and tap 'I've paired'.

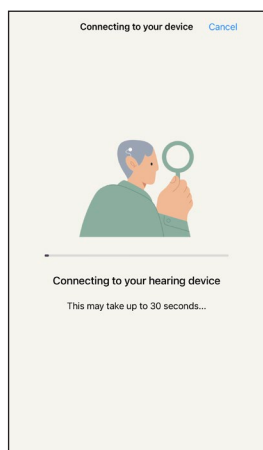


24.
Your devices will commence verification.




25.
When instructed, remove your sound processor from your head.

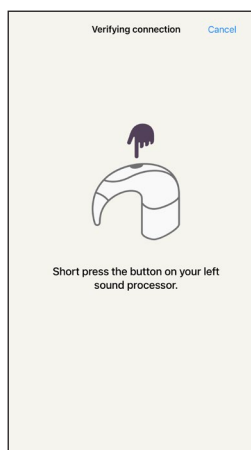


26.
Follow the instructions in the app to verify your sound processor.

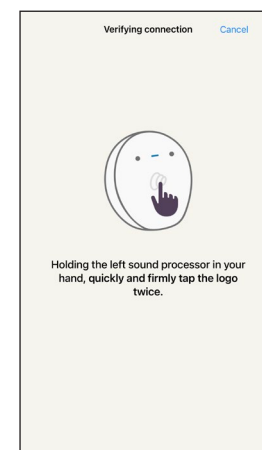


OR

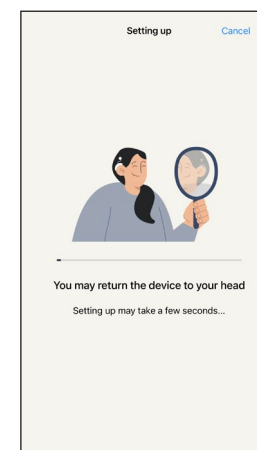


27.
Return your sound processor to your head.

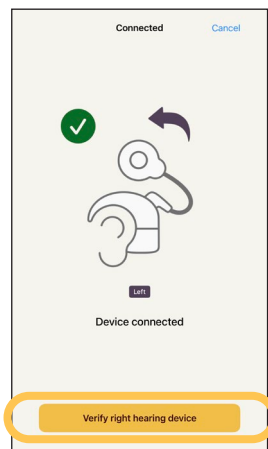

28.
Your sound processor is now connected. Tap 'Verify right hearing device'.


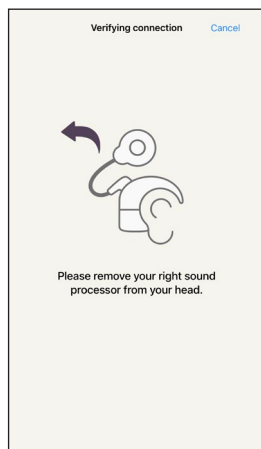


29.
When instructed, remove your sound processor from your head.

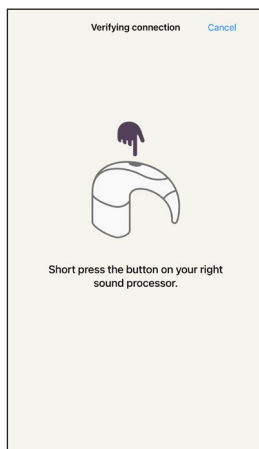


30.
Follow the instructions in the app to verify your sound processor.


OR

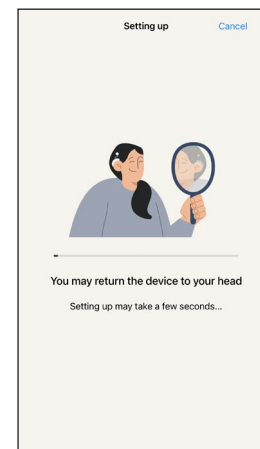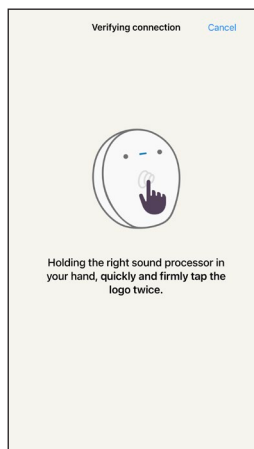

31.
Return your sound processor to your head.

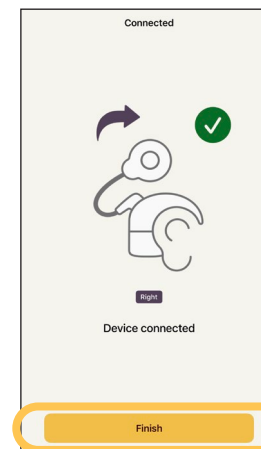

32.
Your sound processor is now connected. Tap 'Finish'.

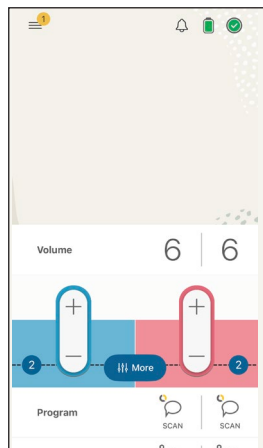

33.
You are now paired and ready to use the app.