# Nucleus<sup>®</sup> 7 SE Aqua+

## User Guide

EN

ID

MS

TH

VI

# Contents

<b>EN</b>	Nucleus® 7 SE Aqua+ User Guide	1
<b>ID</b>	Nucleus® 7 SE Aqua+ Panduan Pengguna	27
<b>MS</b>	Nucleus® 7 SE Aqua+ Panduan Pengguna	53
<b>TH</b>	Nucleus® 7 SE Aqua+ คู่มือผู้ใช้	79
<b>VI</b>	Nucleus® 7 SE Aqua+ Hướng dẫn sử dụng	105

# Contents

Introduction.....	2
Nucleus 7 SE Aqua+ instructions.....	10
Aqua+ coil magnet instructions.....	14
Safety Line instructions.....	15
Mic Lock-Stirrup instructions.....	16
Cautions.....	20
Warnings.....	21
Other information.....	23

# Introduction

## About this guide

This guide is intended for hearing implant recipients and their carers using the Nucleus® 7 SE Aqua+.

### NOTE

- Refer to the relevant sections for cautions and warnings relating to the use of the Nucleus 7 SE Aqua+.
- Please also refer to your **Important Information** document for essential advice that applies to Cochlear implant systems.

## Symbols used in this guide

### NOTE

Important information or advice.

### TIP

Time saving hint.

### CAUTION (no harm)

Special care to be taken to ensure safety and effectiveness. Could cause damage to equipment.

### WARNING (harmful)

Potential safety hazards and serious adverse reactions. Could cause harm to person.

## Nucleus 7 SE Aqua+

The **Nucleus 7 SE Aqua+** is a re-usable sealable cover that, when used with the Nucleus 7 Aqua+ Coil, keeps your Nucleus 7 SE Sound Processor (model number: CP1001) and Nucleus 7 S Sound Processor (model number: CP1002) dry during use in or around water (e.g. when swimming or in the bath). It is worn behind the ear with your sound processor sealed inside.

The Nucleus 7 SE Aqua+ is intended for use by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

**You must use a Cochlear Standard Rechargeable Battery Module for your sound processor to work inside Aqua+.**

Zinc air batteries will not work as they need air and there is no air inside Aqua+ when it's sealed.

The **Nucleus 7 Aqua+ Coil** is used in place of your usual coil while your sound processor is inside your Aqua+. It has a special plug that creates a seal when connected to your sound processor's coil socket through the coil plug hole in Aqua+. As with your usual coil, a coil magnet screws into the Aqua+ coil and holds it against your implant.

**NOTE**

- You cannot use your sound processor in Hybrid™ mode with Aqua+.
- You cannot use your sound processor with any of the Cochlear retention accessories (e.g. Snugfit, Hugfit) with Aqua+.

The **Nucleus Mic Lock-Stirrup** is a retention option to hold Aqua+ more securely behind your ear. The stirrup connects to the base of Aqua+ and the Mic Lock tube (when cut to size) passes under the ear lobe and connects to the Aqua+ earhook.

The **Nucleus Safety Line** is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

While using Aqua+ you can also use the following, providing they are protected from water and kept dry:

- your smartphone (with the Cochlear Nucleus Smart App)
- Cochlear Remote Control (CR310)
- Cochlear wireless accessories.

You may notice a difference in sound quality when using Aqua+.

Testing shows Aqua+ can be:

- used at depths of up to 3 metres for up to 2 hours
- re-used up to 50 times, but this may vary according to how you use it.

There have been no reported undesirable side effects from using Aqua+.

For more information, refer to the *Nucleus 7 SE Sound Processor User Guide* or the *Nucleus 7 S Sound Processor User Guide*.

## Intended purpose

### Nucleus 7 SE Aqua+

The Aqua+ is intended to provide improved water and dust resistance to the sound processor.

### Nucleus 7 Aqua+ Coil

The Aqua+ Coil is intended to be used in combination with other devices as part of a hearing implant system to supply electrical signals and power between a processing unit and a hearing implant.

## Indications

### Nucleus 7 SE Aqua+

The Nucleus 7 SE Aqua+ is indicated for Cochlear Nucleus implant recipients with a compatible Cochlear sound processor and rechargeable battery. Compatible Cochlear sound processors are as follows:

- Nucleus 7 SE Sound Processor
- Nucleus 7 S Sound Processor

Compatible rechargeable battery is the Cochlear Standard Rechargeable Battery Module.

## Nucleus 7 Aqua+ Coil

The Nucleus 7 Aqua+ Coil is indicated for a recipient with a compatible Cochlear Nucleus implant, a compatible Nucleus processing unit and a compatible Cochlear magnet.

Compatible Cochlear Nucleus implants are:

- CI600 Series: CI612, CI622, CI624, CI632
- CI500 Series: CI512, CI522, CI532, ABI541
- CI24RE Series: CI422, CI24REH (Hybrid L24), CI24RE (CA), CI24RE (ST)
- CI24R and CI24M Series: CI24R (CA), CI24R (CS), CI24R (ST), CI24M, ABI24M, CI11+11+2M

Compatible processing units are the Nucleus 7 SE Processing Unit and Nucleus 7 S Processing Unit.

Compatible magnets are Cochlear Magnet strength ½ to 6 and ½(I) to 4(I) and Cochlear Magnet Reverse Polarity strength ½ to 6.

## Nucleus 7 Aqua+ Coil (5(I))

The Nucleus 7 Aqua+ Coil (5(I)) is indicated for a recipient with a compatible Cochlear Nucleus implant, a compatible Nucleus processing unit and compatible Cochlear magnet.

Compatible Cochlear Nucleus implants are:

- CI600 Series: CI612, CI622, CI624, CI632

Compatible processing units are the Nucleus 7 SE Processing Unit and Nucleus 7 S Processing Unit.

Compatible magnets are Cochlear Magnet strength (5(I)).

## Nucleus 7 Aqua+ Coil (N22)

The Nucleus 7 Aqua+ Coil (N22) is indicated for a recipient with a compatible Nucleus 22 cochlear implant (CI22M), a compatible Nucleus processing unit and compatible Cochlear magnet.

Compatible processing units are the Nucleus 7 SE Processing Unit and Nucleus 7 S Processing Unit.

Compatible magnets are Cochlear Magnet strength  $\frac{1}{2}$  to 6 and Cochlear Magnet Reverse Polarity strength  $\frac{1}{2}$  to 6.

## Contraindications

### Nucleus 7 SE Aqua+

The Aqua+ is not indicated for use with processing units other than those listed in *Indications*. The Aqua+ is not indicated for use with the Cochlear Battery Cover (with zinc air batteries).

### Nucleus 7 Aqua+ Coil

The Nucleus 7 Aqua+ Coil is not indicated for use with hearing implants or processing units other than those listed in *Indications*.

The Nucleus 7 Aqua+ Coil is not indicated for use with Cochlear Magnet strength (5(l)).

### Nucleus 7 Aqua+ Coil (5(l))

The Nucleus 7 Aqua+ Coil (5(l)) is not indicated for use with hearing implants or processing units other than those listed in *Indications*.

The Nucleus 7 Aqua+ Coil (5(l)) is not indicated for use with Cochlear Magnet strength  $\frac{1}{2}$  to 6 and  $\frac{1}{2}$ (l) to 4(l) and Cochlear Magnet Reverse Polarity strength  $\frac{1}{2}$  to 6.

## Nucleus 7 Aqua+ Coil (N22)

The Nucleus 7 Aqua+ Coil (N22) is not indicated for use with hearing implants or processing units other than those listed in *Indications*.

## Intended patient population

The sound processor is intended for patients implanted with a compatible Cochlear Nucleus implant. There are no restrictions for the intended patient population of the sound processor and accessories in terms of age, weight, health or other condition.

## Benefits

The sound processor operates in conjunction with a compatible Cochlear Nucleus implant system. Potential benefits of receiving a Cochlear Nucleus implant system include:

- Better understanding of speech in quiet
- Better understanding of speech in noise
- Increased satisfaction based on hearing capabilities.

## Intended users

The Nucleus 7 SE Aqua+ is intended for the following users:

- Recipients of a compatible Cochlear Nucleus implant.
- Carers of recipients, who carry out recipient functions as needed. Carers may include parents of paediatric recipients who are less than 12 years old, and nurses or other carers of otherwise dependant recipients.

## Nucleus 7 SE Aqua+ instructions

**TIP**

Don't forget you must be using a rechargeable battery for your sound processor to work inside Aqua+.

1. Carefully unplug your usual coil from the sound processor. Hold the coil cable grip and firmly pull it straight out of the processor.

**CAUTION**

Do not pull on the coil cable or twist when you pull it out.

2. Pull up on the earhook to remove it.

3. Insert your sound processor into Aqua+.

4. Insert the plug into the base of Aqua+. To make a good seal, push the plug in as far as it will go.

5. Plug the blue Aqua+ coil (with magnet) straight into your sound processor through the hole in Aqua+. **Grip the black ring on the coil connector.** **Do not twist.** The coil plug should feel securely connected.

(See page 14 for magnet instructions)

6. If you're concerned about losing your Aqua+, use one of these retention options:

- A. **Safety Line** clips to your clothing.

(See page 15 for instructions)

- B. **Mic Lock-Stirrup** holds your processor on your ear.

(See page 16 for instructions)

7. Finally, do the **Aqua+ Checklist** to make sure you're ready!


**CAUTION**

Remember, you can't go deeper than 3 metres.

## Aqua+ coil magnet instructions

1. Insert the Cochlear Magnet into the hole in the blue Aqua+ coil.

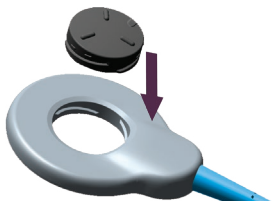2. Turn the magnet clockwise until it stops.

3. Turn the magnet a little more until you feel a click.

**TIP**

The tamper-resistant lock marker aligns with the coil cable when locked.

## Safety Line instructions

### WARNING

Retention lines longer than the Safety Line (short length) are not recommended for use by children under 3 years as they may present a risk of strangulation.

1. Thread the Safety Line loop end through the Aqua+ ring.

2. Create a knot by passing the clip end of the line through the line loop. Pull it tight.


3. Lift the lever to open the clip. Place the clip over your clothing and press the lever down to close the clip.

## Mic Lock-Stirrup instructions

Before using Mic Lock-Stirrup for the first time, cut it to the right length to suit your ear.


1. Snap the stirrup into the dimples at the base of Aqua+.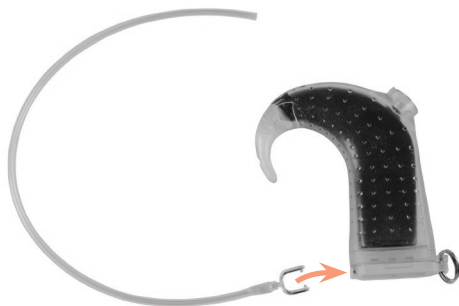

2. Find the right length of tubing to hold Aqua+ in place comfortably behind your ear. Cut the tubing to the required length.

3. Push the cut end of the tubing over the earhook, as far as you can.

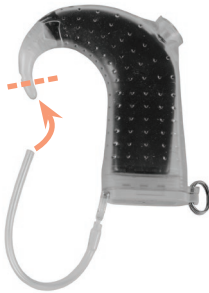4. Wear your Aqua+ with Mic Lock-Stirrup.

**TIP**

Once fitted, you can leave your Mic Lock-Stirrup on your Aqua+.

## Care and storage

1. Rinse Aqua+ and the Aqua+ coil with clean fresh water and dry with a soft cloth.
2. Unplug the blue Aqua+ coil. **Grip the black ring on the coil connector.** Pull the plug straight out.

### CAUTION

Do not pull on the coil cable or twist when you pull it out.

3. Open Aqua+ and remove your sound processor.

4. Re-attach your usual earhook. Click it into place.

5. Re-attach your usual coil. Push your coil cable into the processor until it clicks. **Do not twist.**

6. Place Aqua+ and the blue Aqua+ coil in your dry aid kit overnight.

7. Once they're thoroughly dry, store Aqua+, the blue Aqua+ coil and any accessories in the Nucleus 7 Activity Kit case.

## Cautions

- Always use Aqua+ with the blue Aqua+ coil, never with your usual coil.
- When connecting the coil cable, always push the plug straight in; do not screw or twist the plug.
- Avoid pressing the processor button with your fingernails as this can damage your Aqua+.
- If your Aqua+ appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.

## Warnings

- If your sound processor becomes warmer than usual, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is warmer than usual.
- Small parts (e.g. magnets, Safety Line) may be a choking or strangulation hazard.
- The Aqua+ and sound processor should be protected from excessive heat sources such as prolonged exposure to bright sunlight. Before wearing the Aqua+ and sound processor, check the surface to ensure it is not too hot to wear without discomfort to skin.
- Exposure of the sound processor and Aqua+ to temperatures above the operating range (+5° C – +40° C, +41° F – +104° F) could lead to increased risk of sound processor shutdown or malfunction due to overheating.

- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- Store spare magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

## Other information

### Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage & transport temperature	-10° C (+14° F)	+55° C (+131° F)
Storage & transport humidity	0% RH	90% RH
Operating temperature	+5° C (+41° F)	+40° C (+104° F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

### Environmental protection

Your Aqua+ coil contains electronic components subject to the Directive 2002/96/EC on waste electrical and electronic equipment. Help protect the environment by not disposing of your Aqua+ coil with your unsorted household waste. Please recycle your Aqua+ coil according to your local regulations.

## Warranty

Your Nucleus Global Limited Warranty will not be void in circumstances where Aqua+ is used in water with the Nucleus 7 SE Sound Processor or Nucleus 7 S Sound Processor, in accordance with the Aqua+ instructions.

## Labelling symbols

The following symbols may appear on Aqua+ components and/or packaging.

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Compatible sound processors

Authorised representative in the European Community

Unique Device Identification

Medical Device

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark with notified body number

CE registration mark

**Rx Only**

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

**IP68**

Ingress Protection Rating:

- Protected against failure from dust penetration
- Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

## Certification

The CE mark applies to the Cochlear Nucleus 7 SE Aqua+.

## Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

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**Cochlear Americas** 10350 Park Meadows Drive, Lone Tree, CO 80124, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

**Cochlear Canada Inc** 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada

Tel: +1 416 972 5082 Fax: +1 416 972 5083

**Cochlear AG** EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland

Tel: +41 61 205 8204 Fax: +41 61 205 8205

**Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom

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Tel: +90 216 538 5900 Fax: +90 216 538 5919

**Cochlear (HK) Limited** Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong

Tel: +852 2530 5773 Fax: +852 2530 5183

**Cochlear Korea Ltd** 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea

Tel: +82 2 533 4450 Fax: +82 2 533 8408

**Cochlear Medical Device (Beijing) Co., Ltd**

Unit 2608-2617, 26th Floor, No.9 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China

Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

**Cochlear Medical Device Company India Pvt. Ltd.**

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India

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Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

**Cochlear Middle East FZ-LLC**

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates

Tel: +971 4 818 4400 Fax: +971 4 361 8925

**Cochlear Latinoamérica S.A.**

International Business Park, Building 3835, Office 403, Panama Pacifico, Panama

Tel: +507 830 6220 Fax: +507 830 6218

**Cochlear NZ Limited**

Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand

Tel: +64 9 914 1983 Fax: 0800 886 036

[www.cochlear.com](http://www.cochlear.com)

ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Codacs, Contour, 콘트웬아, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hugfit, Hybrid, Invisible Hearing, Kanso, MET, MicroDrive, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Profile, Slimline, SmartSound, Softip, SPrint, True Wireless, the elliptical logo, and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha SoftWear, BCDrive, DermaLock, EveryWear, Human Design, Piezo Power, SoundArc, Vistafix, and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB.

© Cochlear Limited 2021

P1932180-D1932177 V2

Multilingual version of D1818850 V1 2021-08

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**Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.**