



**Cochlear<sup>™</sup> Osia<sup>®</sup>**

# Magnetic Resonance Imaging (MRI) Guidelines

United States of America

EN

FOR PROFESSIONALS

# Symbols used in this guide



## **Note**

Important information or advice.



## **Caution (no harm)**

Special care to be taken to ensure safety and effectiveness.  
Could cause damage to equipment.



## **Warning (harmful)**

Potential safety hazards and serious adverse reactions.  
Could cause harm to person.

---

# Contents

- Symbols used in this guide . . . . .2**
- How to navigate this guide . . . . .4**
- About this guide . . . . .4**
- Preparation prior to an MRI examination. . . . .5**
  - Cooperation between specialists . . . . .5
- Determine eligibility for MRI. . . . .6**
- Identifying the Cochlear Osia implants. . . . .8**
  - X-ray guidelines . . . . .8
  - X-ray information for identification of Cochlear Osia implants . . . . .8
- Implant magnet and magnet cassette conditions for MRI . . . . .10**
- Image interference and artefacts . . . . .11**
- Risks associated with MRI and Cochlear Osia implants . . . . .18**
  - Considerations for implant magnet removal . . . . .19
  - Considerations for implant magnet in place. . . . .20
- Perform the MRI scan. . . . .21**
  - Patient management and MRI scan steps. . . . .21
  - Bilateral recipients . . . . .23
  - Patient counselling . . . . .23
- MRI machine conditions . . . . .25**
  - Scan conditions for Osia Implants . . . . .25
- Considerations after an MRI examination . . . . .26**
  - With the implant magnet in place. . . . .26
  - With the implant magnet removed. . . . .26
- Trademark legal notice . . . . .27**

# How to navigate this guide

## All professionals:

- Review the content in *About this guide* on page 4.
- Review the content in *Preparation prior to an MRI examination* on page 5.

## Referring physicians:

- To refer a Cochlear™ Osia® implant recipient for an MRI scan, follow the process in *Determine eligibility for MRI* on page 6.

## Radiologists or MR technologists:

- To perform the MRI scan, follow the process in *Perform the MRI scan* on page 21.

# About this guide

This guide applies to the Cochlear Osia implants. It is intended for:

- specialised health care professionals who prepare and perform MRI scans
- physicians who refer a Cochlear Osia implant recipient for an MRI scan
- Cochlear Osia implant recipients and/or their carers.

This guide provides information about the safe application of an MRI scan on Cochlear Osia implant recipients.

MRI scans performed under different conditions than those presented in this guide may result in severe patient injury or device malfunction.

Due to the risks associated with using MRI with an implanted medical device, it is important to read, understand and comply with these instructions to prevent potential harm to the patient and/or device malfunction.

This guide should be read in conjunction with the relevant documents that accompany a Cochlear Osia implant, such as the *Physician's Guide* and *Important information for Osia System recipients*.

For more information, visit [www.cochlear.us/mri](http://www.cochlear.us/mri) or contact your regional Cochlear office. Contact numbers are available on the back cover of these guidelines.

# Preparation prior to an MRI examination

These guidelines are specific to the Cochlear Osia implants and supplement other MRI examination considerations specified by the MRI machine manufacturer or protocols at the MRI facility. Recipients should consult with their implant physician prior to an MRI examination.



Non-clinical testing has demonstrated that the Cochlear Osia implants, in combination with the BI300 Implant, are MR Conditional. A patient with a Cochlear Osia implant can be safely scanned under the conditions described in this document. Failure to follow these conditions may result in injury to the patient and/or device malfunction.

## Cooperation between specialists


Preparing for and conducting an MRI examination for implant recipients requires cooperation between medical professionals.

Medical professional	Role
Cochlear Osia implant device specialist	<ul style="list-style-type: none"><li>• Knows the implant type.</li><li>• Knows where to find the correct MR parameters for the implant type.</li></ul>
Referring physician	<ul style="list-style-type: none"><li>• Knows the location of the MRI scan and diagnostic information required.</li><li>• Decides if the implant magnet or magnet cassette needs to be removed for the MRI examination.</li><li>• Confers with the Osia implant physician regarding the considerations listed in <i>Determine eligibility for MRI</i> on page 6.</li><li>• Confers with the radiologist or MR technologist on artefact size and likely diagnostic value of the scan.</li></ul>
Cochlear Osia implant physician	<ul style="list-style-type: none"><li>• If requested by the referring physician, the implant physician surgically removes the implant magnet or magnet cassette and may temporarily replace it with a non-magnetic plug or a non-magnetic cassette.</li><li>• After the MRI scan, the implant physician implants a new sterile replacement magnet or replacement magnet cassette.</li></ul>
Ear, nose and throat (ENT) physician	<ul style="list-style-type: none"><li>• Prepares the patient for MRI scans by applying the MRI Kit (if required).</li></ul>
Radiologist or MR technologist	<ul style="list-style-type: none"><li>• Sets up the MRI scan using the correct MR parameters and patient positioning.</li><li>• Counsels the implant recipient during the MRI examination.</li></ul>

Table 1: Medical professionals and corresponding roles

# Determine eligibility for MRI

If you are a physician referring a Cochlear Osia implant recipient for an MRI scan, follow the process below.

1. Identify	Instuction details
The recipient's Cochlear Osia implant model.	Refer to <i>Identifying the Cochlear Osia implants</i> on page 8. Bilateral recipients may have two different models. Refer to <i>Bilateral recipients</i> on page 23.
If the recipient has any other implants, active or abandoned.	If another implant is present, verify MRI compatibility before conducting an MRI examination.   <b>Note:</b> Cochlear has evaluated the interaction of implants described in this guide with other nearby implanted devices during MRI scanning, and there is no increased heating risk to the Cochlear Osia implants.
2. Determine	Instruction details
If the implant magnet can remain in place, or if surgical removal is required.	Refer to <i>Implant magnet and magnet cassette conditions for MRI</i> on page 10.
If an MRI Kit is required.	For OSI100 and OSI200, if the implant magnet is retained for an MRI scan at 1.5 T an MRI Kit must be obtained beforehand for use during the MRI scan. Contact the nearest Cochlear office or official distributor to order an MRI Kit. OSI300 does not require an MRI Kit at 1.5 T or 3 T, even with the magnet in place. Refer to <i>Implant magnet and magnet cassette conditions for MRI</i> on page 10.
If the device artefact will obscure the area of interest.	The Cochlear Osia implant will create shadowing on the MR image in the vicinity of the implant, resulting in a loss of diagnostic information.  Confer with the radiologist or MR technologist on artefact size and likely diagnostic value of the MRI scan. Refer to the relevant artefact dimension tables in the section <i>Image interference and artefacts</i> on page 11.  If the required diagnostic information is in the area of the implant, the implant magnet or magnet cassette may need to be surgically removed to minimise the artefact. Refer to <i>Considerations for implant magnet removal</i> on page 19.


3. Confirm understanding	Instruction details
<p>Understand the conditions for an MRI scan and ensure that there is a clear indication for the MRI examination.</p>	<p>Review the implant magnet, MRI kit and artefact information determined in the previous steps and consider whether it is appropriate for the MRI scan to proceed.</p> <p> <b>Note:</b> Also consider:</p> <ul style="list-style-type: none"> <li>• Timing of the implant surgery and MRI exposure, to allow healing of the tissue surrounding the implant.</li> <li>• Age and general health of the implant recipient and time to recover from the implant magnet or magnet cassette surgery or potential trauma.</li> <li>• Existing or potential for tissue scarring in the location of the implant magnet or magnet cassette.</li> </ul>
<p>Understand and inform the patient of the risks associated with MRI.</p>	<p>Refer to <i>Risks associated with MRI and Cochlear Osia implants</i> on page 18.</p> <p>If it was determined that the implant magnet or magnet cassette should be surgically removed, or that an MRI Kit will be used, inform the patient.</p> <p>If required, refer the patient to an appropriate physician to arrange for the implant magnet to be surgically removed before the MRI scan.</p> <p>Additionally, refer to <i>Patient counselling</i> on page 23.</p>

Table 2: Determine eligibility for an MRI scan

# Identifying the Cochlear Osia implants

The implant model can be found on the patient's Cochlear patient implant card. If the patient does not have their patient implant card with them, the implant type and model can be identified without surgical intervention. Refer to *X-ray guidelines* and *X-ray information for identification of Cochlear Osia implants* below.

## X-ray guidelines


Lateral X-ray at 70 kV/ 3 mAs provides sufficient contrast to identify the implant. A modified Stenver's view is not recommended for implant identification as implants may appear oblique. Imaging should include an unobstructed view of antenna coils and implant bodies.

Bilateral recipients may have different implant models on either side of the head. A lateral skull X-ray with a 15 degree cranial tube angle will offset the implants in the image, enabling identifying features to be distinguished.

## X-ray information for identification of Cochlear Osia implants

Cochlear Osia implants are made of metal and are implanted under the skin behind the ear.

Use *Fig.1–Fig.6* to assist with identifying Cochlear Osia implants when using an X-ray.

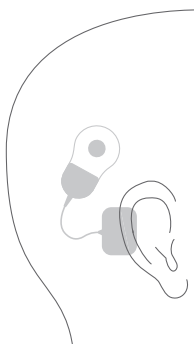


*Fig.1: Approximate location of the OSI100 Implant*




*Fig.2: OSI100 Implant*



OSI300 Implants can be distinguished from OSI200 Implants by the three holes adjacent to the magnet. Refer to **Fig.4** and **Fig.6**.

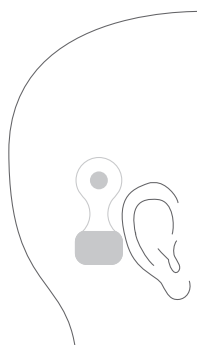


*Fig.3: Approximate location of the OSI200 Implant*

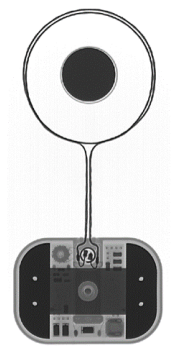

*Fig.4: OSI200 Implant*



*Fig.5: Approximate location of the OSI300 Implant*

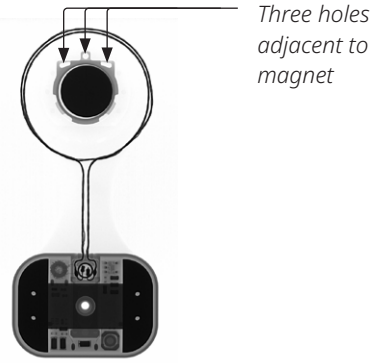

*Fig.6: OSI300 Implant*

# Implant magnet and magnet cassette conditions for MRI

For OSI100 and OSI200 Implants, the implant magnet must be surgically removed for 3 T scans. Bandaging with an MRI Kit is required if the implant magnet is present for 1.5 T scans. If the device artefact will obscure the area of interest, the referring physician may prescribe implant magnet removal. If the implant magnet has been removed, an MRI Kit is not required.

Head bandaging is not required for OSI300 Implants, even with a magnet cassette in place, at 1.5 T or 3 T. Unnecessary use of a head bandage or splint with OSI300 Implants will apply undue pressure and may increase patient discomfort.

Refer to the table below for information on each Osia implant model.

Implant type	MRI field strength (T)	Mandatory to remove magnet or magnet cassette Yes/No	MRI Kit required Yes/No
Osia OSI100 Implant	1.5	No	Yes (if implant magnet is present)
	3	Yes	No
Osia OSI200 Implant	1.5	No	Yes (if implant magnet is present)
	3	Yes	No
Osia OSI300 Implant	1.5	No	No
	3	No	No

Table 3: Implant magnet and magnet cassette conditions for MRI

# Image interference and artefacts

The Cochlear Osia implants will create shadowing on the MR image near the implant, resulting in a loss of diagnostic information.

If inspecting near the implant, consider removing the implant magnet or magnet cassette as MR image quality may be compromised with it in place. The image artefact extends from the centre of the implant.

If the implant magnet or magnet cassette needs to be removed, refer the patient to an appropriate physician before the MRI scan. Refer to *Considerations for implant magnet removal* on page 19.

## Gradient echo sequence



### Note:

The image artefact results are based on worst-case scenarios showing maximum artefact extension from the centre of the implant when scanned using a gradient echo sequence.

The further optimization of scan parameters can be used to minimise the extent of the artefact.

For bilateral recipients, the image artefacts as shown below are mirrored on the opposite side of the head for each implant. There may be some extension of the artefact between the implants.

OSI100 Implant with gradient echo sequence

In non-clinical testing, the image artefact caused by the OSI100 Implant when imaged with a gradient echo pulse sequence scan in the axial plane is as follows:



With implant magnet	Implant magnet removed
	
11.8 cm (4.6 in)	6.2 cm (2.4 in)

Table 4: Maximum image artefact for OSI100 Implants at 1.5 T scans (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.


Implant magnet removed

7.9 cm (3.1 in)

Table 5: Maximum image artefact for OSI100 Implants at 3 T scans (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.

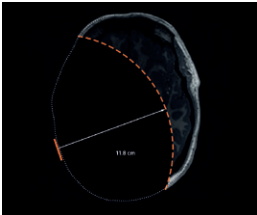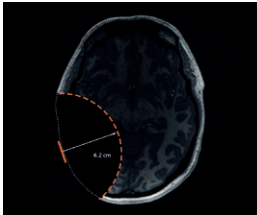

Note:

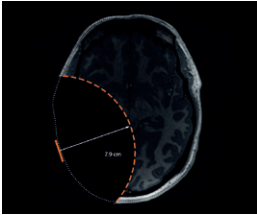The image artefact with implant magnet + magnetic splint may extend further in the axial, coronal or sagittal plane.

OSI200 Implant with gradient echo sequence

In non-clinical testing, the maximum image artefact caused by the OSI200 Implant when imaged with a gradient echo pulse sequence scan in the axial plane is as follows:

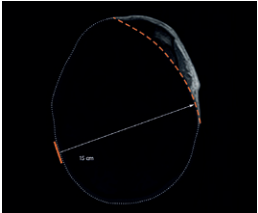
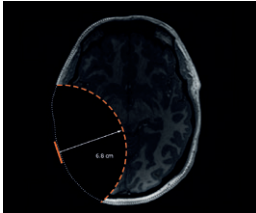
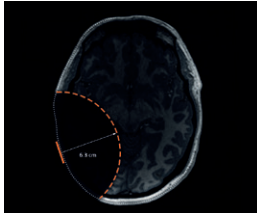
With implant magnet + magnetic splint	With non-magnetic plug	Implant magnet removed
		
15 cm (5.9 in)	6.8 cm (2.7 in)	6.3 cm (2.5 in)

Table 6: Maximum image artefact from centre at 1.5 T (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.

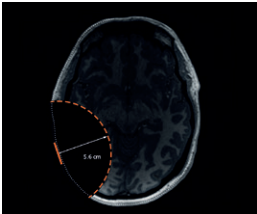
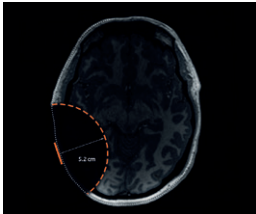
With non-magnetic plug	Implant magnet removed
	
5.6 cm (2.2 in)	5.2 cm (2.0 in)

Table 7: Maximum image artefact from centre at 3 T (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.

**OSI300 Implant with gradient echo sequence**

In non-clinical testing, the maximum image artefact caused by the OSI300 Implant when imaged with a gradient echo pulse sequence scan in the axial plane is as follows:




With implant magnet cassette in place	With non-magnetic cassette	Implant magnet cassette removed
		
12.8 cm (5.0 in)	8.3 cm (3.3 in)	8.2 cm (3.2 in)

Table 8: Maximum image artefact from centre at 1.5 T (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.




With implant magnet cassette in place	With non-magnetic cassette	Implant magnet cassette removed
		
11.6 cm (4.6 in)	5.5 cm (2.2 in)	6.0 cm (2.4 in)

Table 9: Maximum image artefact from centre at 3 T (gradient echo sequence). The image artefact may extend further in the coronal or sagittal plane.

## Metal Artefact Reduction Sequence (MARS)

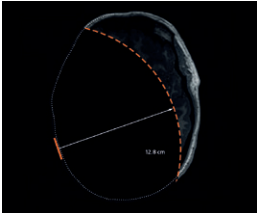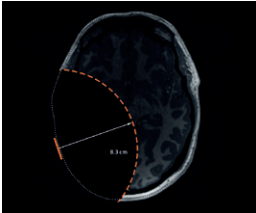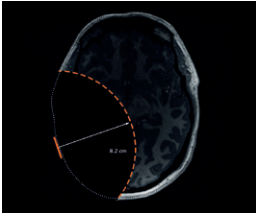

### Note:

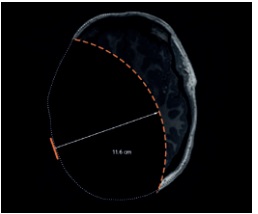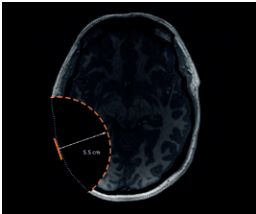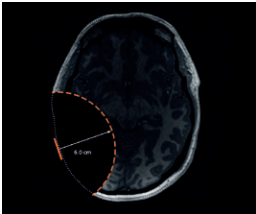The image artefact results show maximum artefact extension from the centre of the implant when scanned using a Metal Artefact Reduction Sequence (MARS).

The further optimization of scan parameters can be used to minimise the extent of the artefact.

For bilateral recipients, the image artefacts as shown on *page 16* and *page 17* are mirrored on the opposite side of the head for each implant. There may be some extension of the artefact between the implants.

### OSI200 Implant with MARS sequence

In non-clinical testing, the maximum image artefact caused by the OSI200 Implant when imaged with a MARS sequence scan in the axial plane is:

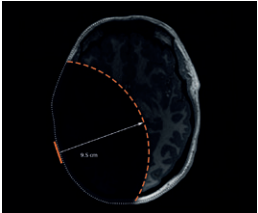
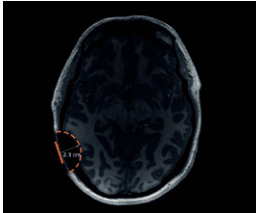
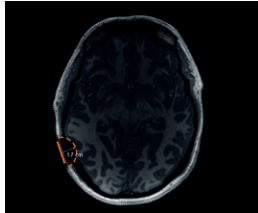
With implant magnet + magnetic splint	With non-magnetic plug	Implant magnet removed
		
9.5 cm (3.7 in)	2.3 cm (0.9 in)	1.7 cm (0.7 in)

Table 10: Maximum image artefact from centre at 1.5 T (MARS sequence)


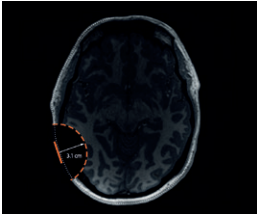
With non-magnetic plug	Implant magnet removed
	
3.1 cm (1.2 in)	2.9 cm (1.1 in)

Table 11: Maximum image artefact from centre at 3 T (MARS sequence)

Parameter	1.5 T MARS Parameters	3 T MARS Parameters
Scanning sequence	Spin echo	Spin echo
Slice selection	Axial	Axial
Slice thickness	3 mm	3 mm
Repetition time	4056 ms	4809 ms
Echo time	80 ms	80 ms
Echo train length	15	12
Bandwidth	217,065 Hz	308,700 Hz
Flip angle	90°	90°

Table 12: MARS scan parameters

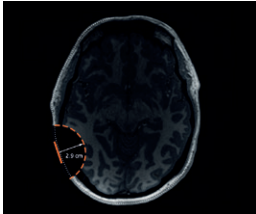

OSI300 Implant with MARS sequence

In non-clinical testing, the maximum image artefact caused by the OSI300 Implant when imaged with a MARS sequence scan in the axial plane is:

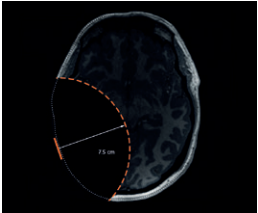
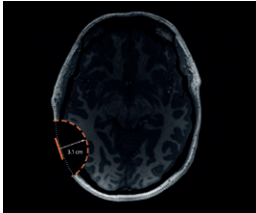
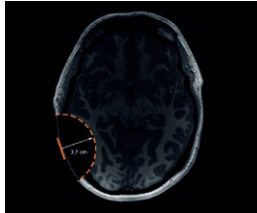
With implant magnet cassette in place	With non-magnetic cassette	Implant magnet cassette removed
		
7.5 cm (3.0 in)	3.1 cm (1.2 in)	3.7 cm (1.5 in)

Table 13: Maximum image artefact from centre at 1.5 T (MARS sequence)

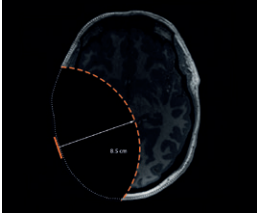
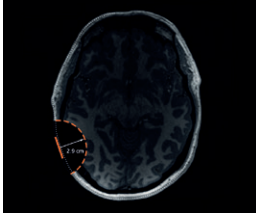
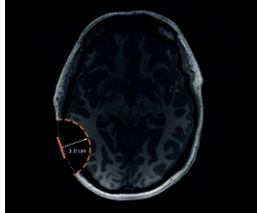
With implant magnet cassette in place	With non-magnetic cassette	Implant magnet cassette removed
		
8.5 cm (3.3 in)	2.9 cm (1.1 in)	3.0 cm (1.2 in)

Table 14: Maximum image artefact from centre at 3 T (MARS sequence)

Parameter	1.5 T MARS Parameters	3 T MARS Parameters
Scanning sequence	Spin echo	Spin echo
Slice selection	Axial	Axial
Slice thickness	5 mm	5 mm
Repetition time	2375 ms	4000 ms
Echo time	17 ms	50 ms
Bandwidth	81,664 Hz	199,936 Hz
Flip angle	90°	90°

Table 15: MARS scan parameters

# Risks associated with MRI and Cochlear Osia implants

Clinicians and recipients should weigh the benefits and risks of completing an MRI scan, and choose one course of action:

1. Keep the magnet in place, and use an MRI Kit if necessary for the implant model and field strength. Refer to *Table 3: Implant magnet and magnet cassette conditions for MRI* on page 10.
2. Remove the implant magnet and replace it via surgical procedures.
3. Do not perform the MRI scan.

The potential risks of performing MRI examinations on patients with Cochlear Osia implants if the MRI safety information is not followed include:

## Device movement

Scanning outside of the parameters contained in these guidelines may lead to the implant magnet or device moving out of position during an MRI examination causing skin or tissue trauma.

## Damage to the device

MRI exposure beyond the values contained in these guidelines may cause damage to the device.

## Weakening of implant magnet (demagnetisation)

- Scanning at static magnetic field strengths at values other than those contained in these guidelines may lead to magnet demagnetisation.
- Incorrect patient positioning prior to the MRI scan or head movement during the MRI scan may result in implant magnet demagnetisation.
- The implant magnet has been designed and verified to state of the art standards. Demagnetisation is highly unlikely when the patient is positioned following the instructions in these guidelines.

## Uncomfortable sensation

MRI exposure beyond the values contained in these guidelines may result in the patient perceiving sound or noise and/or pain.

When an MRI Kit is required, minimise the duration of time that the MRI Kit is applied to reduce possible pain and discomfort. Apply the MRI Kit immediately prior to entering the MRI room.

## Implant heating

Use the recommended MRI RF conditions contained in these guidelines to ensure the implant does not heat beyond safe levels.

## Image artefact

The Cochlear Osia implants will create shadowing on the MR image in the vicinity of the implant, resulting in a loss of diagnostic information.

If inspecting near the implant, removal of the implant magnet or magnet cassette should be considered as MR image quality may be compromised with it in place.

## Considerations for implant magnet removal

If the implant magnet needs to be removed prior to an MRI examination, close coordination is required between the specialists to perform the implant magnet removal, MRI scan, and subsequent implant magnet replacement.

If the MRI examination, implant magnet or magnet cassette removal and replacement will be completed on the same day, the magnet recess can remain empty.

If MRI examinations are needed over a period of time with the implant magnet or magnet cassette removed, it must be replaced in a sterile surgical environment with either a non-magnetic plug or non-magnetic cassette, depending on the implant model.

In the magnet's absence, the non-magnetic plug or non-magnetic cassette prevents fibrous tissue growing into the implant recess. Such growth would make replacement difficult.

For details on implant magnet removal, please refer to the *OSI200 Implant Physician's Guide* or the *OSI300 Implant Physician's Guide* supplied with the system.

### Warning

To minimise the risk of infection or fibrous tissue growing into the implant recess, do not leave the magnet pocket empty for MRI examinations taking place over several days. When removing a magnet cassette or implant magnet, replace with a non-magnetic cassette or non-magnetic plug.

### Caution

Non-magnetic plugs for OSI200 implants are a different size to non-magnetic plugs for OSI100 implants. Ensure the correct non-magnetic plug is used.

### Note:

While the magnet or magnet cassette is removed, the recipient may wear a Cochlear Disk Retainer to hold their sound processor in place. Disk retainers are available from Cochlear.

When there is no further need for MRI examinations, the non-magnetic cassette or non-magnetic plug is removed and replaced by a new sterile replacement magnet or magnet cassette.

The non-magnetic cassette, non-magnetic plugs, magnet cassette and sterile replacement magnets are supplied separately in sterile packs. All are single-use items.

## **Considerations for implant magnet in place**

The information below is being provided to ensure an appropriate treatment decision can be made.

### **Usability of Cochlear Osia MRI Kit**

The Cochlear Osia MRI Kit is intended for use with the OSI200 Implant and the OSI100 Implant with magnet in place at 1.5 T.

Cochlear conducted usability testing on the Cochlear Osia MRI Kit with chinstrap, including associated accompanying documentation. Test results showed that the MRI Kit and accompanying documentation protects users from committing potentially harmful use errors that could lead to patient harm or suboptimal therapy.

#### **Warning**

To minimise possible pain and discomfort, apply the items contained in the MRI Kit immediately prior to entering the MRI room.

Ensure the recipient has left the MRI room, and the MRI procedure is complete, before removing the bandage, splints, and chinstrap.

If a splint becomes loose inside the MRI room, this could lead to MRI equipment damage, and/or could cause injury to the MRI staff or recipient.

#### **Warning**

Do not conduct the MRI scan if the splint does not stay in place.

Misalignment between the splint and implant magnet may result in the dislodgement of the implant magnet and could cause pain or result in explantation.

# Perform the MRI scan







All external components of the Cochlear Osia System (e.g. sound processors and related accessories) are MR Unsafe. The patient must remove all external components of their Cochlear Osia System before entering a room where an MRI scanner is located.

## Patient management and MRI scan steps

A patient with one or two Cochlear Osia implants can be safely scanned in an MR system meeting conditions contained within these guidelines. For MRI scans on a body location away from the implant site, MRI safety information for the recipient's implant model must still be followed.

For medical professionals performing the MRI scan, follow the process below.

1. Confirm prior to scanning	Instruction details
The implant model has been identified.	Refer to <i>Identifying the Cochlear Osia implants</i> on page 8. Bilateral recipients may have two different models. Refer to <i>Bilateral recipients</i> on page 23.
If the recipient has any other implants, active or abandoned.	If another implanted device is present, verify MRI compatibility before conducting an MRI examination.   <b>Note:</b> Cochlear has evaluated the interaction of implants described in this guide with other nearby implanted devices during MRI scanning, and there is no increased heating risk to the Cochlear Osia implants.
Patient positioning requirements are compatible with the MRI scan type, and achievable for the patient.	Refer to <i>Patient positioning</i> on page 23.
If the implant magnet is in place, or has been surgically removed for the MRI scan.	Refer to <i>Implant magnet and magnet cassette conditions for MRI</i> on page 10.   <b>Note:</b> <ul style="list-style-type: none"><li>• Implant magnet removal is required for some implant models and MRI field strengths.</li><li>• Additionally, the referring physician may have prescribed implant magnet removal for the MRI scan, for example, to minimise artefact in the area of interest.</li></ul>

1. Confirm prior to scanning	Instruction details
If required, an MRI Kit has been obtained prior to the MRI scan.	For OSI100 and OSI200, if the implant magnet is retained for an MRI scan at 1.5 T an MRI Kit must be obtained beforehand for use during the MRI scan. Contact the nearest Cochlear office or official distributor to order an MRI Kit. OSI300 does not require an MRI Kit at 1.5 T or 3 T, even with the magnet in place. Refer to <i><b>Implant magnet and magnet cassette conditions for MRI</b></i> on page 10. Refer to <i><b>Usability of Cochlear Osia MRI Kit</b></i> on page 20.
The expected artefact has been considered, and there is still diagnostic value in performing the MRI scan.	Refer to <i><b>Image interference and artefacts</b></i> on page 11.
2. Counsel the patient prior to scanning	Instruction details
If an MRI Kit will be used, explain to the patient how they will be wrapped.	Refer to the Cochlear MRI Kit User Guide provided with the MRI Kit.
Discuss the sensations the recipient may experience during the MRI scan.	Refer to <i><b>Patient comfort</b></i> on page 24.
Explain to the patient how they will be positioned for the MRI scan.	Refer to <i><b>Patient positioning</b></i> on page 23.
3. Perform the MRI scan	Instruction details
<p>Before entering the MRI scan room, remove the sound processor and related accessories, and apply the MRI Kit if it is required.</p> <div data-bbox="93 1171 153 1230">  </div> <p>The sound processor is MR Unsafe.</p>	<p> <b>Note:</b></p> <p>Once the sound processor has been removed, the patient may no longer be able to hear.</p> <p>If an MRI Kit is required, apply it immediately prior to entering the MRI scan room to minimise discomfort. Follow the instructions in the Cochlear MRI Kit User Guide. Additionally, refer to <i><b>Usability of Cochlear Osia MRI Kit</b></i> on page 20.</p>
Position the patient to minimise discomfort.	Refer to <i><b>Patient positioning</b></i> on page 23.
Comply with the MRI machine conditions for the relevant implant models.	Refer to <i><b>MRI machine conditions</b></i> on page 25. Bilateral recipients may have two different implant models. Use the MRI safety information of the patient's implant model with the most restrictive MRI exposure requirements.

4. After the MRI scan	Instruction details
Immediately after the MRI scan, check the patient status.	Refer to <i>Considerations after an MRI examination</i> on page 26.

Table 16: Patient management and MRI scan steps

## Bilateral recipients

Use the MRI safety information of the patient's implant model with the most restrictive MRI exposure requirements.

### Caution

If one of the implants is a CI22M cochlear implant without a removable magnet, MRI is contraindicated.

## Patient counselling

### Patient positioning

For safety and comfort, the patient should be in a supine position (lying flat on back, face upward) prior to entering the MRI bore.

- Align the patient's head with the bore axis of the MRI machine.
- Advise the patient to lie as still as possible and to not move their head during the MRI scan.

Best practise for minimising risk of discomfort:

- Where possible, the patient should enter the scanner feet-first.
- If a detachable MRI table is available, position the patient on the table outside the MRI room. Make sure the patient is comfortable and immobilized in their scanning position before wheeling into the MRI room.
- If scanning head-first, avoid any head movement (pitching or rolling) near the bore entry and within the bore.
  - Place head pillows or supports as far away from the bore entry as practical.
  - Position the patient and immobilize before moving the table into the bore.

### Caution

When scanning with the implant magnet or magnet cassette in place, ensure that the patient does not move more than 15 degrees (15°) from the centreline (Z-axis) of the bore during the MRI scan.

Failure to position the patient correctly prior to the MRI scan may result in increased torque on the implant and cause pain, or may cause demagnetisation of the implant magnet.

## Patient comfort

For patients where an implant magnet or magnet cassette is in place, explain that they might feel the implant magnet moving slightly and might sense resistance to movement as pressure on the skin. To minimise risk of discomfort, follow the instructions in the section *Patient positioning* on page 23.

For devices which require an MRI Kit, the MRI Kit will reduce the likelihood of the implant magnet moving. The sensation will be similar to pressing down firmly on the skin with the thumb.

### **Warning**

To minimise possible pain and discomfort, apply the items contained in the MRI Kit immediately prior to entering the MRI room.

If the patient experiences pain, consult the patient's physician to determine if the implant magnet or magnet cassette should be removed or if a local anaesthetic may be applied to reduce discomfort.

### **Caution**

If administering local anaesthetic, take care not to perforate the implant silicone.

In addition, explain to the patient that they may perceive sounds during the MRI scan.



# MRI machine conditions

The following table details MRI machine conditions for Osia Cochlear implants.

## Scan conditions for Osia Implants


Parameter	Condition
Device name	OSI100 Implant, OSI200 Implant and OSI300 Implant
Static magnetic field strength ( $B_0$ )	1.5 T and 3 T
Type of nuclei	Hydrogen
MRI scanner type	Cylindrical (closed bore or wide bore)
$B_0$ field orientation	Horizontal
Maximum verified spatial field gradient	20 T/m (2000 gauss/cm) for 1.5 T and 3 T
Maximum verified gradient slew rate	200 T/m/s per axis for 1.5 T and 3 T
RF excitation	<p>Circularly Polarized (CP) for 1.5 T and 3 T</p> <p> <b>Warning</b> MRI scans must be performed in quadrature mode or circular polarization (CP) mode for the radio frequency (RF) transmit coil. Using a multi-channel mode may result in localised heating above safe levels.</p>
RF transmitting coil conditions	<ul style="list-style-type: none"><li>Any RF transmitting coil can be used.</li><li>Scan in Normal Operating Mode.</li></ul> <p>There are no additional SAR restrictions for the Osia implants.</p>
RF receiving coil conditions	No restrictions on RF receiving coils.
Operating mode	Normal operating mode
Scan duration	Scan for up to 60 minutes per appointment.
Scan regions	Any landmark is acceptable

Table 17: Scan conditions for the Osia Implants

# Considerations after an MRI examination

## **With the implant magnet in place**

After the patient leaves the MRI room, immediately remove the MRI Kit contents, if used, from the patient's head. Refer to the Cochlear MRI Kit User Guide provided with the MRI Kit for full instructions and warnings.

Ask the patient to place the sound processor on their head and turn it on.

Confirm:

- Placement of the sound processor is correct
- There is no discomfort
- Sound is perceived as normal

If there is discomfort or a change in sound perception, or problems with the placement of the sound processor, ask the patient to seek assistance from their implant clinician as soon as possible.

## **With the implant magnet removed**

Refer to *Considerations for implant magnet removal* on page 19.

# Trademark legal notice

Cochlear, Osia, 科利耳, 玩丸擬莞, 코클리어, Hear now. And always, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of the Cochlear group of companies (unless otherwise noted).



P1906382

**AU Cochlear Ltd** (ABN 96 002 618 073)  
1 University Avenue, Macquarie University, NSW 2109,  
Australia  
Tel: +61 2 9428 6555

**EC REP DE Cochlear Deutschland GmbH & Co. KG**  
Mailänder Straße 4 a, 30539 Hannover, Germany  
Tel: +49 511 542 770

**CH REP CH Cochlear AG**  
Peter Merian-Weg 4, 4052 Basel, Switzerland  
Tel: +41 61 205 8204

**US Cochlear Americas**  
10350 Park Meadows Drive, Lone Tree, CO 80124, USA  
Tel: +1 (800) 523 5798

**CA Cochlear Canada Inc**  
2500-120 Adelaide Street West, Toronto, ON M5H 1T1,  
Canada  
Tel: +1 (800) 523 5798

**GB UK Responsible Person: Cochlear Europe Ltd**  
6 Dashwood Lang Road, Bourne Business Park, Addlestone,  
Surrey KT15 2HJ, United Kingdom  
Tel: +44 1932 26 3400

**BE Cochlear Benelux NV**  
Schaliënhoedreef 20 i, B-2800 Mechelen, Belgium  
Tel: +32 15 79 55 11

**FR Cochlear France S.A.S.**  
135 Route de Saint-Simon, 31035 Toulouse, France  
Tel: +33 5 34 63 85 85 (International) or 0805 200 016  
(National)

**IT Cochlear Italia S.r.l.**  
Via Trattati Comunitari Europei 1957-2007 n.17,  
40127 Bologna (BO), Italy  
Tel: +39 051 601 53 11

**SE Cochlear Nordic AB**  
Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden  
Tel: +46 31 335 14 61

**TR Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.**  
Küçükbakkalköy Mah, Defne Sok, Büyükhanlı Plaza No:3 Kat:3  
Daire: 9-10-11-12, 34750, Ataşehir, İstanbul, Türkiye  
Tel: +90 216 538 5900

**HK Cochlear (HK) Limited**  
Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road,  
Causeway Bay, Hong Kong  
Tel: +852 2530 5773

**KR Cochlear Korea Ltd**  
2nd Floor, Yongsan Centreville Asterium, 25,  
Hangang-daero 30 gil, Yongsan-gu, Seoul, Korea (04386)  
Tel: +82 2 533 4450

**CN Cochlear Medical Device (Beijing) Co., Ltd**  
Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo  
Road,  
Chaoyang District, Beijing 100022, P.R. China  
Tel: +86 10 5909 7800

**IN Cochlear Medical Device Company India Pvt. Ltd.**  
Ground Floor, Platina Building, Plot No C-59, G-Block,  
Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India  
Tel: +91 22 6112 1111

**JP 株式会社日本コクレア(Nihon Cochlear Co Ltd)**  
〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル  
Tel: +81 3 3817 0241

**AE Cochlear Middle East FZ-LLC**  
Dubai Healthcare City, Al Razi Building 64, Block A, Ground  
Floor, Offices IR1 and IR2, Dubai, United Arab Emirates  
Tel: +971 4 818 4400

**PA Cochlear Latinoamérica S.A.**  
International Business Park, Building 3835, Office 403,  
Panama Pacifico, Panama  
Tel: +507 830 6220

**NZ Cochlear NZ Limited**  
Level 4, Takapuna Towers, 19-21 Como St, Takapuna,  
Auckland 0622, New Zealand  
Tel: + 64 9 914 1983

[www.cochlear.com](http://www.cochlear.com)