# Cochlear™ Baha® Attract System

User manual part B

<b>3</b>	<b>8</b>	<b>13</b>	<b>18</b>
EN-GB	ES	FR	PT-BR

## **ZONE 8**

English

Español

Français

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*Hear now. And always*

1

1

2

3

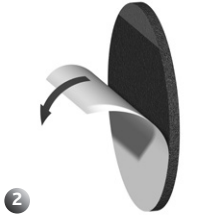
4

2

1

2

3

4

3

3

# Welcome

Congratulations on getting your Cochlear™ Baha® Attract System. This manual provides tips and advice on how to best use and care for your Cochlear Baha Attract System. It is a complement to the sound processor manual part A and does not replace it.

## Introduction

This guide includes instructions on:

- attaching your sound processor to the SP magnet
- care, cleaning and maintenance
- using accessories
- troubleshooting

**Note on figures:** The figures included on the front and back covers correspond to sections of information in this manual which are common to all Baha sound processors. Please reference the appropriate figure when reading. The sound processor images shown are representative only and not to scale. The appearance of the sound processor included in your kit may vary.

## Attaching your sound processor and SP magnet

See figure **1**

The SP magnet attaches to your sound processor to keep it in place over the implant magnet and to transfer sound from your sound processor to the implant.

Use the tilt technique (described below) to connect and disconnect your sound processor from the SP magnet. This will ensure it remains in good working order.

- 1 Place the SP magnet on a flat surface.
- 2 Gently tilt and snap the sound processor into the SP magnet coupling. Ensure you align the button(s) on the sound processor with the arrow mark on the SP magnet.

- 3 With the button(s) and arrow mark facing upwards, place the sound processor over the implant behind your ear. It may be easier to turn the sound processor on before you place it behind you ear.
- 4 Remove the sound processor by grasping the SP magnet with the sound processor still attached. To remove the sound processor from the SP magnet, place one finger under the sound processor and gently tilt until it releases.

### Note:

- During the initial period of use it is recommended to gradually increase the time of use of the SP magnet to allow the skin to adapt to the pressure from the SP magnet. If any irritation is noted the SP magnet should be removed in order to let the skin rest for a while. If discomfort and/or skin irritation persist, contact your hearing care professional.
- To ensure good retention the arrow on the SP magnet should always face upwards and you should try to limit the amount of hair underneath the SP magnet.  
See figure **1** image 3.
- To avoid feedback (whistling), ensure that the sound processor is not touching any other items, such as glasses or a hat.
- It is recommended to wear a safety line attached to the sound processor to reduce the risk of loss or damage.

## Caring for your SP magnet

Keep your SP magnet clean by wiping it with an alcohol-free wipe. Do not use running water to clean your SP magnet.

Clean the SP magnet coupling using a soft brush.

It is recommended to wear the SP magnet with the soft pad attached for maximum wearing comfort.

### Changing soft pads

You should change the SP magnet soft pad regularly. You should replace the soft pad if you observe the following:

- An accumulation of dirt or moisture on the soft pad that cannot be wiped off
- The soft pad appears worn-out or damaged
- You experience a gradual deterioration in wearing comfort (if changing the soft pad does not help please contact your hearing care professional)

Contact your hearing care professional to obtain new soft pads.

See figure **2**

- 1 Remove the soft pad from the SP magnet, by lifting and gripping the unglued portion of the soft material, and then slowly release the soft pad from the SP magnet.

Make sure that the adhesive remains are completely removed from the SP magnet.

- 2 Select a new soft pad and remove the cover sheet to expose the adhesive.
- 3 Attach the soft pad to the SP magnet. Make sure that the soft pad is centralised and covers the entire SP magnet surface.
- 4 Remove the remaining cover sheet from the soft pad before you start wearing the SP magnet.

**Note:** A small section on the edge of the soft pad is not covered by adhesive for easier grip and removal.

## Handling your sound processor

See figure **3**

Once you are familiar with your processor, you can change the program and the volume level while the sound processor is attached to the SP magnet placed on your head. Make sure you support the processor from the bottom with your thumb as you press the buttons with your index finger.

### Storing your processor

Store your sound processor in your storage box when it is not in use. The sound processor can be attached to the SP magnet and the safety line can be attached to the sound processor.

### Warnings and precautions

 **Caution:** If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.

### Other Safety Information

- Remove the sound processor and SP Magnet for any treatment involving ionizing radiation, such as for example X-ray, CT-scan and radiation therapy.
- The implanted magnet may affect diagnostic information/treatment effectiveness in the area around the magnet during a treatment with ionizing radiation (e.g. X-Ray, CTscan, radiation therapy). Consult your treating physician before going through any such treatment.
- The Baha Attract System contains small parts that may be hazardous if swallowed or may cause choking if ingested or inhaled.
- Adult supervision is recommended when the user is a child.
- The Baha Attract System contains magnets that should be kept away from life supporting devices, e.g. cardiac pacemakers and ICDs

(implantable cardioverter defibrillators) and magnetic ventricular shunts, as the magnets may affect the function of these devices. Contact the manufacturer of the specific device to find out more.

- The Baha Attract System (SP Magnet and implanted magnet) contains magnets that may interfere with or damage surrounding electronic devices, for example mobile phones.
  - If the SP magnet is too weak the sound processor may fall off. If it is too strong you may feel discomfort or experience soreness and irritation of the skin. If you are unsure about the strength of your SP magnet, contact your hearing care professional.
  - If you experience persistent irritation or soreness of the skin, or if the SP magnet gives you any discomfort, remove it and contact your hearing care professional.
  - If you wear hats, etc. be sure to avoid pressure on your SP magnet as this may cause irritation to the tissue under this magnet.
  - Do not wear your sound processor and SP magnet while sleeping as this may cause irritation to the skin under the SP magnet.
  - The Baha Attract System (SP Magnet and implanted magnet) contains magnets that may damage magnetic cards (e.g. credit cards, bus cards, etc.) and data storage devices.
  - Remember that your implant contains a magnet, so be aware that it may attract other metallic objects that are held in close proximity (e.g. hairdresser's scissors, etc.).
  - Do not wear any other magnets (e.g. fridge magnets) over your implant than SP magnets from Cochlear supplied to you by your hearing care professional.
  - This product is not suitable for use in flammable and/or explosive environments.
- **If the SP magnet and soft material becomes soaked with water:** Put it in a container with drying capsules such as a Dri-aid kit or similar. Leave it to dry out overnight. Drying kits are available from most hearing care professionals.
  - **For parents or caregivers:** regularly check to see if the user experiences persistent irritation or soreness of the skin, or if the SP magnet causes any discomfort. If so remove it and contact the hearing care professional.

## Troubleshooting

Battery function can be related to a number of problems, such as no sound, intermittent sound and crackling/buzzing. A new battery is often the solution.

If the possible solutions in this chart do not solve your problem, please see your hearing care professional for further assistance.

Problem	Cause	Possible Solution
<b>No sound/weak sound</b>	Weak or dead battery.	Replace battery.
	Volume too low.	Turn up volume.
	Magnet is too weak.	Contact your hearing care professional.
<b>Feedback (whistling)</b>	Hat, glasses or other object in contact with sound processor.	Relocate or remove the object.
	Battery door in wrong position.	Close battery door.
<b>Sound is distorted or intermittent</b>	Too high volume.	Decrease volume.
	Weak battery.	Replace battery.
<b>Sound processor repeatedly falls off</b>	Magnet is too weak.	Contact your hearing care professional.
<b>Persistent skin irritation</b>	Magnet is too strong.	Contact your hearing care professional.
<b>Sound processor does not work</b>	Sound processor not turned on.	Turn on sound processor.
	Battery door is not completely closed.	Gently close the battery door completely.
	Dead battery.	Replace battery.
	Battery inserted wrong way.	Make sure battery is inserted correctly.
	Key lock is activated.	Deactivate key lock.

## Additional Information

### Instructions for use in the USA

**Caution:** Federal law (USA) restricts this device to sale or by the order of a medical practitioner.

#### Warning to hearing aid dispensers

A prospective Baha user must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have an implant placed in the skull bone before a Baha sound processor can be dispensed. The Baha sound processor can be used with a Softband or a SoundArc which does not require an implant. However, a medical evaluation is still warranted.

#### Important notice for prospective users of a Baha sound processor

- A prospective Baha user must always have a thorough medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear, otolaryngologist, otologist, or otorhinolaryngologist) and have an implant placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a Baha candidate. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.

- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

#### A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children, pets, or people of diminished capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency center, or call the National Button Battery Hotline collect at (202) 625 – 3333.

### UKCA mark

The UKCA mark is only applicable to Baha 6 Max.

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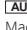
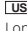
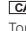
 **[SG] Cochlear Limited (Singapore Branch)**, 238A Thomson Road #25-06, Novena Square Office Tower A, Singapore 307684, Singapore. Phone: +65 65533814

 **[GB] UK Responsible Person: Cochlear Europe Ltd**, 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom. Tel:+44 1932 26 3400

 **[CN] Cochlear Medical Device (Beijing) Co Ltd**, Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China. Tel:+86 10 5909 7800

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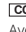
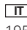
 **[HK] Cochlear (HK) Ltd**, Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong. Tel:+852 2530 5773

 **[BE] Cochlear Benelux NV**, Schaliënhoedreef 20 I, 2800 Mechelen, Belgium. Tel:+32 15 79 55 11

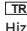
 **[IN] Cochlear Medical Device Company India PVT Ltd**, Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051 India. Tel:+91 22 6112 1111

 **[FR] Cochlear France S.A.S.**, 135 route de Saint Simon, 31035 Toulouse, France. Tel:+33 5 34 63 85 85 (international), Tel: 0805 200 016 (national)

 **[CO] Cochlear Colombia**, Avenida Carrera 9 #115-06 Of. 1201 Edificio Tierra Firme, Bogota D.C., Colombia. Tel:+57 315 339 7169 / +57 315 332 5483

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[www.cochlear.com](http://www.cochlear.com)

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always follow the directions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information. In Australia, Baha bone conduction implant systems are intended for the treatment of moderate to profound hearing loss. For the Cochlear Baha System in Australia: This product is not available for purchase by the general public. For information on funding and reimbursement please contact your healthcare professional.

Cochlear, Baha, 科利耳, コクレア, 코클리어, Hear now. And always, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of the Cochlear group of companies (unless otherwise noted).

© Cochlear Bone Anchored Solutions AB 2023. All rights reserved. 2023-01

94214 613815-V7

  
2797

  
0086

613815-V7

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**[SE] Cochlear Bone Anchored Solutions AB**, Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden. Tel:+46 31 792 44 00
**[SE] Cochlear Nordic AB**, Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden. Tel:+46 31 335 14 61
**[CA] Cochlear Canada Inc**, 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada. Tel:+1 800 483 3123
**[CH] Cochlear AG**, Peter Merian-Weg 4, 4052 Basel, Switzerland. Tel:+41 61 205 8204
**[DE] Cochlear Deutschland GmbH & Co. KG**, Mailänder Straße 4a, 30539 Hannover, Germany. Tel:+49 511 542 770