

## Exercise 3.10 Topic of conversation – unrelated sentences with clue words (set D)

Speaker Copy

---

### Instructions for the Speaker

- 1 Explain to the listener that they should use the clue word to help them repeat the sentence, but that the clue word does not appear in the sentence.
- 2 Say the clue word to the listener with visual cues, or provide the listener with a written list of all the clue words (see next page).
- 3 Say the sentence slowly, but with normal intonation and expression.
- 4 Ask the listener to repeat the sentence.
- 5 Encourage the listener to guess, and give feedback about any correct words.
- 6 If the listener is having difficulty, give one or two key words from the sentence with visual cues, and try again.

| Clue word   | Sentence  |
|-------------|---|
| A Paint     | My favorite color is yellow.                          |
| B Inflation | The price of food is always rising.                   |
| C Exercise  | You must bend your knees when you lift heavy weights. |
| D Church    | All the bells ring on Sunday.                         |
| E Food      | I like to eat lots of fresh fruit.                    |
| F Flies     | When on a picnic, keep all food well covered.         |
| G Wash      | Many people prefer a shower to a bath.                |
| H Game      | Bridge and poker are played with cards.               |
| I Toys      | Boys and girls have bikes and skates.                 |
| J Mailbox   | Receiving letters is more fun than writing them.      |

## Exercise 3.10 Topic of conversation – unrelated sentences with clue words (set D)

Listener Copy

---

### Instructions for the Listener

Here is a list of clue words.

The speaker will say a sentence. Please try to repeat what they have said. The clue word may help.

### Clue word

- A Paint
- B Inflation
- C Exercise
- D Church
- E Food
- F Flies
- G Wash
- H Game
- I Toys
- J Mailbox