

# Cochlear<sup>™</sup> Nucleus<sup>®</sup>

## Implants MRI

### Safety Checklist

This form guides you through the critical aspects of performing an MRI scan safely for patients with a Cochlear<sup>™</sup> Nucleus<sup>®</sup> implant. Before using this form, review the Cochlear Nucleus MRI Guidelines, available on the website:

[www.cochlear.us/mri](http://www.cochlear.us/mri)

Ensure the implant site has healed before an MRI scan is performed. The cochlear implant physician should be consulted if there are any concerns.

Follow these steps prior to inviting the patient into the MRI room and before performing the MRI scan.

- 1** Identify and record the Cochlear Nucleus Implant model using the Cochlear patient identification card, X-ray or surgical/clinical notes in the spaces provided.
- Implant model number for left ear: .....
  - Implant model number for right ear: .....

- 2** Determine if implant magnet removal is required, or if a Cochlear Nucleus MRI Kit is necessary.
- Implant magnet removal may be mandatory at certain field strengths for specific implant types. Refer to **Table 2: MRI safety information and recommended SAR limits**.
  - Implant magnet removal may be necessary to reduce the artefact. Refer to the **MRI Guidelines** for artefact sizes. Metal Artefact Reduction Sequences (MARS) are recommended for optimal results.
  - Implant magnet removal may be avoided at certain field strengths with use of a Cochlear Nucleus MRI Kit.  
The intent of an MRI Kit is to provide pressure over the implant magnet – not the implant body. Refer to **Table 2: MRI safety information and recommended SAR limits**.



#### Note

If there is no implant magnet present, then an MRI Kit is not required.

#### Circle the applicable option:

Implant magnet will be in place for scan / Implant magnet will not be in place for scan

#### Circle the applicable option:

MRI Kit is required for scan / MRI Kit is not required for scan

3 Record the MR parameters and transmit coil to be used for this scan in the spaces provided.

- MRI field strength:

.....

- Maximum spatial gradient the patient will encounter entering the MRI bore:

.....

**Circle the applicable transmit coil to be used:**

Whole body coil / Head coil / Other local cylindrical transmit coil

4 Identify and record the SAR limit and maximum allowable spatial gradient for the implant type and MRI RF transmit coil in the spaces provided. Refer to **Table 1** and **Table 2**.



**Note**

If bilaterally implanted, a patient may have two different cochlear implant types.

Comply with the lowest SAR limit and the lowest allowable spatial gradient of the two devices.

- Lowest SAR limit for recipient's implant types and scan conditions:

.....

- Lowest allowable spatial gradient for recipient's implant types:

.....

5 Counsel the patient on sensations and risks.

- For patients where an implant magnet is in place, explain that they might feel the implant magnet moving slightly and might sense resistance to movement as pressure on the skin.
- For devices which require an MRI Kit, the MRI Kit will reduce the likelihood of the implant magnet moving. The sensation will be similar to pressing down firmly on the skin with the thumb.
- Refer to the **MRI Guidelines** for a complete list of Warnings and Cautions.

6 Remove the sound processor and any accessories before entering the MRI room.  
The sound processor is MR Unsafe.



**Note**

The patient may no longer be able to hear instructions with the sound processor removed.

7 Prior to entering the MRI room, apply the MRI Kit if determined it is required in step 2.

- Ensure you have the contents of the MRI Kit available and within easy reach.
- Full instructions are supplied with MRI Kits, or are listed in the MRI Guidelines.
- Apply the MRI Kit contents to the implant site or sites in accordance with the information in **Table 2**.

8 Comply with patient positioning requirements.

- For safety, the patient should be in a supine position, lying flat on back with face upward, prior to entering the MRI bore.
- Align the patient's head with the bore axis of the MRI machine (max 15° deviation allowed from the z-axis).
- Advise the patient to lie as still as possible and to not move their head during the MRI scan.
- Correctly position the patient prior to the MRI scan to minimise discomfort and reduce the risk of implant magnet demagnetisation.

## Scan conditions and SAR limits

- The MRI safety information in the tables below only applies to 1.5 T and 3 T MRI horizontal scanners (closed bore or wide bore) with a circularly polarised (CP) RF field.
- MRI scans at 3 T must be performed in quadrature mode or CP mode for the radio frequency (RF) transmit coil.
- Using a multichannel mode may result in localised heating above safe levels.
- Maximum active scan time of 60 minutes with the SAR limits in **Table 2** below.
- The SAR limit depends on the selection of MRI RF transmit coil.  
Refer to the Specific conditions for RF transmit coil in **Table 1** below.

RF transmit coil	Specific conditions
Main scanner body coil	<ul style="list-style-type: none"> <li>• Local cylindrical RF receive only coils can be placed anywhere, with respect to the implant.</li> <li>• Comply with the whole body average SAR limit for the relevant implant type, field strength and landmark location. See <b>Table 2</b>.</li> </ul>
Transmit/receive head coil	<ul style="list-style-type: none"> <li>• Comply with the head SAR limit for the relevant implant model and field strength. See <b>Table 2</b>.</li> </ul>
Other local transmit/receive coils (e.g. knee)	<ul style="list-style-type: none"> <li>• Ensure distance between coil and implant is greater than the coil radius.</li> <li>• There are no added SAR restrictions due to the presence of the implant. Limit SAR as you would for a typical patient who does not have an implantable device.</li> </ul>

Table 1: Specific conditions for RF transmit coil

Implant type	MRI field strength (T)	Remove implant magnet Yes/No	MRI Kit required Yes/No	Maximum allowable spatial gradient field (T/m)	Head average SAR limit (W/kg) Using transmit / receive head coil	Whole body average SAR limit (W/kg) Landmark location	
						<40 cm from top of head	≥40cm from top of head
<b>CI600 Series implants</b>							
CI612, CI622, CI624, CI632	1.5	No	No	20	<2	<1	<2
CI612	3	No	No	20	<1	<0.5	<1
CI622, CI624, CI632	3	No	No	20	<1	<0.4	<1
<b>CI500 Series implants</b>							
CI512, CI522, CI532, ABI541	1.5	No	Yes	20	<2	<1	<2
CI512, ABI541	3	Yes	No	20	<1	<0.5	<1
CI522, CI532	3	Yes	No	20	<1	<0.4	<1
<b>CI24RE Series implants</b>							
CI422, CI24REH (Hybrid L24) CI24RE (CA), CI24RE (ST)	1.5	No	Yes	20	<2	<1	<2
	3	Yes	No	20	<1	<0.5	<1
<b>CI24R &amp; CI24M Series implants</b>							
CI24R (CA), CI24R (CS), CI24R (ST), CI24M, ABI24M	1.5	No	Yes	20	<2	<1	<2
	3	Yes	No	20	<1	<0.5	<1
CI 11+11+2M	1.5	No	Yes	20	<1	<0.5	<1
	3	MRI is contraindicated					
<b>CI22M Series implants</b>							
CI22M with removable magnet	1.5	No	Yes	20	<2	<1	<2
	3	MRI is contraindicated					
CI22M without removable magnet	1.5	MRI is contraindicated					
	3	MRI is contraindicated					

Table 2: MRI safety information and recommended SAR limits

# Hear now. And always

As the global leader in implantable hearing solutions, Cochlear is dedicated to helping people with moderate to profound hearing loss experience a life full of hearing. We have provided more than 600,000 implantable devices, helping people of all ages to hear and connect with life's opportunities.

We aim to give people the best lifelong hearing experience and access to innovative future technologies. We collaborate with the industry's best clinical, research and support networks.

That's why more people choose Cochlear than any other hearing implant company.

 **Cochlear Ltd** (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

 **Cochlear Deutschland GmbH & Co. KG** Karl-Wiechert-Allee 76A, 30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770

 **Cochlear AG** Peter Merian-Weg 4, 4052 Basel, Switzerland  
Tel: +41 61 205 8204 Fax: +41 61 205 8205

 **Cochlear Americas** 10350 Park Meadows Drive, Lone Tree, CO 80124, USA  
Tel: +1 303 790 9010

 **Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

[www.cochlear.com](http://www.cochlear.com)

This material is intended for health professionals. If you are a consumer, please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

ACE, Advance Off-Stylet, AOS, Ardium, AutoNRT, Autosensitivity, Baha, Baha SoftWear, BCDrive, Beam, Bring Back the Beat, Button, Carina, Cochlear, 科利耳, コクレア, 코클리어, Cochlear SoftWear, Contour, コントゥア, Contour Advance, Custom Sound, DermaLock, Freedom, Hear now. And always, Hugfit, Human Design, Hybrid, Invisible Hearing, Kanso, LowPro, MET, MP3000, myCochlear, mySmartSound, NRT, Nucleus, Osia, Outcome Focused Fitting, Off-Stylet, Piezo Power, Profile, Slimline, SmartSound, Softip, SoundArc, True Wireless, the elliptical logo, Vistafix, Whisper, WindShield and Xidium are either trademarks or registered trademarks of the Cochlear group of companies.

© Cochlear Limited 2022. All rights reserved.